Artwork and Signature File for:

MAN-05595-702 “ADDENDUM, SV DX/RT WRKSTN USER GUIDE (FOR CENOVA 3.0.1) ITALIAN”

Artwork consists of:

- Two (2) 8 ½ inch x 11 inch sheet(s) attached.

REV AUTHORED BY	DATE		 SIGNATURES ON FILE	
A.PRISCSAK	11/11/2019			
REV DRAFTED BY	DATE	TITLE	DOCUMENT NUMBER	REV
A.PRISCSAK	11/11/2019		AW, ADDENDUM, SV DX/RT WRKSTN USER GUIDE (FOR CENOVA 3.0.1) ITALIAN	AW-17829-702
PROPRIETARY INFORMATION: The content of this document is the exclusive property of Hologic and may not, without prior written permission of Hologic, be reproduced, copied or used for any purpose whatsoever.		ARTWORK	SIZE A	SHEET 1 OF 1

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CAD Mark Info / R2 LesionMetrics™		Hologic Imaging Biomarkers	
Quantra			
	Display	Abbreviation	Name
1	<input type="checkbox"/>	Vd	Volume of dense tissue
2	<input type="checkbox"/>	Vb	Volume of breast
3	<input type="checkbox"/>	Vbd	Volumetric breast density
4	<input type="checkbox"/>	Vd-score	Volume of dense tissue reference score
5	<input type="checkbox"/>	Vbd-score	Volumetric breast density reference score
6	<input type="checkbox"/>	Abd	Area breast density
7	<input type="checkbox"/>	qDC	Quantra Breast Density Category - numerical
8	<input checked="" type="checkbox"/>	QDC	Quantra Breast Density Category

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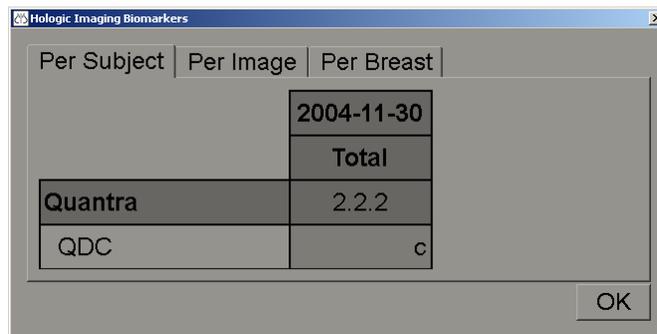
A screenshot of a window titled 'Hologic Imaging Biomarkers'. It has three tabs: 'Per Subject', 'Per Image', and 'Per Breast'. The 'Per Subject' tab is active. The window displays a table with the following data:

2004-11-30	
Total	
Quantra	2.2.2
QDC	c

An 'OK' button is located in the bottom right corner of the window.

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