


Artwork and Signature File for:

MAN-02099 “MNL, TRIDENT, QUICK START GUIDE”

Artwork consists of:

- Two (2) 11 inch x 17 inch sheet(s) attached.

REV AUTHORED BY	DATE	HOLOGIC™	 SIGNATURES ON FILE		
J. SHARP	12/31/14				
REV DRAFTED BY	DATE				
J. SHARP	12/31/14				
PROPRIETARY INFORMATION: The content of this document is the exclusive property of Hologic and may not, without prior written permission of Hologic, be reproduced, copied or used for any purpose whatsoever.		TITLE		DOCUMENT NUMBER	REV
		AW, MNL, TRIDENT QUICK START GUIDE		AW-05674	003
		ARTWORK		SIZE A	SHEET 1 OF 1

Quick Start Guide

THE TASKBAR



1. Manage system alarm messages.
2. Access screens for Admin functions, logging out, editing user settings, and printing the patient list.
3. View job information and manage queues for the destination devices.
4. View system messages, clear alarms and faults, set system defaults, and view system information.

SYSTEM SHUTDOWN

1. Select the Log out button in the Select Patient screen.
2. Select the Shutdown button in the Log In screen.

SYSTEM CARE

General Cleaning Instructions (Refer to the Instructions for Use for full details.)

1. Apply a cleaning solution to a lint-free cloth. Do not spray or flow the liquid on any part of the unit.
2. Wipe the unit with the cloth.
3. Clean the Keyboard surfaces with a CRT wipe.

SYSTEM CALIBRATION

When the system displays a message to perform the calibration procedure, follow the on-screen prompts to complete the procedure.

HOLOGIC®

Corporate Headquarters

35 Crosby Drive,
Bedford, MA 01730-1401 USA
Tel: +1.781.999.7300
Sales: +1.781.999.7453
Fax: +1.781.280.0668
www.hologic.com

EC REP Europe
(EU Representative)

Hologic NV
Leuvensesteenweg 250A
1800 Vilvoorde, Belgium
Tel: +32.2.711.4680
Fax: +32.2.725.2087

Manufacturer

36-37 Apple Ridge Road
Danbury, CT 06810 USA

Refer to the corporate website for more facilities worldwide.


© Copyright Hologic 2014. All rights reserved. Printed in USA. This manual was originally written in English.

Hologic and the Hologic Logo are trademarks or registered trademarks of Hologic, Inc. Other trademarks registered or used by Hologic and its divisions and subsidiaries in the United States and other countries include: Affirm, ATEC, C-View, Dimensions, DSM, Eviva, FAST Paddle, Hologic Connect, HTC, M-IV, MultiCare, Selenia, Smart Paddle, StereoLoc, TechMate, and Trident. Microsoft and Windows are trademarks or registered trademarks of Microsoft Corporation in the United States and other countries. Any other product and company names mentioned herein are the trademarks or registered trademarks of their respective owners.

Quick Start Guide

Trident™

Specimen Radiography System

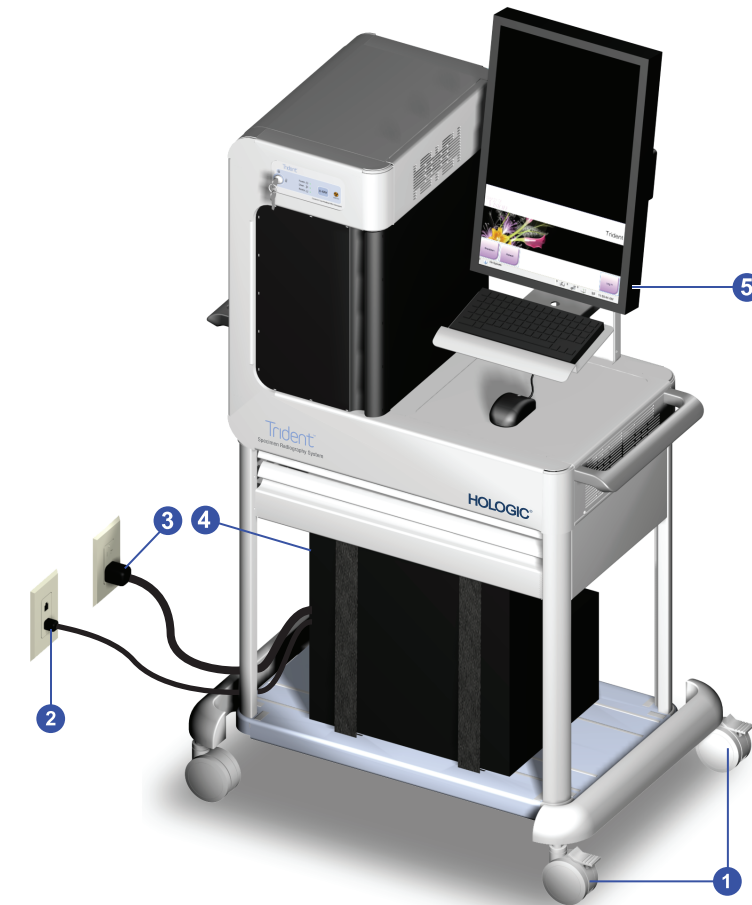

Refer to the User Guide for safety information and complete user instructions.

1. Lock the Casters.
2. Connect to the network.
3. Connect to power.
4. Press the Computer ON button.
5. Select the Log In button.

HOLOGIC®

MAN-02099

B

1. Select an Operator

2. Password

3. Log In

4. Select Patient

5. Patient Name

6. Procedure Selection

7. AEC Mode

8. Add Procedure

C

2.0x

1.5x

1.0x

- Put the specimen in the center of the surface:
 - 2.0x for 2x magnification image
 - 1.5x for 1.5x magnification image
 - 1.0x for Contact image
- Close the door. The Door indicator LED on the Cabinet control panel turns green.

NOTE: If the door is opened during the exposure, the exposure stops and the green Door indicator LED turns off.

D

1. Turn the key to the **Unlocked** position.

2. When the LEDs for Power, Door, and Ready are green, press the **X-RAY** button.

3. To acquire additional images, press the **X-RAY** button. The images are accepted automatically and saved to the local system drive.

4. Turn the key to the **Locked** position when the exam is completed.

Caution: X-Rays Produced When Energized

E

- To delete an acquired image, select the thumbnail image on the left, then select the trash can icon.
- To Archive, Export, or Print an image, select the function, then select from the options in the function screen.
- To enhance the image, select the Tools tab, then select the tool. Below are the tools that are used frequently.

F

- Select the **Comments** tab.
- Select the **Add** button.
- Select an option from the image list.
- Type other comments in the Additional Item area.
- Select the **OK** button.

To add notes to the image which become part of the patient demographics text on the image:

- Select the **Comments** tab.
- Select the **Add** button.
- Select an option from the image list.
- Type other comments in the Additional Item area.
- Select the **OK** button.

Select a tool on the Annotations tab to do the following:

- The **Mag** tool magnifies a section of the image.
- The **Zoom** tools magnifies the whole image.
- The **Window/Level** tools adjust the brightness and contrast of the image.
- The **Enhanced Visualization** button has a slider to adjust the image sharpness.
- The **Camera** button for screen capture.

G

- To end a procedure, select the **Close Patient** button on the Procedure screen.
- The images are transmitted to outputs that were configured during installation.
- Make sure the key is in the **Locked** position.