

Trident™ Specimen Radiography System

Cleaning Instructions for the Trident System



Caution:

Refer to the User Guide for the complete cleaning instructions.



Caution:

To prevent damage:

- Do not spray or flow any liquid on any component of the system.
 - Never use a corrosive solvent or abrasive detergents or polishes.
 - Never expose equipment parts to any steam or temperature sterilization.
 - Never allow any liquids to enter the internal parts of the equipment.
-

How to Clean the Cabinet

General Cleaning with a Mild Solution	Disinfecting
<ol style="list-style-type: none">1. Put diluted dishwashing liquid on a lint-free cloth or pad.2. Wipe the unit or surface with the cloth or pad.	<p>If you use any of the following solutions, finish the cleaning with the steps specified in the section, "Cleaning with a Mild Solution".</p> <ul style="list-style-type: none">• 10% chlorine bleach and water with one part commercial chlorine bleach• Commercial isopropyl alcohol solution (not diluted.)• 3% maximum concentration of hydrogen peroxide solution

How to Clean the Display

- | |
|---|
| <ul style="list-style-type: none">• Use a lint-free cloth or pad. Microfiber cloths, available at most camera stores, are recommended.• Use a commercial cleaning solution for LCD displays.<ul style="list-style-type: none">• Make sure that the solution does not have strong chemicals or abrasives.• Make sure that you follow the directions that the manufacturer of the cleaning solution has provided. |
|---|

How to Remove the Keyboard Cover (if Equipped)



Note

Remove the keyboard cover from the keyboard before cleaning the cover.

- | |
|--|
| <ol style="list-style-type: none">1. Grasp the outer frame of the cover by one corner and lift the frame and cover sheath off in <i>one</i> piece. Do not remove the silicone sheath from its frame unless you are replacing the sheath itself.2. To replace the cover, locate and align the notch on the upper right corner of the cover frame to the upper right corner of the keyboard. See following figure. Press cover firmly back onto keyboard. |
|--|

Trident™ Specimen Radiography System

Cleaning Instructions for the Trident System



Figure 1: Replacing Keyboard Cover

How to Clean Keyboard Cover (if Equipped)

1. Remove the cover from the keyboard.
2. Clean the silicone cover using these cleaning/disinfectant materials:
 - alcohol-based cleaning products
 - chlorine bleach disinfectant wipes products
 - 10% bleach solution
 - 10% hydrogen peroxide
 - disinfectant wipes
 - soap and water

Do not use the following fluids or procedures to clean the keyboard:

- petroleum-based cleaning fluids, such as gasoline (petrol), turpentine, or oil
- acetone
- cleaning fluid with over 10% bleach
- temperature sterilization
- abrasive scrubbing pads

To order a new keyboard cover, go to: www.cherrycorp.com

How to Clean the Keyboard

Clean the key surfaces with a clean cloth. If material falls between the keys, clean the keyboard with a vacuum. If liquids enter the keyboard, or fluids are spilled onto the keyboard, call the Hologic Technical Support for guidance or replacement.

If the keyboard comes in contact with body fluids, the keyboard requires disinfecting. Use a commercially available product or alcohol for this purpose. Do not use bleach or hydrogen peroxide on the keyboard itself.