

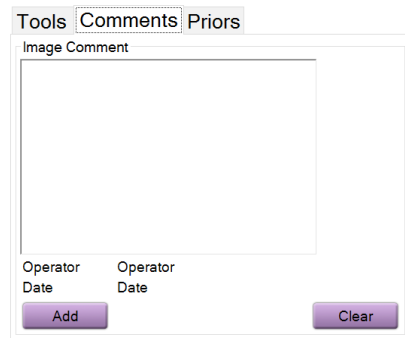
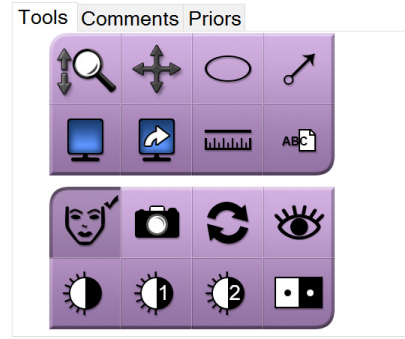



10 Review the image.

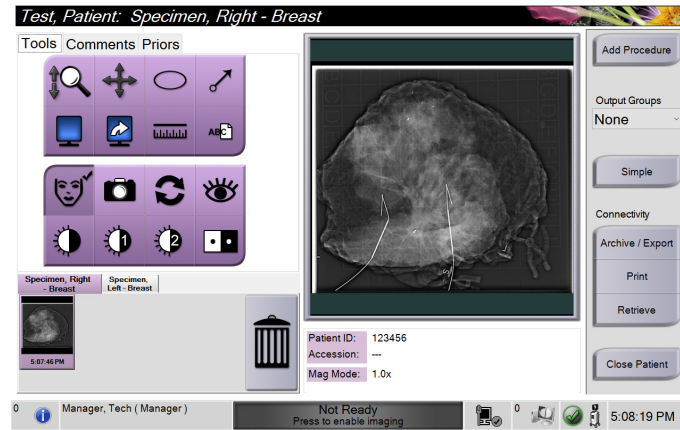



11 Archive or export the images, then close the patient.

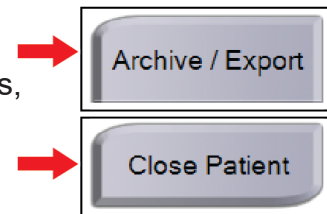

12 Remove the specimen tray from the imaging cabinet. Follow facility policy for specimen handling.

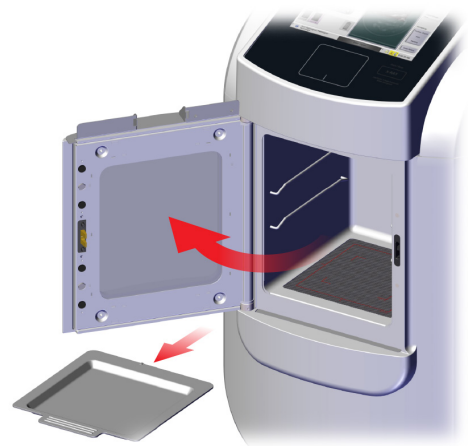

13 Turn off the system.

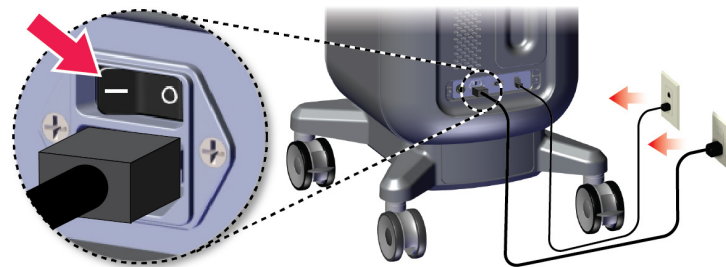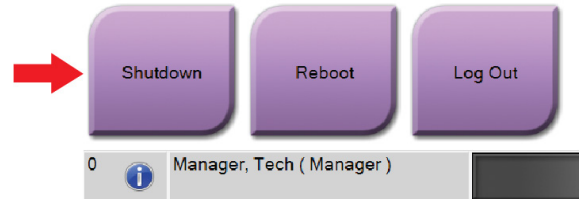

Faxitron™ Trident® HD

Specimen Radiography System

Quick Reference Guide



- 1 Image Display Monitor
- 2 Control Display with Integrated Touch Pad and X-RAY Activation Button
- 3 Imaging Cabinet

Refer to the User Guide for safety information and complete instructions.


1 Move the console to the correct location.



2 Lock the casters.



3 Connect the system to power and the network.

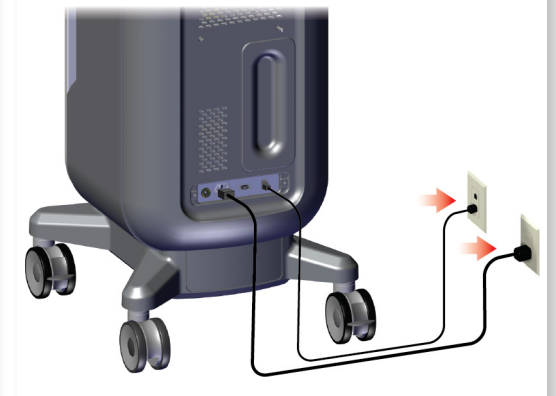

Hologic Inc.
36 Apple Ridge Road
Danbury, CT 06810 USA
Tel: +1.877.371.4372

EC REP Hologic Ltd.
Heron House Oaks Business Park
Crewe Road
Wythenshawe, Manchester
M23 9HZ, UK
Tel: +44 (0)161 946 2206

Brazilian Imex Medical Group do Brasil
Contact: Rua das Embaúbas, 601- Fazenda
Santo Antônio
São José /SC - Brasil - 88104-561
+55 48 3251-8800
www.imexmedicalgroup.com.br



HOLOGIC®

Refer to the corporate website for more facilities worldwide.
© Copyright Hologic 2018 - 2019. All rights reserved. Printed in USA. This manual was originally written in English.


Hologic, Faxitron, Trident and the Hologic Logo are trademarks or registered trademarks of Hologic, Inc. Any other product and company names mentioned herein are the trademarks or registered trademarks of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patents.



4 Start the system and log in.

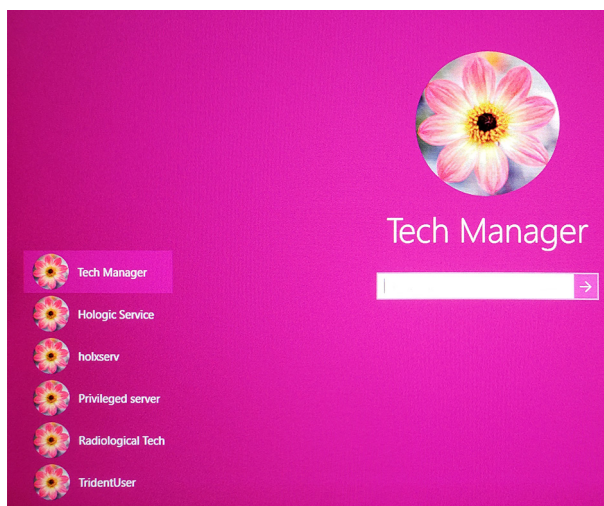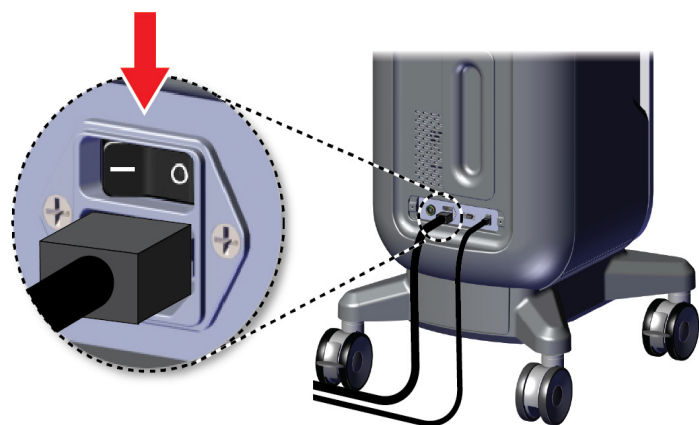


5 Select the patient and procedure.

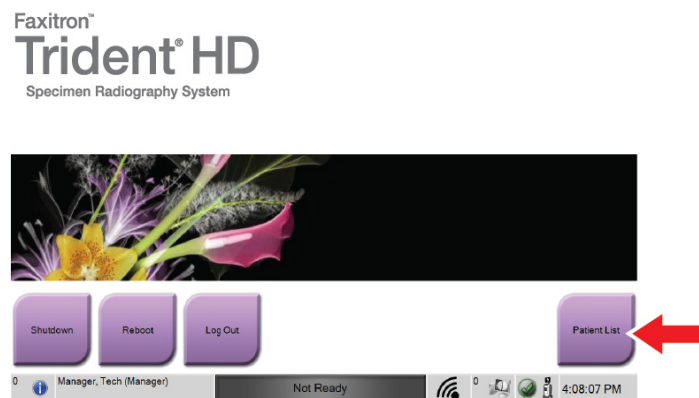


Name	Date of Birth	Exam
Patient*Test	2/3/1979	Specimen, Rig...
Test*Test Patient	3/4/1972	Specimen, Un...
Patient*Patient Test	4/5/1957	Specimen, Left...
Test*Patient	12/17/1972	Specimen, Rig...

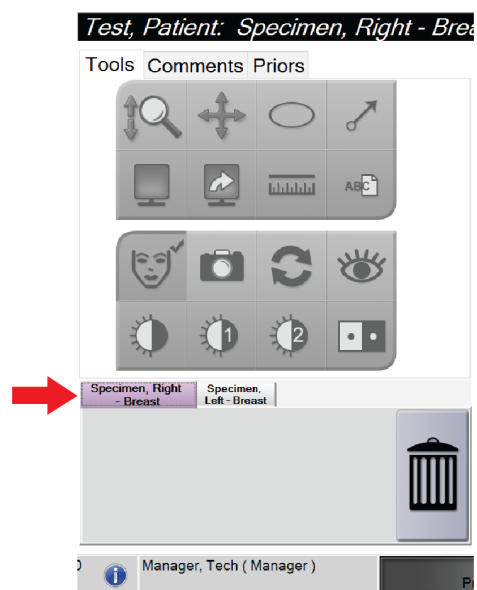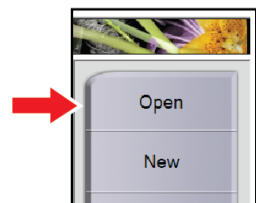


6 Prepare the specimen per facility policy and center on the specimen tray.

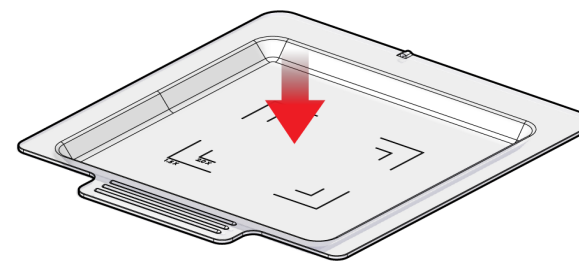

7 Position the specimen tray in the imaging cabinet for the required magnification level.

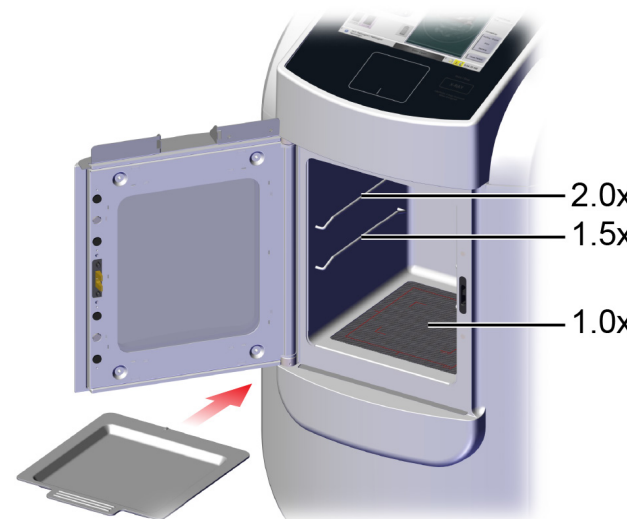

8 Close the door.

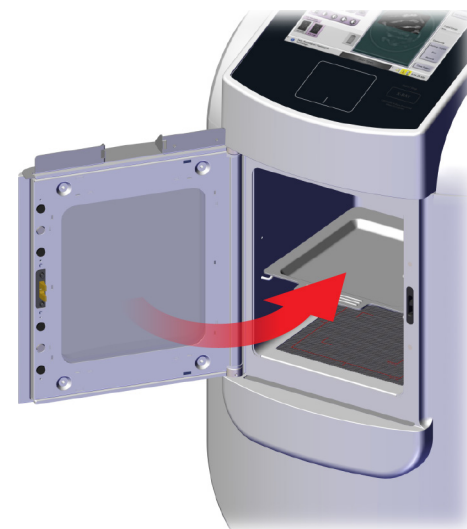

9 Acquire the image.

