

Faxitron[™] OR

Specimen Radiography System

QUICK START GUIDE

1. POWER UP

- Lock back wheels before proceeding to Power Up
- Ensure the power switch on the back of the Faxitron OR* cabinet is set to the “**on**” position
- Press gray **on/off** button until audible beep is heard

2. START SOFTWARE

- Verify there is nothing in the Faxitron OR x-ray chamber
- **Vision** software will automatically start and prompt for the password
- Enter the password, default is “**faxitron**”

3. CALIBRATE

- You will be prompted to calibrate once you log in. Press the “**Calibrate**” button to proceed

4. PATIENT DATA

- Start procedure
- Select from Worklist
- Locate patient’s name and double click it
- Enter any missing required fields such as Laterality
- Accept

5. X-RAY IMAGING

- Place specimen(s) in the Faxitron OR x-ray chamber - use magnification tray if desired
- Press the green **x-ray** button to activate x-rays - image will appear after a few seconds

6. IMAGE MANIPULATION

- Hold down the **left mouse button** and drag the mouse to change brightness and contrast for window/level function
- **Adjust zoom** to enlarge the image for viewing

7. STORING TO PACS

- Choose **PACS Store** button on left side of screen

8. PRINTING

- Choose **PACS Print** button on left side of screen

9. SHUT DOWN

- Select the **System Shutdown** button on the left side and then click **OK**
- An audible beep is heard when the system has completed shutdown

*If you experience a situation where the following occurs; system indicates it is on battery when plugged in, system beeps with no inputs, system turns off unexpectedly, please ensure power switch near the power input cable is on.

www.faxitron.com/support | servicedept-faxitron@hologic.com | +1 520.399.8139

Faxitron OR system also known as BioVision™ System.

05-5765-00 Rev. 031 (6/19) Hologic Inc. ©2019 All rights reserved. Hologic, BioVision, Faxitron and The Science of Sure are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and other markets and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative.