

InSight[®] Software Release Notes

1 InSight 5.2.2

Note

- Hologic only to support use of systems with Instant Off enabled
- 5.2.2 software only runs on Windows 7 Operating System

2 InSight 5.2.1

Resolved Issues

- Merge libraries corrupt on install

3 InSight 5.2

Features

- User interface adjusted for widescreen monitor
- Imaging firmware updated for motherboard revision
- Wi-Fi drivers updated for Windows 7

4 InSight 5.1.1

Resolved Issues

- White halo appears at edge of image circle on InSight 2 systems

5 InSight 5.1

Features

- DICOM tag (0008,0033) populated

Resolved Issues

- DAP data not saved if system is unplugged without ending session
- X-rays disabled after 15 minutes of continuous Cine acquisition
- User can acquire image in Select patient screen
- Study description lost if unit is powered off during acquisition
- Study description is not displayed on image Acquisition screen
- DAP report does not round time to nearest second
- Unrecoverable error on InSight 2 systems after 10 to 15 Cine loops are acquired

6 **InSight 5.0.6**

Features

- Include Performed Series Sequence, Patient ID, and exposure dose in MPPS N-SET message

7 **InSight 5.0.5**

Features

- Improve flattening algorithm
- Correct black image appearance in Perform
- Remove message box on startup

8 **InSight 5.0.4**

Features

- Reset imaging board at startup

9 **InSight 5.0.3**

Features

- DICOM images have all post processing applied such as image flipping and sharpening before being sent

10 **InSight 5.0.2**

Features

- Reinitialize detector at startup

11 **InSight 5.0.1**

Features

- Dynamically update detector flattening at startup