

CoV HKU1 Probe**Intended Use**

Analyte Specific Reagent (ASR). Analytical and performance characteristics are not established. This ASR is intended to be used as a component of a laboratory developed test (LDT) used exclusively by laboratories. Laboratories may develop their own LDTs using one or more ASRs and/or general laboratory reagents for *in vitro* diagnostic purposes.

Product Description

The Coronavirus (CoV) HKU1 Probe ASR contains an oligonucleotide designed to detect the ORF 1a gene of CoV HKU1. The oligonucleotide contains a 5' Cal Fluor Orange 560 label and an internal BHQ-1 label.

Materials Provided

CoV HKU1 Probe (Cat. No. PRD-05589)

Component	Quantity	Concentration
Tubes	1 x 0.53 mL	25 µmol/L

Warnings and Precautions

- A. Analytical and performance characteristics are not established.
- B. Use routine laboratory precautions. Wear disposable, powderless gloves, protective eye wear, and laboratory coats when handling specimens and kit reagents. Wash hands thoroughly after handling reagents.
- C. Dispose of all materials that have come in contact with reagents according to local, state, and federal regulations.

Quality Control

This analyte specific reagent was manufactured and released in accordance with the Hologic quality control and quality assurance procedures.

Storage and Handling

Upon receipt, store material at -15 to -85 °C. Multiple freeze/thaw cycles should be minimized by appropriate aliquoting and should not exceed 3 freeze/thaw cycles.

***In silico* Binding Affinity**

The CoV HKU1 probe covers 100% of the aligned Human Coronavirus HKU1 sequences available in the NCBI Database*.

Additionally, the CoV HKU1 probe did not show binding affinity for the organisms indicated in the table below or to human DNA.

Adenovirus-Type 1	Flu B	<i>Mycoplasma genitalium</i>
Bocavirus	<i>Haemophilus influenzae</i>	<i>Mycoplasma hominis</i>
<i>Bordetella bronchiseptica</i>	Human DNA	<i>Mycoplasma pneumoniae</i>
<i>Bordetella holmseii</i>	Human metapneumovirus	<i>Neisseria elongata</i>
<i>Bordetella parapertussis</i>	Human Parainfluenza virus 1	<i>Neisseria meningitidis</i>
<i>Bordetella pertussis</i>	Human Parainfluenza virus 2	<i>Pseudomonas aeruginosa</i>
<i>Candida albicans</i>	Human Parainfluenza virus 3	Rhinovirus
<i>Chlamydia trachomatis</i>	Human Parainfluenza virus 4	RSV
CMV	<i>Lactobacillus acidophilus</i>	<i>Staphylococcus aureus</i>
<i>Corynebacterium diphtheriae</i>	<i>Legionella micdadei</i>	<i>Staphylococcus epidermidis</i>
<i>E. coli</i>	Measles Virus	<i>Streptococcus pneumoniae</i>
EBV	<i>Moraxella catarrhalis</i>	<i>Streptococcus pyogenes</i>
Echovirus	Mumps	<i>Streptococcus salivarius</i>
Flu A	<i>Mycobacterium tuberculosis</i>	

*As of August 2019



Hologic, Inc.
10210 Genetic Center Drive
San Diego, CA 92121 USA

Customer Support: +1 800 442 9892
customersupport@hologic.com

Technical Support: +1 888 484 4747
molecularsupport@hologic.com

For more contact information visit www.hologic.com.

Hologic and associated logos are trademarks or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries.

“Black Hole Quencher,” “BHQ,” and “CAL Fluor” are trademarks of and licensed from Biosearch Technologies, Inc., Novato, CA. The Black Hole Quencher (BHQ) and CAL Fluor dye technologies are the subject of existing or pending patents and are licensed and sold under agreement with Biosearch Technologies, Inc.

All other trademarks that may appear in this package insert are the property of their respective owners.

This product may be covered by one or more U.S. patents identified at www.hologic.com/patents.

© 2020 Hologic, Inc. All rights reserved.

AW-20482-001 Rev. 001
2020-01