

Selenia Dimensions 1.0.4.2 Release Notes

1. Introduction

This document is provided as an overview of the enhancements in the Selenia Dimensions Version 1.0.4.2.



Note...

This document is not meant to replace the Operator's Manual.

2. QC Section

This software upgrade improves internal system communications. It does not alter and does not affect the performance of the Selenia Dimensions FFDM system.

Radiologic Technologist

This software upgrade requires the evaluation of the following tests by a radiologic technologist according to the corresponding QC sections described in the Selenia QC manual in the technologist section:

- Signal-to-noise and contrast-to-noise measurements;
- Phantom image evaluation.

Medical Physicist

This software upgrade does not require any evaluation by a medical physicist.

3. Features of Selenia Dimensions 1.0.4.2

Software Enhancements

- No major software enhancements.

Other minor Software Updates:

- Improve reliability of the internal system communication.
- Reduce likelihood of reported faults during the system start-up and after the system reboot.
 - Improve the operating efficiency of the paddle shift motor and grid control motor