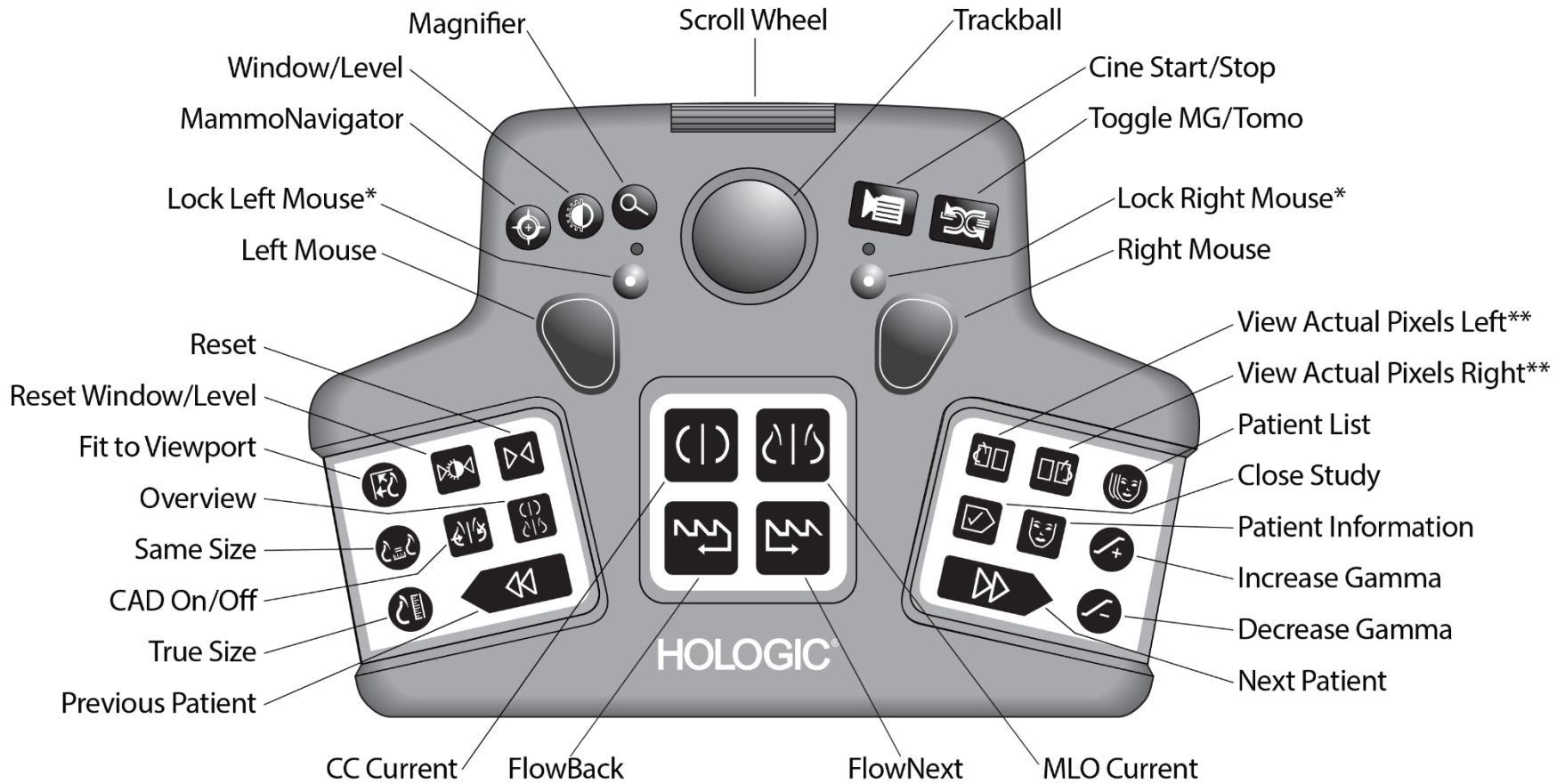


SecurView[®] DX Keypad Quick Reference


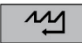
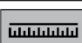

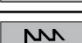
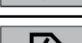
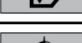
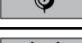


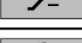

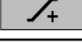





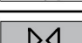
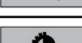
















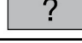
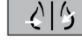


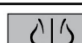
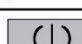
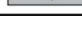

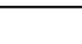

*Allows for locking left and right mouse buttons. When the mouse button is locked in the down position the LED above this button illuminates.

**Viewport(s) must be in single tiling mode.

SecurView®

Breast Imaging Workstation

Tool	Key	Function
	[O] or [Ins]	Overview
	[1] or [End]	FlowBack
	[2] or [Down Arrow]	Measurement
	[3] or [Pg Down]	Patient Information
	[4] or [Left Arrow]	FlowNext
	[5]	Close Study
	[6] or [Right Arrow]	MammoNavigator
	[7], [D], [Home] or [Up Arrow]	View Actual Pixels Left
	[9], [8], [F] or [Page Up]	View Actual Pixels Right
	[F1] or [M]	Decrease Gamma
	[F2]	Reset Window/Level
	[F3] or [N]	Increase Gamma
	[F4]	Display Mouse Cursor Position (if configured)
	[F5]	Fit to Viewport
	[F6]	Cine Start/Stop
	[F7]	Continuous Zoom
	[F7]	Reset Cont. Zoom
	[F8]	Reset
	[F9]	Window / Level
	[F10]	Arrow
	[F11]	Ellipse
	[F12]	Freehand

Tool	Key	Function
	[A]	Magnifier
	[E]	Export Images
	[G]	Patient List
	[H]	Display full DICOM header content
	[I]	Invert All Images
	[J]	Toggle MG / Tomo
	[Q]	Exit
	[R]	Synchronize
	[X]	True Size
	[Y]	Same Size
	[Y]	Right Size
	[Z]	Undo Delete Marking
	[?]	Help
	[/]	CAD
	[Delete]	Previous Patient
	[Enter]	Next Patient
	[+]	MLO Current
	[-]	CC Current
	[*]	Display/Hide Toolbar
	[Spacebar]	Tag Tomo Images
	[Backspace]	Delete Selected Marking
	[CTRL] and [t]	Toggle current / prior color indication