



Artwork and Signature File for:

MAN-07586 “App Sync 4.5 Release Notes”


Artwork consists of:

- Nineteen (19) 8 ½ inch x 11 inch sheet(s) attached

REV AUTHORED BY	DATE			 SIGNATURES ON FILE
M. GROSNER	10/1/2020			
REV DRAFTED BY	DATE			
M. GROSNER	10/1/2020			
PROPRIETARY INFORMATION: The content of this document is the exclusive property of Hologic and may not, without prior written permission of Hologic, be reproduced, copied or used for any purpose whatsoever.		TITLE	DOCUMENT NUMBER	REV
		AW, APP SYNC 4.5 RELEASE NOTES	AW-22204	001
		ARTWORK	SIZE A	SHEET 1 OF 1



SecurView®
Breast Imaging Workstation

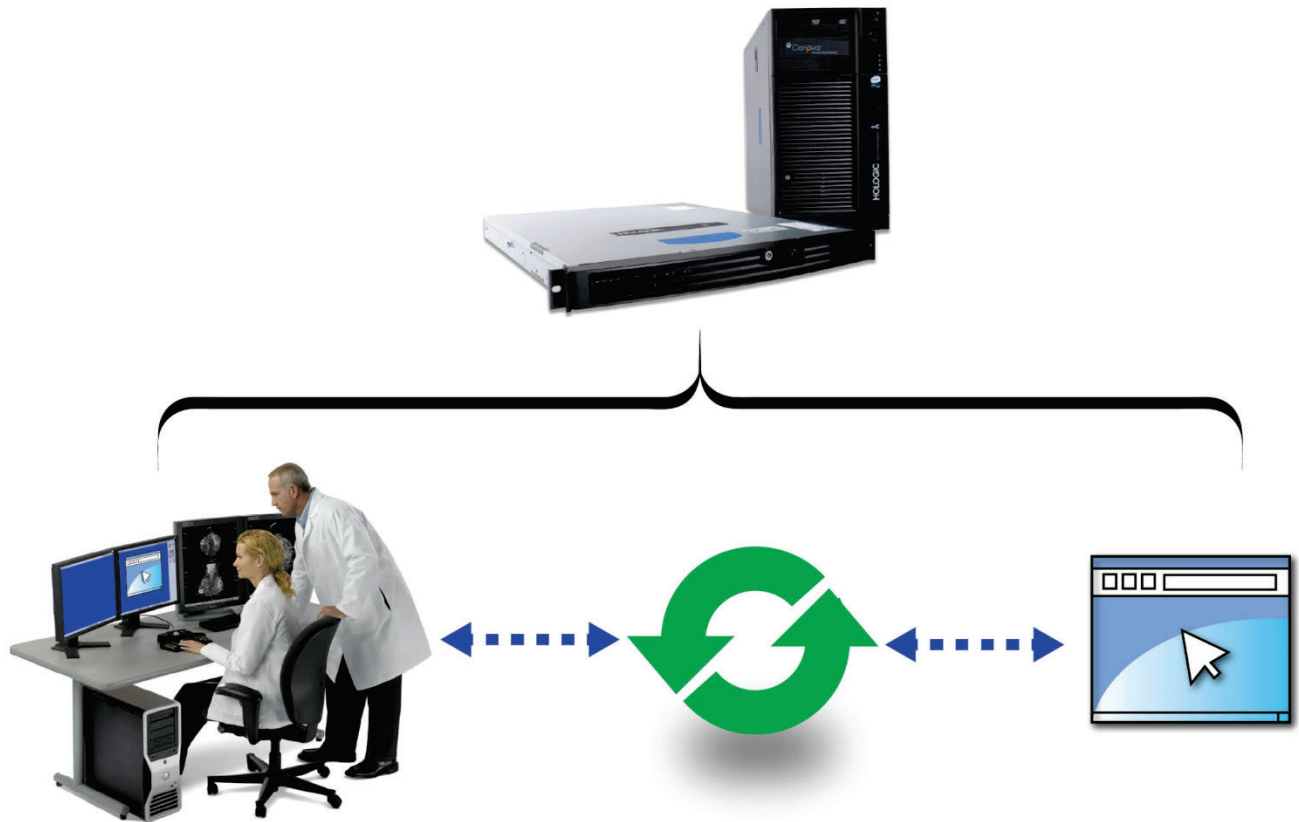

Application Synchronization 4.5
Release Notes
MAN-07586 Revision 001

Application Synchronization

Between Hologic and Third-Party Applications

Release Notes

For Software Version 4.5

Part Number MAN-07586

Revision 001

October 2020

Technical Support

USA: +1.877.371.4372

Asia: +852 37487700

Europe: +32 2 711 4690

Australia: +1 800 264 073

All Other: +1 781 999 7750

Email: BreastHealth.Support@hologic.com

© 2020 Hologic, Inc. Printed in the USA. This manual was originally written in English.

Hologic, SecurView, MultiView, Selenia, Aegis, Unifi, and associated logos are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patent-information.



Hologic Inc.
36 Apple Ridge Road
Danbury, CT 06810 USA
1.800.447.1856
www.hologic.com

Asia Pacific Hologic Hong Kong, Inc.
7th Floor, Biotech Centre 2
No. 11 Science Park West Avenue
Hong Kong Science Park
Shatin, New Territories
Hong Kong

Australia /
New Zealand Hologic (Australia) Pty Ltd.
Suite 402, Level 3
2 Lyon Park Road
Macquarie Park NSW 2113
Australia

HOLOGIC®

Table of Contents

- 1: Introduction** **1**

- 2: Notes for Version 4.5** **2**
 - 2.1 Enhancements in Version 4.5 2
 - 2.2 Problems Corrected in Version 4.5 2
 - 2.3 Known Issues in Version 4.5 2
 - 2.4 Supported Applications in Version 4.5 2

- 3: Configuration Information** **16**
 - 3.1 Using Synchronization on SecurView 16
 - 3.2 Using Synchronization on MultiView 16
 - 3.3 Using Synchronization on Unifi Workspace 16

Introduction

The Release Notes provide an overview of the most recent release of the Application Synchronization option for the SecurView® DX diagnostic workstation, MultiView™ web client or workstation, and Unifi™ Workspace. This document provides a listing of:

- Enhancements
- Supported Applications
- Configuration Information

When an application supports synchronization protocols with a Hologic workstation, a patient is selected only once, and both the Hologic workstation and the other application will synchronize to the same patient automatically. The interface can be unidirectional or bidirectional, depending on the capabilities of the third-party application.

The third-party application can run on a separate computer or as a thin client on the Hologic workstation. Section [2.4. Supported Applications in Version 4.5](#) provides a list of currently supported third-party applications.

The table below lists SecurView and Unifi Workspace versions supported by different versions of Application Synchronization.

Application Synchronization Versions	SecurView Versions	Unifi Workspace
4.5	10.3 and later (running on Microsoft Windows 10)	1.1
4.4		1.0
4.3		
2.1 to 4.2	7.3 to 10.3 (running on Microsoft Windows 7)	

Application Synchronization versions 3.0 and later support MultiView and Aegis.

Notes for Version 4.5

2.1 Enhancements in Version 4.5

2.1.1 Updated Application Type

Application Synchronization 4.5 includes updates to the Hologic Unifi Workspace application type supported message types.

- Login: unidirectional, Unifi Workspace sends
- Logout: bi-directional
- OpenPatient: Additional message content allows Application Synchronization to apply study filters to messages Unifi Workspace sends

2.2 Problems Corrected in Version 4.5

2.2.1 Unifi Workspace driven workflow with Epic Radiant RIS

Now that Unifi Workspace supports sending Login messages to Application Synchronization, Unifi Workspace driven workflow is possible with Epic Radiant RIS.

2.2.2 Unifi Workspace driven workflow with GE Centricity RIS-IC

Now that Unifi Workspace supports sending Login messages to Application Synchronization, Unifi Workspace driven workflow is possible with GE Centricity RIS-IC.

2.3 Known Issues in Version 4.5

2.3.1 GE Centricity RIS-IC driven workflow is not supported

Application Synchronization cannot receive OpenPatient messages from GE Centricity RIS-IC 10.7 and later due to stricter user context switching. GE Centricity RIS-IC requires Application Synchronization to reinitialize listening for OpenPatient messages each time a user logs into GE Centricity RIS-IC, but currently Application Synchronization does not have a way to detect GE Centricity RIS-IC user login.

2.4 Supported Applications in Version 4.5

Hologic Partners are applications that have implemented one of the three available Hologic partner interfaces (Command Line Proxy, SharedFile, or TCP/IP) and use the corresponding general application type.

Each non-Hologic partner interface has its own specific application type, with a default configuration that matches the implementation.

2.4.1 Hologic Proxy Partner

A Hologic Proxy Partner is one that uses the Hologic Command Line Proxy interface.

2.4.2 Hologic SharedFile Partner

A Hologic SharedFile Partner is one that uses the Hologic XML SharedFile interface.

2.4.3 Hologic TCP/IP Partner

A Hologic TCP/IP Partner is one that has implemented the Hologic TCP/IP AS Adapter interface.

- 3M Radiology Link Software (Hologic SharedFile Partner)
 - 3M Radiology Link Software version 2.0.0.15 or higher for VoiceScript Recorder and ChartScriptMD Document Builder Platform 7, feature pack 2 with 3rd service pack or higher
 - 3M Radiology Link Software, VoiceScript, and ChartScriptMD can run only on a separate computer.
- Agfa TalkStation (Custom: COM API)
 - Agfa TalkStation version 3.2 or higher
 - TalkStation can be run on the SecurView workstation or on a separate computer.
- Allscripts (Eclipsys) Sunrise Radiology (Hologic TCP/IP Partner)
 - Sunrise Radiology version 5.6 w/SP3 or higher
 - Sunrise Radiology can be run on the SecurView workstation or on a separate computer.
- Amicas Vision Series v4.0-5.1 (Custom: XML file)
 - Amicas Vision Series PACS version 4.0 - 5.1
 - Vision Series can run only on a separate computer.
- Amicas Vision Series v5.5 (Custom: XML file)
 - Amicas Vision Series PACS version 5.5 or higher
 - Vision Series can run only on a separate computer.
- Avreo InterWorks PACS (Hologic TCP/IP Partner)
 - Avreo InterWorks PACS version 4.6 or higher
 - Avreo InterWorks can run only on a separate computer.
- Candelis ImageGrid RIS (Hologic SharedFile Partner)
 - Candelis ImageGrid RIS version 2.0.3 or higher
 - ImageGrid RIS can run only on a separate computer.
- Carestream RIS (Hologic TCP/IP Partner)
 - Carestream RIS (2010) version 10.x, 11.x
 - Carestream RIS can be run on the SecurView workstation or on a separate computer.

- CDI Multifondacs (Hologic TCP/IP Partner)
 - CDI Multifondacs MF-SIS version 4.3.2 or higher
 - CDI Multifondacs can be run on the SecurView workstation or on a separate computer.
- Clario zVision Worklist (Hologic TCP/IP Partner)
 - Clario zVision Worklist version 3.5.7.11523.20 or higher
 - Clario zVision Worklist can run only on a separate computer.
- Commit; Screening MIS (Hologic TCP/IP Partner)
 - Commit; Screening MIS version 2.2.4 or higher
 - Commit; Screening MIS can be run on the SecurView workstation or on a separate computer.
- Compressus VM2 (Hologic TCP/IP Partner)
 - Compressus VM2 version 2.0.0 or higher
 - VM2 can be run on the SecurView workstation or a separate computer.
- Crescendo CSCIntegration (Hologic SharedFile Partner)
 - Crescendo CSCIntegration version 3.8.2134.0 or higher for DigiDictate-IP version 3.6.287.12 or higher
 - CSCIntegration and DigiDictate-IP can run only on a separate computer.
- Dataworxs AudioWav Mic (Hologic SharedFile Partner)
 - Dataworxs AudioWav Mic version 2.00.017 or higher
 - AudioWav Mic can be run on the SecurView workstation or on a separate computer.
- Dolby FusionDictate (Custom: XML file)
 - Dolby FusionDictate version 1.4.28 (a.k.a. version 8) or higher
 - FusionDictate can be run on the SecurView workstation or on a separate computer.
- Dolby FusionExpert (Custom: XML file)
 - Dolby FusionExpert version 5.1.30 or higher
 - FusionExpert can be run on the SecurView workstation (with or without front-end speech recognition module) or on a separate computer.
 - Fusion Text version 8.6.13 or higher is required for Windows 10 (Pro, Enterprise, Enterprise LTSB).
 - The current released version is Fusion Text 9.0.7.

- DR Systems Send ID XML Interface (Hologic SharedFile Partner)
 - DR Systems Send ID XML Interface version 8.2 or higher and AppLink version 10.0 or higher
 - DR Systems web application can be run on the SecurView workstation or on a separate computer.
- Epic Radiant RIS (Custom: COM API)
 - Epic Radiant RIS version Spring 2006 or higher
 - Radiant RIS can be run on the SecurView workstation or on a separate computer.
 - Version 2015 or higher is required for Windows 10 with IE 11.
 - The current released version is Epic Feb 2019.
- Euromed MedSpeech (Hologic TCP/IP Partner)
 - Euromed MedSpeech version 3.0 Rev 10 or higher
 - MedSpeech can be run on the SecurView workstation or on a separate computer.
- G2 Speech MediSpeech (Custom: COM API)
 - G2 Speech MediSpeech version 5.0 or higher
 - MediSpeech client can be run on the SecurView workstation or on a separate computer.
- GE Centricity PACS (Custom: COM API)
 - GE Centricity PACS version 3.2
 - Centricity PACS can run only on a separate computer.
- GE Centricity RIS-IC (Custom: COM API)
 - GE Centricity RIS-IC version 10.5 SP25 or higher
 - Centricity RIS-IC client can be run on the SecurView workstation or on a separate computer.
- GE/IDX Imagecast (Custom: COM API)
 - GE/IDX Imagecast (a.k.a. Centricity RIS-IC) version 10.3.9 or higher
 - Imagecast client can be run on the SecurView workstation or a separate computer.
- HSS CRIS (Hologic TCP/IP Partner)
 - HSS CRIS version 1.06.12n or higher
 - HSS CRIS can be run on the SecurView workstation (without Speech Recognition or Digital Dictation) or on a separate computer (with Speech Recognition or Digital Dictation).

- Ikonopedia Breast Imaging Suite (Hologic SharedFile Partner)
 - Ikonopedia Breast Imaging Suite version 1.0 or higher
 - Breast Imaging Suite can be run on the SecurView workstation or on a separate computer.
- Infinitt G3 PACS (Hologic TCP/IP Partner)
 - Infinitt G3 PACS version 3.0.11.4 BN2 or higher
 - Infinitt G3 PACS web viewer can be run on the SecurView workstation or on a separate computer.
- Insight OmniCare (Hologic SharedFile Partner)
 - Insight OmniCare version 7.5.2 or higher
 - OmniCare can be run on the SecurView workstation or on a separate computer.
- Intelerad IntelViewer (Hologic SharedFile Partner)
 - Intelerad IntelViewer version 4.6.1 or higher
 - IntelViewer can run only on a separate computer.
- Lanier Serial (Custom: Serial)
 - MedQuist/Lanier VoiceWriter Digital Stations
 - Application runs on separate hardware, with a serial cable connecting the SecurView workstation and the Lanier station.
 - MedQuist DocQvoice Workstation (formerly DocQment Ovation) version 1.1.5 or higher
 - DocQvoice can be run on the SecurView workstation or on a separate computer.
- M*Modal Fluency for Imaging Reporting (Hologic SharedFile Partner)
 - M*Modal Fluency for Imaging Reporting version 3.3.0 or higher
 - Fluency for Imaging Reporting can be run on the SecurView workstation or on a separate computer.
 - Version 4.0 or higher runs on Windows 10 with IE 11.
 - The current released version is 4.9.
- MagView (Hologic SharedFile Partner)
 - MagView version 6.3 or higher
 - MagView can be run on the SecurView workstation or on a separate computer.
 - MagView 6.x or higher runs on Windows 10.
 - The current released version is MagView Luminary (8.x).



Note

This interface is preferred to use instead of the MagView TCP/IP interface.

- MagView (Custom: TCP/IP)
 - MagView version 5.3.500 or higher
 - MagView can be run on the SecurView workstation or on a separate computer.
 - KVB MammaSoft (Germany) version 1.0 or higher
 - MammaSoft can be run on the SecurView workstation or on a separate computer.
- MaSc RIS (Custom: HTTP)
 - KVWL MaSc RIS (Germany) version 2.0 or higher
 - MaSc RIS can be run on the SecurView workstation or on a separate computer.
- Medicalis Professional (Hologic TCP/IP Partner)
 - Medicalis Professional version 2.4.2.15 or higher
 - Professional can be run on the SecurView workstation or on a separate computer.
- MedOutlook RIS (Custom: TCP/IP)
 - Keymind Computing MedOutlook RIS (Norway) version 1.0.0.1 or higher
 - MedOutlook can run only on a separate computer.
- MedQ Dictation (Hologic SharedFile Partner)
 - MedQ Dictation version 5.0.x or higher
 - Dictation can be run on the SecurView workstation or on a separate computer.
- MedQuist SpeechQ (Custom: COM API)
 - MedQuist SpeechQ for Radiology version 1.2 or higher
 - SpeechQ can be run on the SecurView workstation or on a separate computer.
- MRS (Custom: TCP/IP)
 - MRS Mammography Reporting System version 6.6.46 or higher
 - MRS Aspen version 8.0 or higher
 - MRS client can be run on the SecurView workstation or on a separate computer.
 - MRS version 7.1 or higher is required for Windows 10 (Pro).
 - MRS Aspen runs on Windows 10 (Pro).
 - The current released versions are MRS 7.3.025, Aspen Breast 8.0.2548.

- MultiView or Aegis (Custom: TCP/IP)
 - Hologic MultiView version 4.0 or higher and Hologic Aegis version 3.2
 - MultiView can be run on the SecurView workstation or on a separate computer.
 - The current released version is 4.1.
- NBSS British Screening RIS (Hologic Proxy Partner)
 - NHSBSP NBSS British Screening RIS can be run on the SecurView workstation or on a separate computer.
- peerVue qiVue (Hologic TCP/IP Partner)
 - peerVue qiVue version 6.3.3 or higher and QICS version 7.0.2 or higher
 - qiVue can run only on a separate computer.
- PenRad (Custom: XML file or Hologic SharedFile Partner)
 - PenRad version 5.1.84 or higher
 - PenRad can be run on the SecurView workstation or on a separate computer.
 - Version 5.7.76 or higher runs on Windows 10 (Pro) with IE 10 or 11, Edge, Chrome.
 - The current released version is Series 7.



Note

This interface is preferred to use instead of the serial interface.

- Philips iSite PACS (Custom: HTTP plug-in)
 - Philips iSite PACS plug-in for iSite Radiology Client version 3.6 and 4.1
 - Philips IntelliSpace version 4.4
 - iSite PACS can run only on a separate computer.
- PowerScribe or EWS (Custom: XML file)
 - Nuance/Dictaphone PowerScribe version 4.6 or higher with PACSBridge 4.7 1009
 - Nuance Enterprise Workstation (EWS) version 3.0 with PACSBridge 6.5.1
 - PowerScribe or EWS client can be run on the SecurView workstation or on a separate computer.

- PowerScribe bi-directional (Custom: XML file)
 - Nuance/Dictaphone PowerScribe version 4.7 or higher with PACSBridge 4.7 1021 supports bi-directional open patient
 - PowerScribe client can be run on the SecurView workstation or on a separate computer.
 - Nuance RadWhere version 3.0.18.14 or higher
 - RadWhere client can be run on the SecurView workstation or a separate computer.
 - Nuance PowerScribe 360 version 1.0 or higher
 - PowerScribe 360 (including Dragon SDK client edition 10) can be run on the SecurView workstation or on a separate computer.
 - Version 4.0 or higher runs on Windows 10.
 - The current released version is 4.0.
- QDoc StudyViewer (Custom: COM API)
 - Agfa QDoc StudyViewer version 5.5.1 or higher
 - StudyViewer client can be run on the SecurView workstation or on a separate computer.
- RadNet VDI (Custom: COM API)
 - Cerner Millenium RadNet VDI version 2007.18 or higher
 - RadNet VDI can be run on the SecurView Workstation or on a separate computer.
 - Version 2015.01.22 or higher is required for Windows 10.
 - The current released version is 2018.01.
- RadNet VDI prior to 2007.18 (Custom: COM API)
 - Cerner Millenium RadNet VDI versions prior to 2007.18
 - RadNet VDI can be run on the SecurView Workstation or a separate computer.
- ScImage Picom Client (Hologic SharedFile Partner)
 - ScImage Picom Client version 3.03.059 or higher
 - Picom Client can be run on the SecurView workstation or a separate computer.
- SecurView (Custom: TCP/IP)
 - SecurView DX Diagnostic Workstation version 5-0-0 or higher and SecurView RT Technologist Workstation version 6-0-0 or higher

- SecurView Serial Partner (Custom: Serial)
 - Currently, these partners include PenRad, MRS, MagView
 - Third-party application runs on a separate computer, with a serial cable connecting the SecurView workstation and the third-party application computer.
- Selenia® AWS (Custom: TCP/IP)
 - Hologic Selenia Acquisition Workstation version 3.3 or higher
 - Selenia AWS can run only on a separate computer.
- SERGAS SIDI (Hologic TCP/IP Partner)
 - SERGAS SIDI (Spain) version 2.10 or higher
 - SERGAS SIDI can run only on a separate computer.
- SESCOAM Dulcingest RIS (Hologic TCP/IP Partner)
 - SESCOAM Dulcingest RIS version 7.8.3.1 or higher
 - Dulcingest RIS can be run on the SecurView workstation or on a separate computer.
- Softek Illuminate InSight (Hologic SharedFile Partner)
 - Softek Illuminate InSight version 2.12 or higher
 - Illuminate InSight can be run on the SecurView workstation or on a separate computer.
- Spantel Audio Recorder (Custom: COM API)
 - Spantel Audio Recorder version 2.0 or higher
 - Audio Recorder can be run on the SecurView workstation or on a separate computer.
- syngo Workflow MLR (Europe) (Hologic Proxy Partner)
 - Siemens syngo Workflow MLR (Europe) browser version VB30 or higher
 - Siemens syngo Workflow MLR (Europe) can be run on the SecurView workstation or on a separate computer.
- syngo Workflow Portal Radiologist (Custom: HTTP)
 - Siemens syngo Workflow Portal Radiologist (with optional Voice) version 30 or higher
 - Portal Radiologist can be run on the SecurView workstation or on a separate computer.
- syngo Workflow ReadExam (Custom: XML file)
 - Siemens syngo Workflow (formerly Novius RIS) ReadExam (with optional Voice) version 27.1 or higher
 - ReadExam can be run on the SecurView workstation or on a separate computer.

Application Synchronization 4.5 Release Notes Notes for Version 4.5

- Unifi Workspace (Custom: WebAPI)
 - Hologic Unifi Workspace version 1.0 or higher
- WinScribe Internet Author (Hologic SharedFile Partner)
 - WinScribe Internet Author version 3.8 or higher
 - WinScribe Internet Author can be run on the SecurView workstation or on a separate computer.

	Partner Type	Receives				Sends			
		Login	Logout	Open Patient	Update Patient	Login	Logout	Open Patient	Update Patient
Hologic Proxy Partner+	Proxy	-	-	-	-	✓	✓	✓	-
Hologic SharedFile Partner+	SharedFile	✓	✓	✓	✓	✓	✓	✓	✓
Hologic TCP/IP Partner+	TCP/IP	✓	✓	✓	✓	✓	✓	✓	✓
3M Radiology Link Software	SharedFile	✓	✓	✓	-	-	-	-	-
Agfa TalkStation*	COM API	✓	✓	✓	-	-	-	-	-
Allscripts (Eclipsys) Sunrise Radiology	TCP/IP	-	-	-	-	✓	✓	✓	-
Amicas Vision Series (v4.0-5.1, v5.5)	XML file	-	-	-	-	-	-	✓	-
Avreo InterWorks PACS	TCP/IP	-	-	-	-	✓	✓	✓	-
Candelis ImageGrid RIS	SharedFile	-	-	✓	-	-	-	✓	-
Carestream RIS	TCP/IP	-	-	-	-	✓	✓	✓	-
CDI Multifondacs	TCP/IP	-	-	-	-	-	-	✓	-
Clario zVision Worklist	TCP/IP	-	-	-	-	-	-	✓	-

Application Synchronization 4.5 Release Notes
Notes for Version 4.5

	Partner Type	Receives				Sends			
		Login	Logout	Open Patient	Update Patient	Login	Logout	Open Patient	Update Patient
Commit; Screening MIS	TCP/IP	-	-	-	-	-	-	✓	-
Compressus VM2	TCP/IP	-	-	-	-	✓	✓	✓	✓
Crescendo CSCIntegration	SharedFile	✓	-	✓	-	-	-	-	-
Dataworx AudioWav Mic	SharedFile	-	-	✓	-	-	-	-	-
Dolbey FusionDictate	XML file	-	-	✓	-	-	-	-	-
Dolbey FusionExpert	XML file	-	-	✓	-	-	-	-	-
DR Systems Send ID	SharedFile	-	-	-	-	-	-	✓	-
Epic Radiant RIS*	COM API	✓	✓	✓	-	-	-	✓	✓
Euromed MedSpeech	TCP/IP	-	-	✓	-	-	-	-	-
G2 Speech MediSpeech*	COM API	✓	✓	✓	-	-	-	-	-
GE Centricity PACS	COM API	-	-	-	-	-	-	✓	-
GE Centricity RIS-IC*	COM API	✓	✓	✓	-	-	-	✓	-
GE/IDX Imagecast*	COM API	✓	✓	✓	-	-	-	-	-
HSS CRIS	TCP/IP	✓	✓	✓	-	✓	✓	✓	-
Ikonopedia Breast Imaging Suite	SharedFile	-	-	✓	-	-	-	✓	-
Infinitt G3 PACS	TCP/IP	✓	✓	✓	-	✓	✓	✓	-
Insight OmniCare	SharedFile	-	-	-	-	-	-	✓	-

Application Synchronization 4.5 Release Notes
Notes for Version 4.5

	Partner Type	Receives				Sends			
		Login	Logout	Open Patient	Update Patient	Login	Logout	Open Patient	Update Patient
Intelrad InteleViewer	SharedFile	-	-	-	-	✓	✓	✓	-
Lanier Serial	Serial	-	-	✓	-	-	-	-	-
MedQuist DocQvoice	Serial	-	-	✓	-	-	-	-	-
M*Modal Fluency for Imaging Reporting	SharedFile	✓	✓	✓	-	-	-	✓	✓
MagView	SharedFile	✓	-	✓	-	-	-	✓	✓
MagView	Cust TCP/IP	-	-	✓	-	-	-	✓	-
MammaSoft	Cust Proxy	-	-	-	-	-	-	✓	-
MaSc RIS	HTTP	-	-	-	-	-	-	✓	-
Medicalis Professional	TCP/IP	-	✓	✓	✓	✓	✓	✓	✓
MedOutlook RIS	Cust TCP/IP	-	-	✓	-	-	-	✓	-
MedQ Dictation	SharedFile	-	-	✓	-	-	-	✓	-
MedQuist SpeechQ*	COM API	✓	✓	✓	-	-	-	✓	-
MRS	Cust TCP/IP	✓	✓	✓	-	✓	✓	✓	-
MultiView or Aegis	Cust TCP/IP	✓	✓	✓	✓	✓	✓	✓	-
NBSS British Screening RIS	Proxy	-	-	-	-	-	-	✓	-
peerView qiVue	TCP/IP	✓	✓	✓	-	-	-	✓	-
PenRad	SharedFile	-	-	✓	-	-	-	✓	✓
PenRad	XML file	-	-	✓	-	-	-	✓	-

Application Synchronization 4.5 Release Notes
Notes for Version 4.5

	Partner Type	Receives				Sends			
		Login	Logout	Open Patient	Update Patient	Login	Logout	Open Patient	Update Patient
Philips iSite PACS	HTTP plug-in	-	-	-	-	-	-	✓	-
PowerScribe or EWS	XML file	✓	✓	✓	-	-	-	-	-
PowerScribe bi-directional	XML file	✓	✓	✓	-	-	-	✓	✓
Nuance RadWhere	XML file	✓	✓	✓	-	-	-	-	-
PowerScribe 360	XMLfile	✓	✓	✓	-	-	-	✓	✓
QDoc StudyViewer	COM API	✓	✓	✓	-	✓	✓	✓	-
RadNet VDI*	COM API	✓	✓	✓	-	-	-	-	-
RadNet VDI prior to version 2007.18*	COM API	✓	✓	✓	-	-	-	-	-
ScImage Picom Client	SharedFile	-	-	-	-	✓	✓	✓	-
SecurView	Cust TCP/IP	✓	✓	✓	✓	✓	✓	✓	✓
SecurView Serial Partner	Serial	-	-	✓	-	-	-	-	-
Selenia AWS	Cust TCP/IP	-	-	-	-	-	-	✓	-
SERGAS SIDI	TCP/IP	-	-	-	-	✓	-	✓	-
SESCAM Dulcingest RIS	TCP/IP	-	-	-	-	✓	✓	✓	-
Softek Illuminate Insight	SharedFile	✓	✓	✓	-	-	-	-	-
Spantel Audio Recorder*	COM API	-	-	✓	-	-	-	-	-
syngo Workflow (Europe)	Proxy	-	-	-	-	-	-	✓	-

Application Synchronization 4.5 Release Notes

Notes for Version 4.5

	Partner Type	Receives				Sends			
		Login	Logout	Open Patient	Update Patient	Login	Logout	Open Patient	Update Patient
syngo Workflow Portal Radiologist*	HTTP	-	-	-	-	-	-	✓	-
syngo Workflow ReadExam	XML file	-	-	-	-	-	-	✓	-
Unifi Workspace	Cust WebAPI	-	✓	✓	-	✓	✓	✓	-
WinScribe Internet Author	SharedFile	✓	✓	✓	-	-	-	-	-

+ Denotes available functionality

* Application Synchronization must be installed on the same computer as this third-party application. Application Synchronization can be installed and configured on a computer other than SecurView, MultiView, or Unifi Workspace if necessary.



Note

Third-party application versions listed are the earliest versions of software validated by Hologic, usually the earliest version of the third-party application that supported the interface.

While synchronization with newer versions of third-party applications is expected to work, their usage on the Hologic workstation may result in performance degradation, or there may be interface incompatibilities. If you have questions regarding support of a particular third-party application version, contact Hologic service.



Note

Consult your Sales Account Manager to check on a third-party application that is not shown above.

Configuration Information

3.1 Using Synchronization on SecurView

For complete information on configuring synchronization features within the SecurView workstation, see the latest *SecurView DX/RT Workstation User Guide*. For information on configuring synchronization features within a third-party application, see the appropriate third-party documentation.

When an approved third-party application is running on SecurView and displayed on an additional monitor that is shared with the SecurView Patient List and/or the SecurView Multimodality Viewer or MultiView Multimodality, use [Alt] + [Tab] to toggle through the applications.

3.2 Using Synchronization on MultiView

For complete information on configuring synchronization features within MultiView, see the latest *MultiView Multimodality User Guide*. For information on configuring synchronization features within a third-party application, see the appropriate third-party documentation.

3.3 Using Synchronization on Unifi Workspace

For complete information on configuring synchronization features within Unifi Workspace, see the latest *Unifi Workspace User Guide*. For information on configuring synchronization features within a third-party application, see the appropriate third-party documentation.