GE Senographe DS® Stereotaxy and GE Senographe® Essential Stereotaxy Systems Setup Guide (MammoTest®) MAN-09292 Revision 001

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PLEASE READ PRIOR TO PERFORMING PROCEDURE:

This setup guide is designed to be used in conjunction with, not replace, the Eviva® breast biopsy device instructions for use and/or the GE Senographe® Essential stereotaxy operator manual and/or the GE Senographe DS® stereotaxy operator manual. Prior to performing the procedure, the physician must review and be familiar with the full operating instructions for the Eviva breast biopsy device and GE system(s). Consult the Eviva breast biopsy device instructions for use, the GE Senographe Essential stereotaxy operator manual or the GE Senographe DS stereotaxy operator manual for full operational instructions, as well as any contraindications and safety information. Verification testing was conducted to ensure accuracy of the Eviva biopsy device on the GE Senographe DS Stereotaxy system and the GE Senographe Essential stereotaxy system. However, the setup numbers provided herein are for guidance purposes only. The setup numbers must be verified onsite with the corresponding Eviva biopsy device prior to performing any procedure.

Please retain this setup guide. Note that while Hologic frequently updates the contents of this guide, medical information and technology can change rapidly. Therefore some of the information in this guide may be out of date. Use of these instructions with any product other than those specifically noted may result in injury to the patient and/or damage to the equipment.

Table of Contents

Setup of GE Senographe® Essential Stereotaxy for Eviva® Biopsy Device with Vertical Approach	Page 2
Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe <u>Essential</u> Stereotaxy System with <u>Vertical</u> Approach	Page 3
Setup of GE Senographe <u>Essential</u> Stereotaxy for the Eviva Biopsy Device with <u>Lateral</u> Arm Approach	Page 3
Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe <u>Essential</u> Stereotaxy System with <u>Lateral</u> Arm Approach	Page 4
Setup of GE Senographe <u>DS</u> Stereotaxy for Eviva Biopsy Device with <u>Vertical</u> Approach	Page 5
Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe <u>DS</u> Stereotaxy System with <u>Vertical</u> Approach	Page 6
Setup of GE Senographe <u>DS</u> Stereotaxy for Eviva Biopsy Device with <u>Lateral</u> Arm Approach	Page 7
Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe <u>DS</u> Stereotaxy System with <u>Lateral</u> Arm Approach	Page 8
Cleaning Instructions for Eviva Adapter, Eviva Adapter for GE, Eviva Stylet Bracket and Eviva Bushing for GE	Page 8


Setup of GE Senographe® Essential Stereotaxy for Eviva® Biopsy Device with Vertical Approach

Program the GE Senographe Essential stereotaxy system for Eviva biopsy device according to the needle setup instructions provided in the GE Senographe Essential stereotaxy operator manual, in conjunction with the steps below.

When “Creating” needles, select “**VAD with needle holder**” for type of needle. Use the dimensions provided in the table below. The diagram below the table shows a schematic of the dimensions.

NOTE: Do not select “CB” because this will cause the needle to move to an incorrect depth

	Eviva 0910-20 Eviva 1210-20	Eviva 0910-12	Eviva 0910-12T
Recommended Name in Software	Eviva XX10-20ver	Eviva 0910-12ver	Eviva 0910-12Tver
A (mm)	101.0	97.2	101.0
B (mm)	20	12	12
C (mm)	8	4	8
D (mm)	260	260	260
E (mm)	-16.4	-16.4	-16.4
All values entered above must be verified with the appropriate Eviva biopsy device and applicable test to ensure the target is near the center of the biopsy device aperture.			

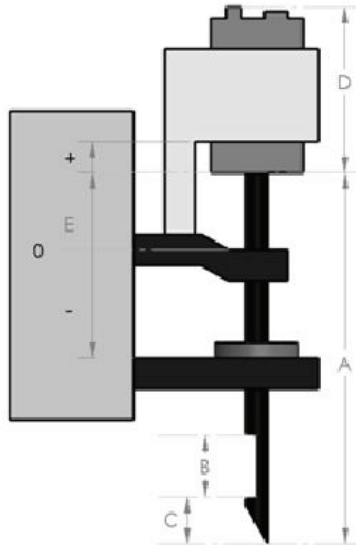Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe Essential Stereotaxy System with Vertical Approach

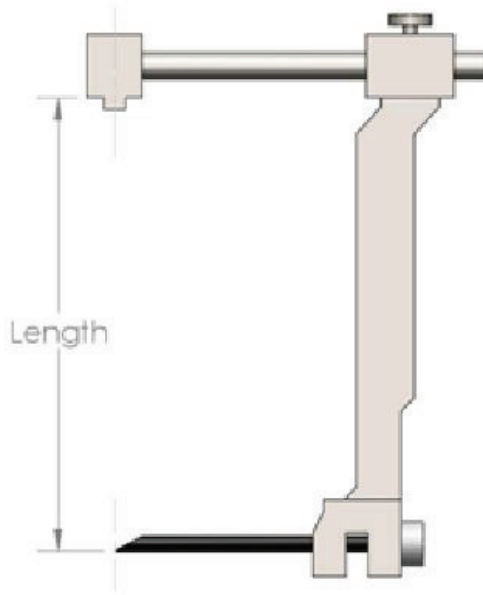
1. Setup GE Senographe Essential stereotaxy system for vertical approach using appropriate bracket(s). Refer to GE Senographe Essential stereotaxy system operator manual for setup instructions
2. Install the *Eviva Adapter for GE* onto the GE Senographe Essential stereotaxy system.
3. Install the *Eviva Bushing for GE* into the lower needle guide holder of the GE Senographe Essential stereotaxy system.
4. Install the appropriate disposable needle guide, *Eviva_NG09R* or *Eviva_NG12R*, into the *Eviva Bushing for GE*.
5. Ensure the proper Eviva biopsy device is selected in the GE Senographe Essential stereotaxy software prior to sending the target to the stereotactic system. The selected Eviva biopsy device should match the biopsy device that is being used in the procedure.
6. Perform the biopsy procedure per the Eviva breast biopsy device instructions for use.
7. Clean the *Eviva Adapter for GE* and *Eviva Bushing for GE* per the cleaning instructions found on page 8 of this document.

Setup of GE Senographe Essential Stereotaxy for Eviva Biopsy Device with Lateral Arm Approach

Program the GE Senographe Essential stereotaxy system for a lateral needle according to the instructions provided in the GE Senographe Essential stereotaxy operator manual, in conjunction with steps below.

- a. When "Creating" needles, select "LAT" for type of needle. The recommended name for the needle is **ATEC-EvivaLAT**.
- b. For (A) *Length* enter **107.8** mm to create the needle. The diagram below shows a schematic of the dimension.

NOTE: This length ensures that the needle will be aligned with the needle guide. This length is common for all Eviva and ATEC devices. No information about the length of the needle, notch (aperture) or end (dead space) is entered into the system; therefore, only one needle needs to be programmed for all current Eviva and ATEC devices. **This distance does not control the insertion depth of the needle into the breast.** The insertion depth will be controlled by advancing the biopsy device to the recommended position in the table on page 5.

All values entered above must be verified with the appropriate Eviva biopsy device and applicable test to ensure the target is near the center of the biopsy device aperture. The Eviva biopsy device must be advanced to the proper device position on the lateral adapter bar. See the table on page 5.

Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe Essential Stereotaxy System with Lateral Arm Approach

1. Setup GE Senographe Essential stereotaxy system for lateral approach using appropriate bracket(s). Refer to GE Senographe Essential stereotaxy operator manual for setup instructions.
2. Install the *Eviva Adapter* onto the GE Senographe Essential stereotaxy system.
3. Install the appropriate disposable needle guide, *Eviva_NG09R* or *Eviva_NG12R*, into the GE Senographe Essential lateral arm.
4. Ensure the proper Eviva needle is selected (**ATEC-EvivaLAT**) in the GE Senographe Essential stereotaxy software prior to sending the target to the stereotactic system.
5. Advance the Eviva biopsy device to the proper position on the lateral adapter bar. See the table below for this position.
6. Perform the biopsy procedure per the Eviva breast biopsy device instructions for use.
7. Clean the *Eviva Adapter* per the cleaning instructions found on page 8 of this document.

	Eviva 0913-20 Eviva 1213-20	Eviva 0913-12	Eviva 0913-12T
Proper Device Position on the Lateral Adapter Bar (mm)	68	72	72
All values entered above must be verified with the appropriate Eviva biopsy device and applicable test to ensure the target is near the center of the biopsy device aperture. The Eviva biopsy device must be advanced to the proper device position on the lateral adapter bar. See above chart.			


Setup of GE Senographe DS Stereotaxy for Eviva Biopsy Device with Vertical Approach

Program the GE Senographe DS Stereotaxy system for the Eviva biopsy device according to the needle setup instructions provided in the GE Senographe DS stereotaxy operator manual, in conjunction with the step below.

When “Creating” needles, select “**CB**” for type of needle. Use the dimensions provided in the table below. The diagram below the table shows a schematic of the dimensions.

NOTE: For GE Senographe DS stereotaxy systems with updated biopsy needle setup procedures as noted in operator manual 5308544-3-C-1EN or later, please refer to the Setup of GE Senographe Essential Stereotaxy for Eviva Biopsy Device with Vertical Approach sheet on page 2 of this document to complete needle setup instructions.

	Eviva 0910-20 Eviva 1210-20	Eviva 0910-12	Eviva 0910-12T
Recommended Name in Software	Eviva XX10-20ver	Eviva 0910-12ver	Eviva 091012Tver
A (mm)	121.6	117.6	121.6
B (mm)	20	12	12
C (mm)	8	4	8
All values entered above must be verified with the appropriate Eviva biopsy device and applicable test to ensure the target is near the center of the biopsy device aperture.			

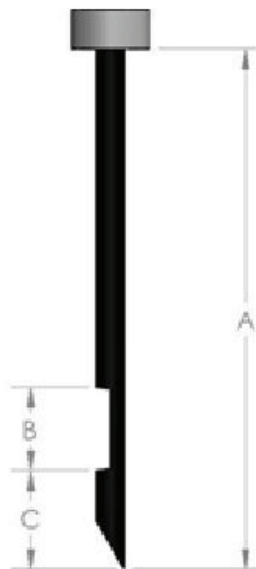Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe DS Stereotaxy System with Vertical Approach

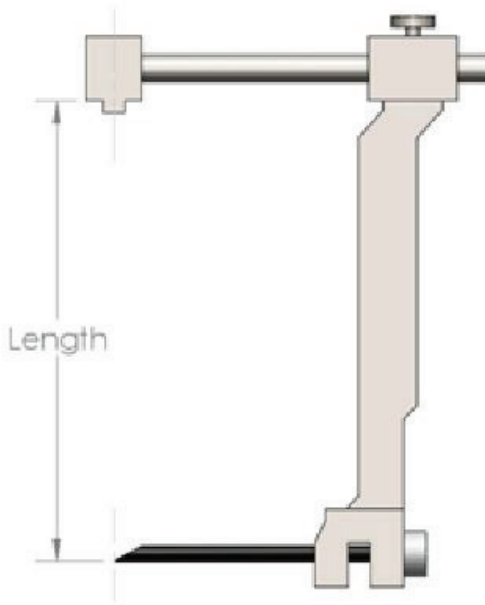
1. Setup GE Senographe DS stereotaxy system for vertical approach using appropriate bracket(s). Refer to GE Senographe DS operator manual for setup instructions.
2. Install the *Eviva Adapter for GE* onto the GE Senographe DS stereotaxy system.
3. Install the *Eviva Bushing for GE* into the lower needle guide holder of the GE Senographe DS stereotaxy system.
4. Install the appropriate disposable needle guide, *Eviva_NG09R* or *Eviva_NG12R*, into the *Eviva Bushing for GE*.
5. Ensure the proper Eviva biopsy device is selected in the GE Senographe DS stereotaxy software prior to sending the target to the stereotactic system. The selected Eviva biopsy device should match the biopsy device that is being used in the procedure.
6. Perform the biopsy procedure per the Eviva breast biopsy device instructions for use.
7. Clean the *Eviva Adapter for GE* and *Eviva Bushing for GE* per the cleaning instructions found on page 8 of this document.

Setup of GE Senographe DS Stereotaxy for Eviva Biopsy Device with Lateral Arm Approach

Program the GE Senographe DS stereotaxy system for a lateral needle according to the instructions provided in the GE Senographe DS stereotaxy operator manual, in conjunction with the steps below.

- a. When "Creating" needles, select "**LAT**" for type of needle. The recommended name for the needle is **ATEC-EvivaLAT**.
- b. For (A) *Length* enter **107.8** mm to create the needle. The diagram below shows a schematic of the dimension.

NOTE: This length ensures that the needle will be aligned with the needle guide. This length is common for all Eviva and ATEC devices. No information about the length of the needle, notch (aperture) or end (dead space) is entered into the system; therefore, only one needle needs to be programmed for all current Eviva and ATEC devices. **This distance does not control the insertion depth of the needle into the breast.** The insertion depth will be controlled by advancing the biopsy device to the recommended position in the table on page 8.

All values entered above must be verified with the appropriate Eviva biopsy device and applicable test to ensure the target is near the center of the biopsy device aperture. The Eviva biopsy device must be advanced to the proper device position on the lateral adapter bar. See table on page 8.

Performing a Stereotactic Biopsy Using the Eviva Biopsy Device on the GE Senographe DS Stereotaxy System with Lateral Arm Approach

1. Setup GE Senographe DS stereotaxy system for lateral approach using appropriate bracket(s). Refer to GE Senographe DS operator manual for setup instructions.
2. Install the *Eviva Adapter* onto the GE Senographe DS stereotaxy system.
3. Install the appropriate disposable needle guide, *Eviva_NG09R* or *Eviva_NG12R*, into the GE Senographe DS lateral arm.
4. Ensure the proper Eviva biopsy device is selected (**ATEC-EvivaLAT**) in the GE Senographe DS stereotaxy software prior to sending the target to the stereotactic system.
5. Advance the Eviva biopsy device to the proper position on the lateral adapter bar. See the table below for this position.
6. Perform the biopsy procedure per the Eviva breast biopsy device instructions for use.
7. Clean the *Eviva Adapter* per the cleaning instructions found on page 8 of this document.

	Eviva 0913-20 Eviva 1213-20	Eviva 0913-12	Eviva 0913-12T
Proper Device Position on the Lateral Adapter Bar (mm)	68	72	72
All values entered above must be verified with the appropriate Eviva biopsy device and applicable test to ensure the target is near the center of the biopsy device aperture. The Eviva biopsy device must be advanced to the proper device position on the lateral adapter bar. See table above.			

Cleaning Instructions for *Eviva Adapter*, *Eviva Adapter for GE*, *Eviva Stylet Bracket* and *Eviva Bushing for GE*

1. Upon completion of the biopsy procedure, immediately rinse the *Eviva Adapter*, *Eviva Adapter for GE*, and/or the **EVIVA BUSHING FOR GE** with warm water.
2. Spray all components with a cleaner

The following cleaning products are **NOT RECOMMENDED** as they may damage the exposed components:

- Bleach based cleaning agents
- Hydrogen Peroxide
- Any cleaners/disinfectants with a pH of less than 4 or greater than 10

3. Allow the necessary time for the chosen cleaner to disinfect the components.
4. Rinse with warm water.
5. Wipe dry with cloth or paper towel.

Symbols:

The following symbols may be found on the product labeling for the Eviva adapter:

Symbol	Description	Standard
	Authorized Representative in the European Community	ISO 15223-1, Reference 5.1.2
	Prescription use only	FDA 21 CFR 801.109
	Caution	ISO 15223-1, Reference 5.4.4
 www.hologic.com/package-inserts	Consult instructions for use	ISO 15223-1, Reference 5.4.3 Hologic
	Follow instructions for use	IEC 60601-1, Reference No. Table D.2, Safety sign 10 (ISO 7010-M002)
	CE Mark European Conformity	MDR Regulation (EU) 2017/745
	Medical device	ISO 15223-1, Reference 5.7.7
	Translations in Box	Hologic
	Do not use if package is damaged	ISO 15223-1, Reference 5.2.8
	Catalog number	ISO 15223-1, Reference 5.1.6
	Quantity	Hologic
	Manufacturer	ISO 15223-1, Reference 5.1.1
	Country of Manufacture	ISO 15223-1, Reference 5.1.11
YYYY-MM-DD	Date format: YYYY represents the year MM represents the month DD represents the day	Hologic
	Keep dry	ISO 15223-1, Reference 5.3.4
	Serial number	ISO 15223-1, Reference 5.1.7

Symbol	Description	Standard
	Batch code	ISO 15223-1, Reference 5.1.5
	Patents	Hologic
	Non-sterile	ISO 15223-1, Reference 5.2.7
	Not safe for magnetic resonance imaging	ASTM F2503 Reference no. Table 2, Symbol 7.3.3;7.4.9.1; Fig. 9
	Country code for translation	ISO 3166

©2024 Hologic, Inc. All Rights Reserved. Hologic and Eviva are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks and product names used with this document are the property of their respective owner.

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