# eviva<sup>®</sup>

Stereotactic Guided Breast Biopsy System

Instructions For Use

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Mode d'emploi

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**HOLOGIC<sup>®</sup>**

# Eviva® Breast Biopsy Device

## Instructions for Use (IFU)

Please read all information carefully. Failure to properly follow the instructions may lead to unintended consequences.

Important: This package insert is designed to provide instructions for clinical use (IFU) for the Eviva breast biopsy device to be used with the ATEC® breast biopsy and excision system console. It is not a reference to surgical techniques.

---

## Indications

The Eviva breast biopsy device is indicated to provide breast tissue samples for diagnostic sampling of breast abnormalities under stereotactic breast biopsy guidance. The Eviva device is intended to provide breast tissue for histologic examination with partial or complete removal of the imaged abnormality. The extent of histologic abnormality cannot be reliably determined from its mammographic appearance. Therefore, the extent of removal of the imaged evidence of an abnormality does not predict the extent of removal of histologic abnormality, e.g., malignancy. When the sampled abnormality is not histologically benign, it is essential that the tissue margins be examined for completeness of removal using standard surgical procedure.




## Contraindications

The Eviva device is for diagnostic use only and is NOT indicated for therapeutic use. The Eviva device is contraindicated for those patients who, based on physician's judgment, may be at increased risk or develop complications associated with core removal or biopsy. Patients receiving anticoagulant therapy or who may have bleeding disorders may be at increased risk of procedural complications.

## Device Description

The Eviva device is a vacuum-assisted biopsy device which is used to remove breast tissue in a minimally invasive manner using stereotactic imaging. The Eviva device is designed for biopsy and diagnosis only. The device is NOT intended to be used outside the standards, specifications and limitations as outlined in this IFU and other related materials. The device is for single patient use and the entire device is fully disposable. The device primarily consists of a hollow needle with a side aperture and a sharpened inner cannula that rotates at high speeds and extends across the aperture to acquire targeted tissue. Throughout this process, vacuum is created inside the device to assist in pulling tissue into the aperture. Saline is supplied through the device to lavage the cavity and deliver tissue to the tissue filter. An introducer system, specifically designed for use with the Eviva device, is included with the device. This introducer system maintains access to the targeted area of interest and allows for deployment of a biopsy site marker.

Refer to the “ATEC Breast Biopsy and Excision System Operator’s Manual” for detailed console instructions.

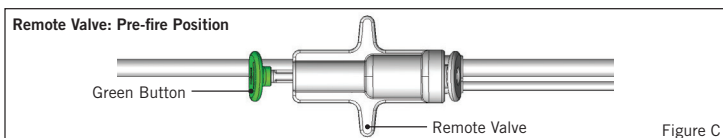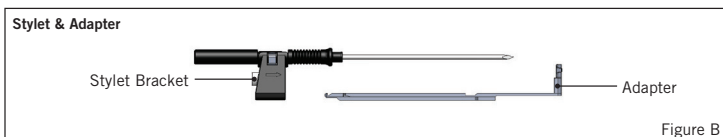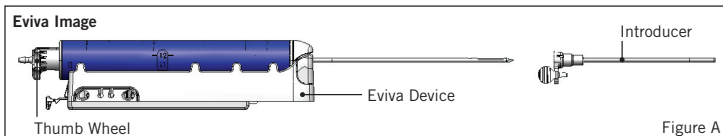## Device Preparation – Firing or Non-firing Device (all steps to be performed per standard interventional technique)

1. Assemble canister, place in the canister holder on the top of the console and connect vacuum-line assembly from console to the top port of canister lid. Ensure canister lid and large port cap are secure and sealed to avoid vacuum leaks.

**WARNING:** Before use, inspect the protective packaging and needle to verify that neither has been damaged during shipment. If it appears that the packaging or needle has been compromised, do not use the needle.

2. Place device package on console workspace.
3. Open package. Leave protective sheath on the device tip.
4. Open the saline bag, remove cap, spike saline bag and place the bag on the hook on the side of the console. Place saline line through pinch valve.
5. Connect tubing set to console and canister. Attach red sleeve to red device receptacle on console, black sleeve to black device receptacle on console and blue vacuum into the “Vacuum” port on the canister lid. For more detailed console connection instructions, please refer to the “ATEC Breast Biopsy and Excision System Operator’s Manual.”

6. Power console on (console will default to “Biopsy” mode).
7. Ensure device is in post-fire position by confirming the green button on the remote valve is fully depressed (firing device only).
8. Select “Setup” to prime device (saline visible in aperture) and confirm adequate vacuum is achieved (“Vacuum Ready” light should be illuminated green). If ample vacuum is not achieved, the “Vacuum Ready” light will flash. If the light flashes, refer to the Troubleshooting section of this IFU and/or the Troubleshooting section of the “ATEC Breast Biopsy and Excision System Operator’s Manual.”
9. Select “Test” and ensure one diagnostic biopsy cycle is completed (console automatically switches to “Biopsy” after passing the “Test” cycle). If the “Return to Setup” or “Retest Handpiece” lights illuminate, then refer to the Troubleshooting section of this IFU and/or the “ATEC Breast Biopsy and Excision System Operator’s Manual.”
10. Depress the white button on the remote valve to place the device in the pre-fire position. Refer to Figure C. Then, depress the green button on the remote valve to ensure that the outer cannula travels to the post-fire position.
11. Attach adapter and needle guide to the stage of the imaging system. Some imaging systems require input dimensions for the Eviva device. In these cases, refer to the documentation included with the adapter for your system.

## **Performing a Biopsy (all steps to be performed per standard interventional technique)**

1. Identify targeted area.
2. Prepare the site as required.

### **Firing Device (20mm or 12mm aperture)**

- a. Carefully remove the device from tray.
- b. Remove protective sheath from the device.
- c. Remove introducer from tray and carefully assemble introducer onto device, (refer to Figure A).
- d. Confirm the console is in “Biopsy” mode, prior to manipulation of the remote valve.  
Note: If console is not in the “Biopsy” mode, then the device may not function appropriately when attempting to place it in the “pre-fired” position.

- e. Set the device to the pre-fire position by depressing the white button on the remote valve, (refer to Figure C). Note: When the Eviva device is in the pre-fire position, the ATEC console must be powered on and in “Biopsy” mode to prevent migration of device to the post-fire biopsy position.
- f. Slide device/introducer onto the adapter until it locks into place with an audible click when the device is securely engaged it will no longer slide on the adapter.
- g. Advance the device to the desired pre-fire coordinates provided by the imaging system.
- h. Fire the outer cannula by fully depressing the green button of the remote valve. This will advance the outer cannula by the stroke noted on the product label.

### **Non-firing Device (12mm aperture)**

- a. Remove disposable stylet from tray and remove protective sheath from the stylet.
  - b. Place stylet into the stylet bracket, (refer to Figure B).
  - c. Remove introducer from tray and carefully assemble introducer onto stylet.
  - d. Slide stylet bracket assembly onto the adapter. Advance the stylet bracket forward until the bracket and introducer lock into place with an audible click.
  - e. Advance to the desired coordinates provided by the imaging system.
  - f. While depressing the button on the back of the stylet bracket, remove the stylet bracket assembly from the patient and slide off the adapter. Introducer will remain connected to adapter and in the patient.
  - g. Confirm the console is in “Biopsy” mode.
  - h. Slide the device onto the adapter while guiding the blunt needle into the introducer until it locks into place with an audible click. When the device is securely engaged it will no longer slide on the adapter.
3. To administer preferred anesthetic, attach 10cc needleless syringe of anesthetic to Y-valve. To initiate automatic delivery of anesthetic, inject 1-2cc of anesthetic manually. Note: The device will not allow Y-valve injections when in the pre-fire position. Attempting to inject in the pre-fire position may result in device malfunction.
  4. To begin tissue acquisition, press down on the footswitch and hold in the depressed position throughout the biopsy procedure. Removing your foot from the footswitch will inactivate or stop the device after completing the current cycle.

5. To rotate the aperture on the outer cannula of the device, turn the rear thumbwheel (refer to Figure A). Turn the thumbwheel from one position to the next desired position as the console beeps until the desired target area has been sampled. The numbers in the window on the device and the line on the thumbwheel indicate position of the aperture.
6. Select “Lavage” mode to irrigate and aspirate the cavity and clear device of tissue. (It is not necessary to depress the footswitch.)
7. To vacuum cavity, disconnect luer lock of Y-valve to open saline line to atmosphere resulting in constant aspiration to the biopsy cavity.
8. Disconnect filter chamber to remove tissue cores from tissue filter. Note: If blood loss from patient is noted, then the device can be placed in the biopsy mode. This closes the aperture and prevents any fluid loss from the biopsy cavity.
9. Place console in “Biopsy” mode in order to close the aperture prior to removal of device from biopsy cavity.
10. Remove device from adapter by lifting rear latch and sliding the device back and free of the introducer. Introducer will remain connected to adapter and in the patient.
11. Power console off.
12. For deployment of the biopsy site marker, refer to the appropriate Instructions for Use (IFU).
13. Remove introducer from adapter by pinching tabs and sliding back and free of the adapter.
14. Appropriately dispose of all used and unused biopsy items.
15. Properly clean and disinfect adapter as necessary.

## Warnings and Precautions

- Observe caution when attaching sharp devices to the adapter.
- When the Eviva device is in the pre-fire position, the ATEC console must be powered on and in “Biopsy” mode to prevent migration of device to the post-fire biopsy position. The Eviva device must be in the post-fire position, to perform “Setup”, “Test”, “Biopsy”, “Lavage” or “Manual Aspiration”.
- Do not cut tubing between device and remote valve in order to avoid inadvertent firing. This is indicated by the following symbol:

- As with any medical procedure, please ensure that users wear appropriate personal protective equipment to guard against potential contact with bodily fluids.
- The Eviva device is not recommended for use with MRI and ultrasound.
- The Eviva device biopsy procedure should be performed only by persons having adequate training and familiarity with this procedure. Consult medical literature relative to techniques, complications and hazards prior to performance of any minimally invasive procedure.
- The Eviva device should be used only by physicians trained in percutaneous biopsy procedures. The Eviva breast biopsy device is not recommended for use with MRI.
- **Rx ONLY** Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
- Sound professional judgment should be used when the Eviva device is used on patients with breast implants.
- Avoid operator or instrument contact with the sheathed needle portion of the Eviva device.
- Minimally invasive instruments and accessories manufactured or distributed by companies not authorized by Hologic, Inc. may not be compatible with the Eviva device. Use of such products may lead to unanticipated results and possible injury to the user or patient.
- Instruments or devices which come into contact with bodily fluids may require special disposal handling to prevent biological contamination.
- Dispose of all opened instruments whether used or unused.
- Do not resterilize and/or reuse the Eviva device and/or introducer. Resterilization and/or reuse may compromise the integrity of the instrument. This may lead to potential risks of failure of the device to perform as intended and/or cross-contamination associated with using inadequately cleaned and sterilized devices.
- A complete and comprehensive preoperative medical history and physical examination are suggested. Radiographic evaluation and laboratory tests may be included.

## How Supplied

The Eviva device is supplied sterile for single patient use. Discard into an appropriate container after use. The biopsy device is sterilized by gamma radiation to sterility assurance level (SAL) 10<sup>-6</sup>.

## For More Information

For technical support or reorder information in the United States, please contact:


Hologic, Inc.  
250 Campus Drive  
Marlborough, MA 01752 USA  
Phone: 877-371-4372  
BreastHealth.Support@hologic.com

International customers, contact your distributor or local Hologic Sales Representative:

European Representative  
Hologic BV  
Da Vincilaan 5  
1930 Zaventem  
Belgium  
Tel: +32 2 711 46 80

## Troubleshooting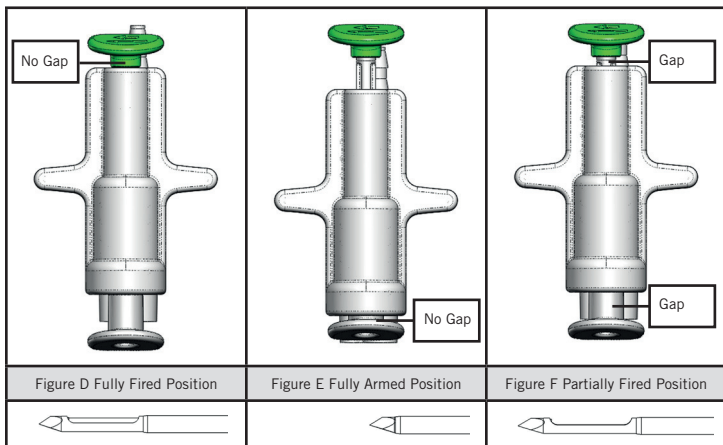

Remote Valve Position	Fully Fired	Fully Armed	Partially Fired
<b>Needle Position</b>	Needle in fired position. Aperture will open and close as intended.	Needle in armed position. Aperture will remain closed in all modes.	Needle in fired position. Aperture may not fully close during biopsy.
<b>Symptom(s)</b>	Device functions as intended.	No saline flow in all modes. Device sounds normal during biopsy but unable to acquire tissue.	Saline flow is normal in all modes. Device sounds normal during biopsy but acquires inadequate or no tissue.*
<b>Remedy</b>	Not Applicable	Fully fire the needle. See Figure D.	Fully fire the needle. See Figure D.

\* May also lead to failures during setup and test.

Failure Mode	Possible Cause	Possible Remedy
Device will not assume the pre-fire or post-fire position	Console is not in “Biopsy” mode	Press “Biopsy” on console  If device will still not pre-fire/post-fire, replace the device and repeat the setup procedure
During “Setup,” “Test,” “Biopsy,” “Lavage” or “Manual Aspiration” fluid and/or tissue is not being pulled to the tissue filter or aperture is not opening or closing as expected	Remote valve is not in full post-fired position	Place device in post-fire position by fully depressing green button on remote valve
Unable to depress syringe to administer anesthetic	Device is not in post-fire position	Place device in post-fire position by depressing green button on remote valve

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## Additional Symbols Used on Labeling

Symbol	Description	Standard
<b>R<sub>x</sub> ONLY</b>	Prescription use only	21 CFR 801.109
<div style="border: 1px solid black; padding: 2px; display: inline-block;"><b>EC</b></div> <div style="border: 1px solid black; padding: 2px; display: inline-block;"><b>REP</b></div>	Authorized Representative in the European Community	ISO 15223-1:2016, Reference 5.1.2
	Caution	ISO 15223-1:2016, Reference 5.4.4
<b>CE</b> XXXX	CE Mark with notified body reference number	MDR Regulation (EU) 2017/745
 <small>www.hologic.com/package-inserts</small>	Consult electronic instructions for use	ISO 15223-1:2016, Reference 5.4.3
<div style="border: 1px solid black; padding: 2px; display: inline-block;"><b>Translations in Box</b></div>	Translations in Box	Hologic
	Do not use if package is damaged	ISO 15223-1:2016, Reference 5.2.8
<div style="border: 1px solid black; padding: 2px; display: inline-block;"><b>REF</b></div>	Catalog number	ISO 15223-1:2016, Reference 5.1.6
<div style="border: 1px solid black; padding: 2px; display: inline-block;"><b>LOT</b></div>	Batch code	ISO 15223-1:2016, Reference 5.1.5

	Quantity	Hologic
	Manufacturer	ISO 15223-1:2016, Reference 5.1.1
	Date of manufacture	ISO 15223-1:2016, Reference 5.1.3
	Keep dry	ISO 15223-1:2016, Reference 5.3.4
	Serial number	ISO 15223-1:2016, Reference 5.1.7
	Patents	Hologic
	Non-sterile	ISO 15223-1:2016, Reference 5.2.7
	Do not re-sterilize	ISO 15223-1:2016, Reference 5.2.6
	Do not re-use	ISO 15223-1:2016, Reference 5.4.2
	MR Conditional	ASTM F2503 Reference no. Table 2; 7.4.6.1; Fig 6,7
	MR Unsafe	ASTM F2503 Reference no. Table 2, Symbol 7.3.3;7.4.9.1; Fig. 9
	Sterilized using irradiation	ISO 15223-1:2016, Reference 5.2.4
	Use-by Date	ISO 15223-1:2016, Reference 5.1.4

<b>Made in Costa Rica</b>	Made in Costa Rica	Hologic
<b>YYYY-MM-DD</b>	Expiration date is represented by the following: YYYY represents the year MM represents the month DD represents the day	Hologic
	Upper limit of temperature	ISO 15223-1:2016, Reference 5.3.6
	Do not cut this line	N/A

[www.hologic.com/package-inserts](http://www.hologic.com/package-inserts)

# Labeling Translations

## Eviva® Stereotactic Guided Breast Biopsy System

Product Code: EVIVA 0913-12, Eviva 0913-12T, Eviva 0910-20, Eviva 0913-20, Eviva 0910-12, Eviva 0910-12T, Eviva 1213-20 and Eviva 1210-20.

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**PL**

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# Labeling Translations

## Eviva® Stereotactic Guided Breast Biopsy System

Product Code: EVIVA NG09L, EVIVA NG12L, EVIVA NG09R, EVIVA NG12R.

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**PL**

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# Labeling Translations

## Eviva® Stereotactic Guided Breast Biopsy System

Product Code: EVIVA 0913-12, Eviva 0913-12T, Eviva 0910-20, Eviva 0913-20, Eviva 0910-12, Eviva 0910-12T, Eviva 1213-20 and Eviva 1210-20.

LBL-00785 Rev. 002

**EN**

1. DO NOT CUT THIS LINE
2. CUT LINE ADJACENT TO GREEN BUTTON

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