



InSight[™] FD Mini C-arm User Guide
MAN-04858 Revision 004

InSight™ FD

Mini C-arm

User Guide

Part Number: MAN-04858
Revision 004

HOLOGIC®

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

Customer Support

Toll Free in USA: +1.800.321.4659

Email: dxasupport@hologic.com

In Europe, South America, or Asia, contact your local dealer or distributor.

HOLOGIC®



Manufactured for Hologic, Inc.
36 Apple Ridge Road
Danbury, CT 06810 USA

Made in Mexico



Hologic Ltd.
Heron House
Oaks Business Park
Crewe Road
Wythenshawe, Manchester
M23 9HZ, UK
Tel: +44 (0)161 946 2206
Fax: +44 (0)161 602 0995



Refer to corporate website for additional information www.Hologic.com.

© 2016 – 2017 Hologic, Inc. This manual was originally written in English.

Hologic, Fluoroscanner, InSight, MegaView and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the U.S. and/or other countries. Any other product and company names mentioned herein are the trademarks of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patents.

Table of Contents

1 Indications for Use	1
2 Radiation Safety	2
2.1 General.....	2
2.2 Radiation Dose and Dose Rate.....	2
2.3 X-ray Shielding.....	3
3 Regulations for X-ray Equipment	3
3.1 United States Federal and State Regulations.....	3
3.2 IEC Regulations.....	3
3.3 Canadian Regulations.....	4
4 EMI	4
5 Accessories	4
6 InSight FD System Labels	5
6.1 System Main Label.....	5
6.2 X-ray System Warning Label.....	5
6.3 Electric Shock Warning Label.....	6
6.4 Field of View.....	6
7 Symbols	6
8 Safety Hazards	8
9 System Components	13
10 Controls and Indicators	14
10.1 X-ray Head Control Panel.....	14
10.2 Keyboard Controls.....	15
10.3 InSight FD System Footswitch Functions.....	17
10.3.1 Acquire an X-ray Image.....	18
10.3.2 Display a MegaView Image.....	18
10.3.3 Tag an Image.....	18
10.3.4 Save an Image.....	18
10.3.5 Print an Image.....	18
11 Using the System	19
11.1 Moving the System.....	19
11.2 Adjusting the C-arm.....	20
11.3 Sterile C-arm Drape.....	21
11.4 Powering Up the System.....	22
11.5 Powering Down the System.....	22

12 Configuring the Application	23
12.1 System Configuration User Interface	23
12.2 System	23
12.3 DICOM Send Options.....	26
12.3.1 DICOM Print Options	28
12.4 Worklist Configuration.....	30
12.4.1 Users	32
12.5 Physician Preferences.....	33
12.6 Physician Preferences – Image Acquisition	35
12.7 Physician Preferences – Image Processing.....	37
12.8 Physician Preferences – Image Management	39
12.9 Administrative Settings Page	40
13 Using the InSight FD Application.....	41
13.1 Main Screen	41
13.2 Image Acquisition Session	47
13.2.1 Select or Enter Patient Window	47
13.2.2 Image Acquisition Screen	48
13.2.3 Select Reference Images Window	54
13.3 Review Images Session.....	55
13.3.1 Select Study for Review Window	55
13.3.2 Review Screen.....	57
13.3.3 Enhanced Displayed Image Screen	60
13.4 Maintaining Patient Study Records	61
13.4.1 Update Patient Study Information Window.....	61
13.5 Export Images	62
13.5.1 Export from Review Images Screen	62
13.5.2 Export from Review Images Screen	63
13.6 Import Images Window	66
13.7 Delete Images Window.....	68
13.8 View DICOM Queue Window	71
14 System Maintenance	72
14.1 System Backup	72
14.2 System Recover	72
14.3 Cleaning	72
14.3.1 Recommended Disinfectants.....	72
14.4 Preventive Maintenance	73

List of Figures

Figure 1: InSight FD Mini C-arm	1
Figure 2: System Main Label	5
Figure 3: X-ray System Warning.....	5
Figure 4: Electric Shock Warning.....	6
Figure 5: Field of View	6
Figure 6: Wireless Footswitch	17
Figure 7: Transport Position Knob	19
Figure 8: Rear Casters.....	20
Figure 9: Adjusting the C-arm.....	21
Figure 10: Sterile C-arm Drape.....	21
Figure 11: System Configuration Menu.....	23
Figure 12: System	24
Figure 13: DICOM Send Options.....	26
Figure 14: DICOM Print Options.....	28
Figure 15: Worklist Configuration.....	30
Figure 16: Users	32
Figure 17: Physician Preferences	33
Figure 18: Physician Preferences — Image Acquisition.....	35
Figure 19: Physician Preferences — Image Processing	37
Figure 20: Physician Preferences — Image Management	39
Figure 21: Main Screen	41
Figure 22: System Configuration Pull-down Menu	42
Figure 23: System Configuration Menu	43
Figure 24: Tools Pull-down Menu	45
Figure 25: Select or Enter Patient Window.....	47
Figure 26: Image Acquisition Screen.....	48
Figure 27: Image Effects Menu.....	49
Figure 28: Configure Image Area Menu.....	50
Figure 29: Image Acquisition Screen Buttons	50
Figure 30: Reference Images Window	54

Figure 31: Study for Review	55
Figure 32: Review Images Screen	57
Figure 33: Patient Details	59
Figure 34: VISTA Details.....	59
Figure 35: Enhanced Displayed Image Screen.....	60
Figure 36: Tools Menu.....	61
Figure 37: Image Export from Review Screen	62
Figure 38: Export Images	63
Figure 39: Tool Menu Export.....	63
Figure 40: Export Images from Tool Menu	64
Figure 41: Tools Menu.....	66
Figure 42: Import Images Window	66
Figure 43: Delete Images.....	68
Figure 44: Patient Selected	69
Figure 45: Image Selected for Deletion	69
Figure 46: View DICOM Queue.....	71

List of Tables

Table 1 Symbols	6
Table 2 Safety Hazards	8
Table 3 X-ray Head Control Panel	14
Table 4 Keyboard Controls	15
Table 5 Footswitch Pedal Functions	17
Table 6 Conditioning Schedule	22
Table 7 System Configuration User Interface	23
Table 8 System Page	25
Table 9 DICOM Send Options Page	26
Table 10 DICOM Send Options Page, Button Functions	27
Table 11 DICOM Print Options Page, Section/Field Descriptions	28
Table 12 DICOM Print Options Page, Button Functions	29
Table 13 Worklist Configuration, Section Descriptions	30
Table 14 Worklist Configuration, Button Functions	31
Table 15 Users Page, Section/Field Descriptions	32
Table 16 Users Page, Button Functions	33
Table 17 Physician Preferences, Section/Field Descriptions	34
Table 18 Physician Preferences, Button Functions	34
Table 19 Physician Preferences, Image Acquisition Tab, Section/Field Descriptions	36
Table 20 Physician Preferences, Image Processing Tab, Section/Field Descriptions	38
Table 21 Physician Preferences, Image Management Tab, Section/Field Descriptions	39
Table 22 Administrative Settings, Section Descriptions	40
Table 23 Administrative Settings, Button Functions	40
Table 24 Main Screen Options	41
Table 25 System Configuration Menu	43
Table 26 Tools Menu	45
Table 27 Select or Enter Patient Window, Section/Field Descriptions	47
Table 28 Select or Enter Patient Window, Button Functions	48
Table 29 Image Acquisition Screen	48
Table 30 Image Effects Menu	49

Table 31 Configure Image Area Menu	50
Table 32 Image Acquisition Screen, Button Functions	51
Table 33 Tool Set	52
Table 34 Select Reference Images Window, Section Descriptions	54
Table 35 Select Reference Images Window, Button Functions	55
Table 36 Select Study for Review Window, Section/Field Descriptions	56
Table 37 Select Study for Review Window, Button Functions	56
Table 38 Review Screen, Button Functions	57
Table 39 Detailed Image Viewing Screen, Button Functions	60
Table 40 Update Patient/Study Information Window, Section Descriptions	61
Table 41 Update Patient/Study Information Window, Button Functions	62
Table 42 Export Window, Section Descriptions	64
Table 43 Export Window, Button Functions	65
Table 44 Import Window, Section Descriptions	67
Table 45 Import Window, Button Functions	67
Table 46 Delete Images Window, Section Descriptions	70
Table 47 Delete Images Window, Button Functions	70
Table 48 Queue Window, Section Description and Button Functions	71
Table 49 Recommended Wipes	72
Table 50 Recommended Chemicals	73

1 Indications for Use

The InSight™ FD System is a Mini C-arm Fluoroscopic Imaging System designed to provide physicians with general fluoroscopic visualization of a patient, including, but not limited to surgical orthopedic and podiatry use, critical and emergency care procedures, and light anatomy imaging situations.

Figure 1 InSight FD Mini C-arm



2 Radiation Safety

Improper imaging procedures or the failure to observe radiation safety and technique factor recommendations could result in increased exposure. It is the operator's responsibility to observe all safety procedures and to keep radiation exposures As Low As Reasonably Achievable (ALARA) and to comply with all national, state and local regulatory requirements.

Hologic recommends that all members of the operating staff wear a radiation monitoring device. This is usually a film badge or TLD (thermoluminescent dosimetry) dosimeter which will record any radiation dose over a specific period of time. The physician should wear a ring badge to record any exposure to his/her hands.

2.1 General

A fluoroscopic device emits a continuous beam of X-rays in order to view an object in real-time.

Primary radiation is present in the space between the X-ray source and the detector screen. The operator should never place his/her own hands in the beam.

Secondary radiation or X-ray scatter is caused by X-rays that do not fully penetrate the object placed in the primary beam. X-ray scatter has considerably less energy than the primary beam and is absorbed by other objects in the immediate area. The amount of scatter depends upon the power setting used, the density and proximity of the body part to the beam, and the length of exposure time.

2.2 Radiation Dose and Dose Rate

Radiation received by each kilogram of a patient's body is called *dose*. *Dose rate* refers to the dose received per unit time.

Distance from the X-ray source reduces dose to both the patient and the operator. For example, at maximum output the dose is 2.6 mGy/minute at the image receptor and 45 mGy/minute at the Tube Housing exit port, or 17 times greater. This higher value is known as the *maximum Entrance Exposure Rate (EER)*.

Scattered radiation decreases with the square of the distance in exactly the same way as noted previously for X-rays in the primary beam. However, as the dose rate is much less, scatter is usually expressed in mR/hour (1 mR = 0.01 mGy).

Dose and scatter data for the InSight FD system is available from Hologic upon request.

2.3 X-ray Shielding

The scattered radiation from the InSight FD system is low compared to many other radiological devices, and the requirement to wear protective lead aprons and/or attenuating gloves is governed by applicable facility policy. Also many states, provinces, countries, and individual institutions may require persons in the same room with any operating fluoroscopic device to wear lead aprons. The strict observation of safe operating practices and the use of minimal fluoroscopic times is the key to reducing user exposure.

3 Regulations for X-ray Equipment

3.1 United States Federal and State Regulations

InSight FD Mini C-arm Imaging Systems conform to the United States Code of Federal Regulations, 21 CFR 1020.32 “Performance Standards for Ionizing Radiation Emitting Products: Fluoroscopic Equipment.” However, additional state regulations may apply to X-ray products. This may require notifying your state regulatory agency about your installation and/or operator training and certification.

It is the responsibility of the user to contact the applicable state radiation control agency to verify that your installation is in compliance with regulations in your state governing installation and use of X-ray equipment.

The InSight FD Mini C-arm Imaging System complies with 21CFR Subchapter J.

The InSight FD Mini C-arm Imaging System is intended for special surgical applications that would be prohibited at the Source to Skin Distances (SSD) specified in 21CFR 1020.32 (g) but are permitted under 21CFR 1020.32 (g) (2). The SSD is 10 cm minimum.

3.2 IEC Regulations

InSight FD Mini C-arm Imaging Systems comply with the requirements of IEC 60601-1. The classification of the InSight FD Mini C-arm Imaging Systems under this specification is Class 1, Type B.

The InSight FD Mini C-arm Imaging Systems comply with the following IEC standards:

IEC/EN 60601-1	IEC/EN 60601-1-3	IEC/EN 60601-2-28
IEC/EN 60601-1-2	IEC/EN 60825-1	IEC/EN 60601-2-54

3.3 Canadian Regulations

The InSight FD Mini C-arm Systems comply with CSA C22.2 No. 601.1

4 EMI

This instrument is designed to be compatible with the electromagnetic environments specified in IEC60601-1-2 and will operate satisfactorily when placed in an environment that includes other equipment complying with that standard. If equipment that is not electromagnetically compatible is placed into the environment where the InSight FD Mini C-arm System is used, the InSight FD Mini C-arm System or the non-compliant equipment may malfunction or exhibit other anomalous behavior. Consult the appropriate field service engineer.

5 Accessories


This instrument is supplied with hardware and software accessories that are needed for safe and reliable operation. Additional accessories may be used to perform optional procedures. Do not use any accessories with this instrument other than those approved by Hologic.

6 InSight FD System Labels

6.1 System Main Label

The System Main Label is on the back of the base cabinet.

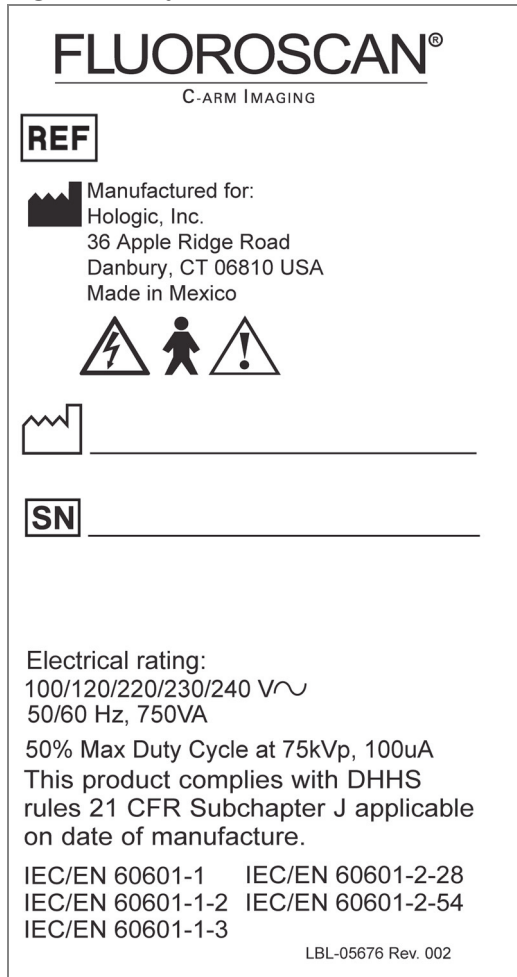
Figure 2 System Main Label



6.2 X-ray System Warning Label

The X-ray System Warning Label is on the front of the base cabinet below the Computer On/Standby switch.

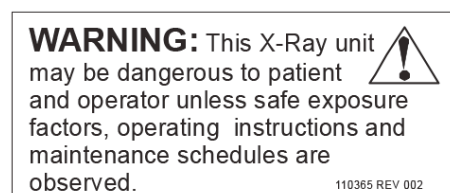
Figure 3 X-ray System Warning



6.3 Electric Shock Warning Label

The Electric Shock Warning Label is located near the fasteners for panels covering high voltage components.

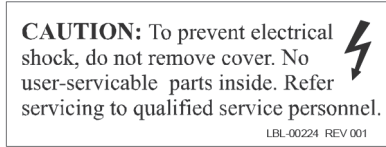Figure 4 Electric Shock Warning



6.4 Field of View

Field of View is represented by raised rectangles on the underside of the High Voltage Power Supply. When the Field of View selection lever is positioned at the large rectangle, full field of view is selected. When the Field of View selection lever is positioned at the small rectangle limited field of view is selected.

Figure 5 Field of View



7 Symbols

Table 1 Symbols































	CSA listed device		CE marked device
	Dangerous voltage		Attention - Read the Caution or Warning statement that follows
	Type B equipment		X-ray filtration
	Time delay fuse		Protective ground
	Functional ground		Equipotential ground

Table 1 Symbols (Continued)

	USB connector		Electrostatic sensitive device
	WEEE dispose of electrical/ electronic equipment properly		Computer On/Standby switch
	Rear wheels free to move in all directions		Rear wheels move forward and back only
	Rear wheels locked		Manufacturer name and location
	Manufacturing date		Name and address of authorized representatives
	Laser on switch		X-ray on switch
	Store		Print
	Increase or decrease kV/mA		Rotate image
	Cycle through the noise suppression settings		Serial number
	Part number		AC Voltage

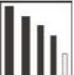8 Safety Hazards

The following table itemizes various safety hazards. Each hazard is cross referenced with the respective cause and proper course of action.

Table 2 Safety Hazards




Safety Hazard	Cause	Course of Action
Radiation Safety	 <p><i>This system produces ionizing radiation and may be dangerous to patients and operators unless the safety and operating instructions in this manual are followed.</i></p>	<p>Although operating this system can result in low scatter radiation levels, it is recommended that protective X-ray clothing be worn and radiation safety precautions be observed when operating the system.</p> <p>Operators should avoid placing their hands or any part of their bodies in the path of the X-ray beam.</p> <p>When making X-ray exposures, place the anatomy close to the CMOS detector for the best image quality and to reduce exposure to radiation. An alarm sounds after every five minutes of accumulated exposure time.</p>
Electrical Safety	 <p><i>Electrical circuits inside the system can cause serious injury or death.</i></p> <p><i>This system operates on high voltages. High voltages are at the following locations: X-ray tube: 75,000 Volts.</i></p>	<p>Safety covers have been designed for this system to mitigate the hazards caused by electrical circuits and high voltages. Never remove the covers or operate the system while the covers are off. If any cover is off, request service before using the system.</p> <p> <i>Unplugging the power cord from the wall is the only way to disconnect main power from this system.</i></p>

Table 2 Safety Hazards




Safety Hazard	Cause	Course of Action
<p>Tipping Hazard</p>	<p>Placing too much weight on the C-arm CMOS detector may cause the system to tip over. The amount of weight that can be supported by the CMOS detector is highly variable, depending on the position of the flex-arm, C-arm, and other factors. This hazard is greatest when the flex-arm is extended and moved to the far left or right of the system.</p>	<p>If weight is to be applied to the C-arm CMOS detector, caution must be used to prevent the device from becoming unstable.</p> <p> <i>Do not place the full weight of the lower extremity directly on the CMOS detector.</i></p>
<p>Imaging Function Loss</p>	<p>Detector damage from surgical tools (i.e., drills coming into contact with the detector surface)</p>	<p> Do not use the C-arm CMOS detector as a working surface.</p>
<p>Unauthorized Modifications to X-ray Tube</p>	<p> <i>Never remove the housing around the X-ray tube or make any changes to the X-ray tube.</i></p> <p>When the product leaves the manufacturing plant and at the time of installation, the X-ray source and assembly meet the requirements of 21 CFR 1020.30-32. Any unapproved change to the system could cause death, injury or expensive damage to the equipment, and could void the service agreement.</p>	<p>Do not make any changes to the X-ray tube. Contact a Hologic Representative to replace the X-ray tube.</p>

Table 2 Safety Hazards

Safety Hazard	Cause	Course of Action
Electrical Fire	Fire from any source.	<p>If there is a fire in the area, and it is safe to do so, turn the system off and unplug the power cord from the wall outlet.</p> <p>Do not touch a system that is on fire. Leave the area. Only use fire extinguishers approved for electrical fires.</p>
Explosion Hazard	The system is not designed for use in explosive atmosphere (e.g., in the presence of a flammable anaesthetics mixture with air or oxygen or nitrous oxide).	<p>If an abnormal condition occurs, such as the room filling with flammable gas, take steps to prevent the gas from contacting the equipment. Follow these guidelines:</p> <ul style="list-style-type: none"> • Do not turn the system ON or OFF, or unplug it from an AC power receptacle. • Do not operate any other electrically powered equipment. • Evacuate the area immediately. Ventilate the area with fresh air. • Avoid operating any automated (electrically operated) doors or windows. • Contact your local fire department as soon as possible.
Foot Switch Safety	The foot switch can be damaged by falling objects, being rolled over by the system wheels, and if it is dropped.	Remain aware of the foot switch at all times during operation and transport to avoid these hazards.

Table 2 Safety Hazards



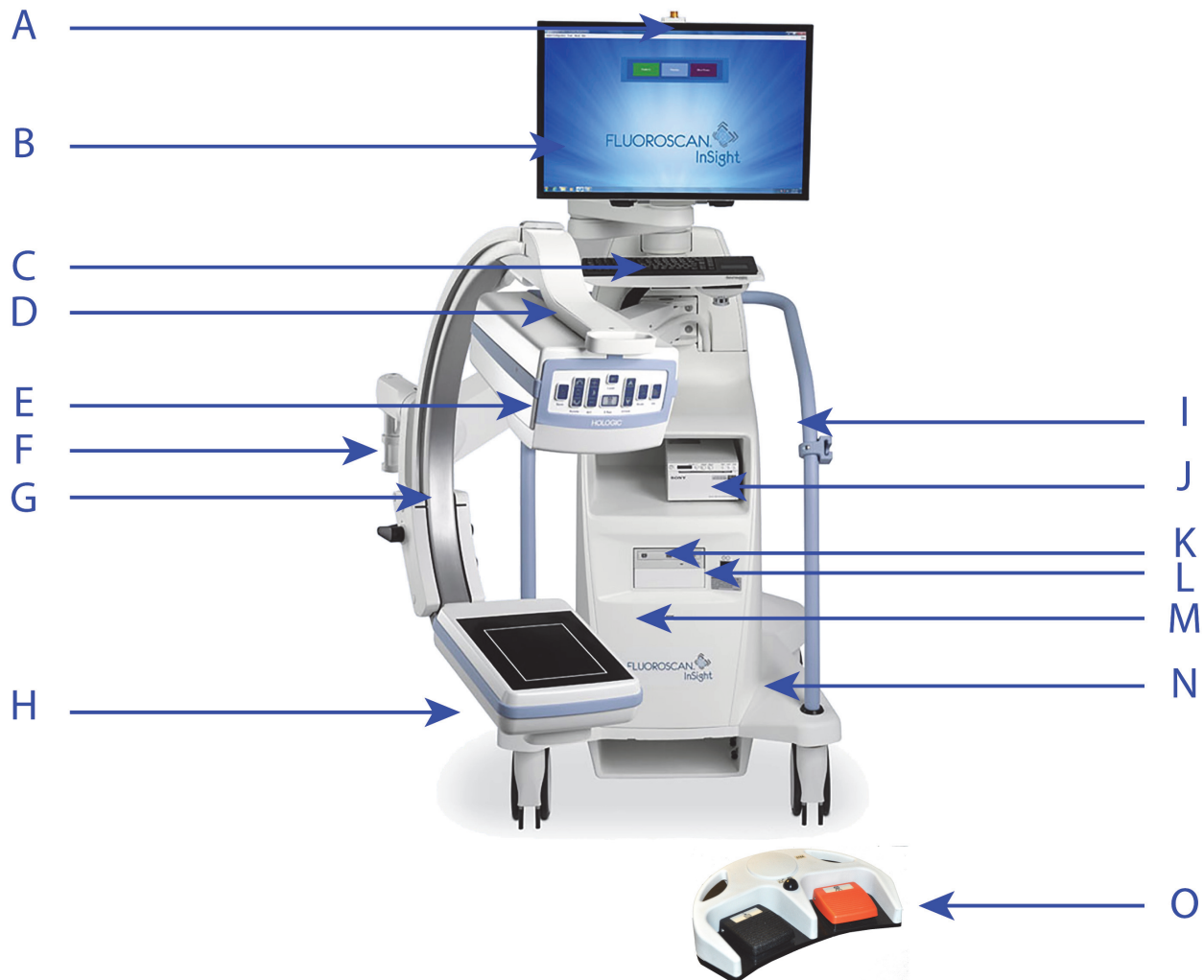
Safety Hazard	Cause	Course of Action
<p>Collision Hazard</p>	<p>If improperly moved or if the wheel brakes are not used correctly, the system could roll out of control. If the articulating flex arm is not locked in place during transport, the C-arm could swing out of control and cause injury to the patient or medical personnel.</p>	<p> <i>Use caution when moving the system on an incline.</i></p> <p>Do not move the system on an incline greater than ten degrees. One person can move the system easily on a level surface. Always lock the wheel brakes before procedures. Never leave the system unattended on an incline greater than five degrees. Always lock the flex arm after changing the position of the C-arm.</p>
<p>Cleaning Safety</p>	<p>Almost any liquid spilled into electronic equipment automatically becomes a conductor of electricity.</p>	<p>This system is not waterproof. If liquid is spilled on the system, turn the system off immediately. Do not turn the system on until the liquid has dried completely. Do not splash or spray cleaning solution on the system. Clean exterior surfaces of the InSight FD Console, the flex-arm, and the C-arm with a soft cloth or disposable lint-free tissue moistened with water, isopropanol or methanol. Wipe surfaces; do not scrub. For additional detail on disinfectants, see Section 14.3.1 Recommended Disinfectants on Page 72.</p>

Table 2 Safety Hazards

Safety Hazard	Cause	Course of Action
<p>Device Safety</p>	 <p><i>Injury could result if unapproved devices are connected to the system.</i></p> <p>Certain devices were tested for use with the InSight FD System, use only these or compatible devices.</p>	<p>Do not connect any devices to the system other than those described in this manual. Internally connected devices should be installed only by persons trained in the servicing of this system.</p>
<p>Electrosurgical Device</p>	<p>This system is shielded from electrical discharges. However, there is still a possibility that an electrosurgical device discharged directly into the system could adversely affect system operation. In addition, shielding might not always protect the system from the amount of energy produced by a spark-gap type device.</p>	<p>If a strong electrosurgical surge causes the system to malfunction: Turn the system off Wait five seconds. Turn the system on. If the problem persists after resetting the system twice, request service.</p>

9 System Components



A	X-ray ON light	I	C-arm retainer
B	Touch screen monitor	J	Image printer shelf
C	Keyboard with integrated pointing device	K	CD/DVD RW recorder
D	X-ray head	L	Computer On/Standby switch
E	Collimation switch	M	USB port (behind the CMOS detector)
F	Flex-arm	N	Footswitch shelf
G	C-arm	O	Wireless Footswitch
H	CMOS Detector		

10 Controls and Indicators

10.1 X-ray Head Control Panel

Table 3 X-ray Head Control Panel










Symbol	Label	Function
	Save	Saves selected images or cine loop to the record of the patient. If the number of stored images or minutes of cine loop exceeds the storage limit, a message displays.
	Rotate	Rotates the image 10 degrees clockwise or counterclockwise.
	B/C	Allows adjustment of image brightness and contrast.
	X-ray	Acquires an X-ray exposure. Press switch once to actuate X-rays. A yellow indicator light located on top of the monitor shows that X-rays are being produced.
	Laser	Turns on the Positional Laser.
	Noise Suppression	Cycles through the Noise suppression settings during image acquisition.

Table 3 X-ray Head Control Panel

Symbol	Label	Function
	kV/mA	Increases or decreases technique factors based on selected mode (Table 13 Worklist Configuration, Section Descriptions on Page 30).  Pressing both the up and down power buttons at the same time returns the system to Auto Power mode.
	Mode	Acquisition Mode Selection cycles through: Continuous – Displays continuous images when X-rays are actuated. Snapshot – Captures and displays one image when X-rays are actuated. Cine Record – Gathers all images acquired into a cine loop. Continuous and/or Snapshot mode may be disabled (Section 12.5 Physician Preferences on Page 33).

10.2 Keyboard Controls

Table 4 Keyboard Controls

















Icon	Function Key	Function
	F1	Displays help window.
	F2	Displays the Patient Information window in Image Acquisition session. Displays the Select Study for Review window in Review session.
	F3	Flips image horizontally.
	F4	Rotates image 10 degrees clockwise.
	Shift-F4	Rotates image 10 degrees counterclockwise.
	F5	Decreases kV/mA if simultaneous is on or decreases kV if simultaneous is off. Pressing this key puts the system into manual mode.
	Shift-F5	Decreases mA if simultaneous is off. Pressing this key puts the system into manual mode.

Table 4 Keyboard Controls (Continued)

Icon	Function Key	Function
	F6	Increases kV/mA if simultaneous is on or increases kV if simultaneous is off. Pressing this key puts the system into manual mode.
	Shift-F6	Increases mA if simultaneous is off. Pressing this key puts the system into manual mode.
	F7	Resets fluoro alarm timer.
	F8	Saves the current image to the patient's record.
	F9	Prints the current image to the local printer.
	F10	Toggles Play back of a recorded cine loop on/off.
	F11	Restores physician's defaults.
	Shift-F11	Returns the system to Auto Power mode.
	F12	Cycles through the Noise suppression settings during image acquisition.

10.3 InSight FD System Footswitch Functions

The footswitch can be used for quick access to several software functions.

Figure 6 Wireless Footswitch



Table 5 Footswitch Pedal Functions

	Icon	Pedal	Function
A		Save / Tag	Save or Tag images
B		MegaView™ Image and/or Print	<ul style="list-style-type: none"> • In Perform Mode, you can toggle between standard view and full-screen view (MegaView™ Image) • Print images
C		X-ray	Take X-ray

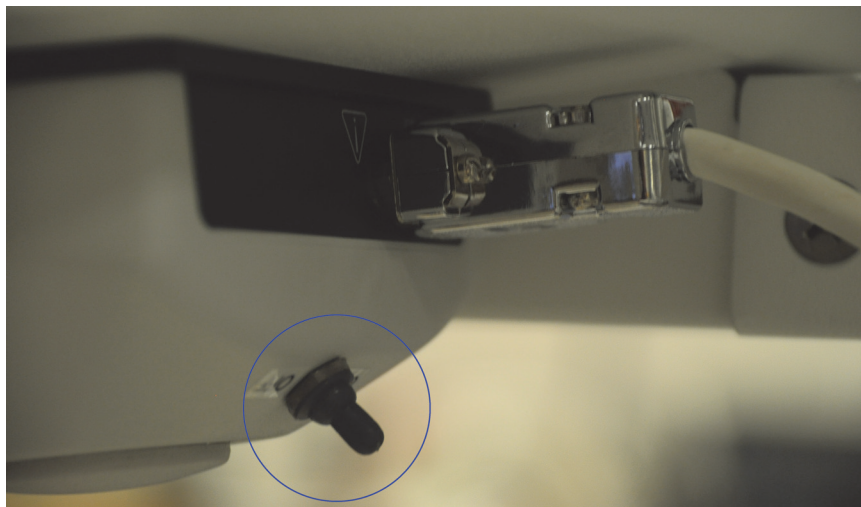


Note

*The receiver must be turned **ON** in order to use the footswitch functions.*

Specific footswitch functions must be enabled in the Physician's Preferences window options:

- *Tag References from Footswitch*
- *Toggle Full Screen View from Print Footswitch*



10.3.1 Acquire an X-ray Image

1. To acquire an image, press the **X-ray** pedal (C).
2. Press again to take subsequent images.

10.3.2 Display a MegaView Image

When in Perform Mode, press and release **MegaView Image and/or Print (B)** to toggle between **Standard** and **Full-Screen View**.

10.3.3 Tag an Image

To mark an acquired image as a reference image, press and release **Save or Tag (A)** once.

10.3.4 Save an Image

1. To save the image and deselect as a reference, press and release **Save or Tag (A)** a second time.
2. To save the image and mark as a reference, press and release **Save or Tag (A)** a third time.

10.3.5 Print an Image

To print an image, press and hold **MegaView Image and/or Print (B)**.

11 Using the System



Note

InSight FD operators should read the Radiation Safety section before operating the system.

11.1 Moving the System




Warning:

Before moving the system, the flex-arm and C-arm must be in the locked storage position to prevent tipping.







1. Ensure that the system is powered off.
See [Section 11.5 Powering Down the System](#) on Page 22.)
2. Unplug the power cord and wrap it around the cord guides on the back of the cabinet.
3. Store the foot switch in the foot switch shelf.
4. Flip the monitor forward and down.
5. Place the flex-arm and C-arm in the locked position ([Figure 7](#)) using the Transport Position Knob on the side of the C-arm housing.

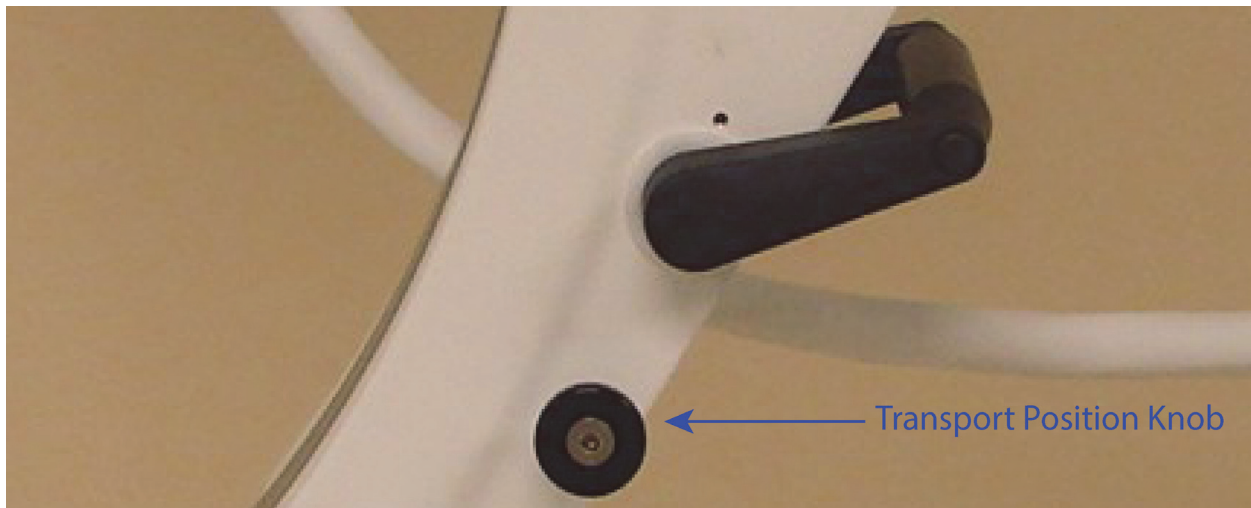Figure 7 Transport Position Knob



6. Lock the rear casters to prevent the system from moving. The swiveling angle of the rear casters can be locked to prevent lateral movement of the system or unlocked to allow lateral movement.

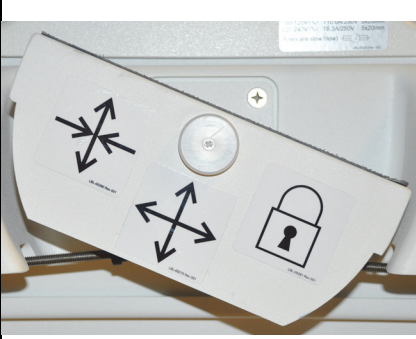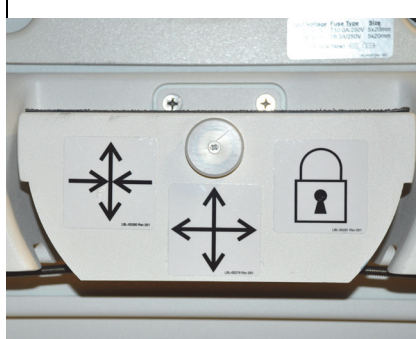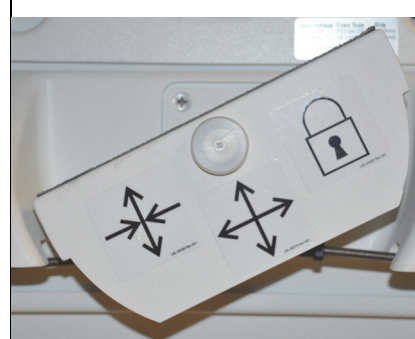Figure 8 Rear Casters

		
Rear Wheels Locked	Rear Wheels Move Forward and Back Only	Rear Wheels Free to Move in All Directions
		

11.2 Adjusting the C-arm


Turn the flex-arm lock counterclockwise to loosen; then adjust the flex-arm height.

To reposition the C-arm:

1. Turn the C-arm lock to loosen.
2. Roll the arm to the desired position.
3. Turn to lock the arm in place.

The locking lever can also be positioned between open and locked to provide various levels of resistance to arm roll.

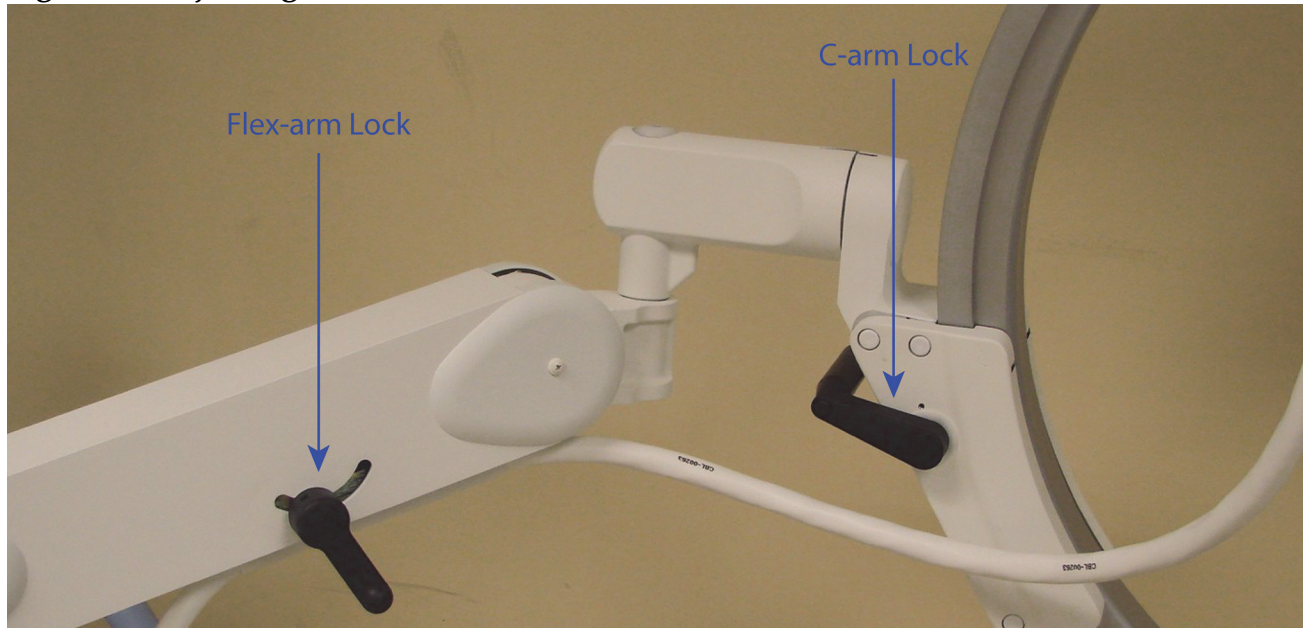Figure 9 Adjusting the C-arm




11.3 Sterile C-arm Drape

Before using the InSight FD Mini C-arm in the operating room, fit the C-arm with a sterile drape. Small labels on the drape indicate how to fit the drape to the X-ray head and to the CMOS detector.



Note *After use, dispose of sterile C-arm drape properly (in a biohazard container).*

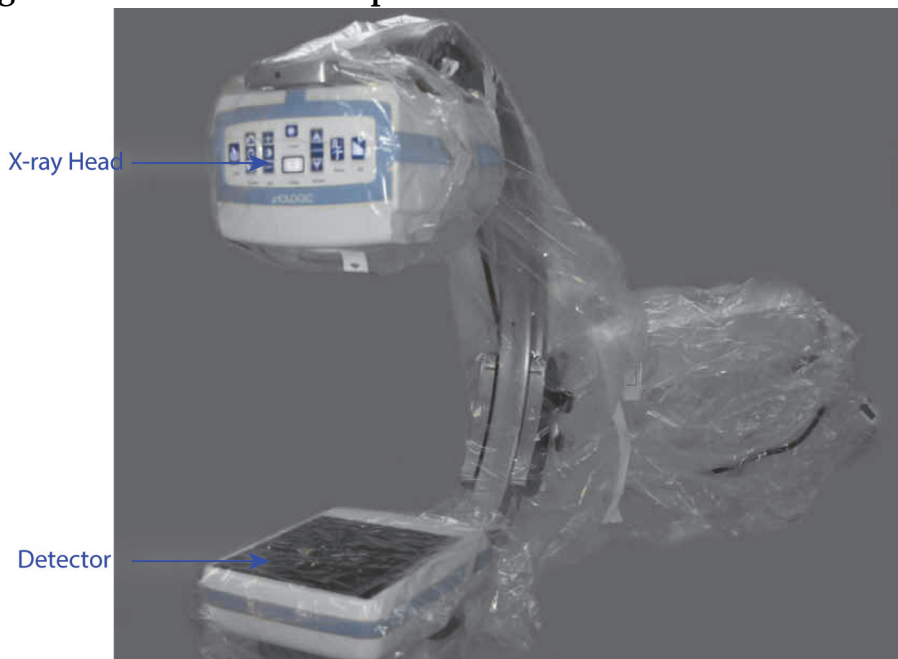Figure 10 Sterile C-arm Drape



11.4 Powering Up the System



Warning: *The InSight FD system operates on standard 115 Volt, 60 Hz AC power for United States, Japanese and Canadian models, 230 Volt, 60 Hz AC, and 240 volt, 50 Hz AC power for other export models.*

- The system should only be plugged into a properly grounded three-wire wall outlet.
- Never cut the third (grounding) pin from the power cord to plug the unit into an ungrounded two wire outlet. Never use a three wire-to-two wire adapter.

The system should be at ambient room temperature.

The power cord must be plugged into a standard single-phase three-wire grounded outlet. A special dedicated outlet is not required.

To power up the system, press the **Computer On/Standby** switch. InSight FD will prompt the user to log in.

If the system has been off for more than two weeks, use the following source conditioning procedure to safeguard and prolong the life of the X-ray source:

Table 6 Conditioning Schedule

Step	kV	mA	Minimum Time
1	40	0.05	2 minutes
2	50	0.10	2 minutes
3	60	0.10	2 minutes
4	70	0.10	3 minutes
5	75	0.10	3 minutes



Caution *The system must warm up for FOUR MINUTES before generating X-rays. Failure to provide adequate warm up could result in damage to the system.*

11.5 Powering Down the System

To close the application and place the system in standby:

- From the **InSight FD Main Screen** click the **Shut Down** button.
- From the **Exit Menu** on the **InSight FD Main Screen**, click **Exit** before unplugging the power cord.

12 Configuring the Application

12.1 System Configuration User Interface

System configuration user interface consists of a multiple-page tab-selected window. Each tab displays a page of configuration data.

Four buttons display at the bottom of the window that are independent of the selected page:

Table 7 System Configuration User Interface

Button	Function
OK	Saves data for all System Configuration pages and closes the window.
Apply	Saves data for the currently selected System Configuration page.
Cancel	Closes the window without saving any data.
Help	Displays help for the selected page.

12.2 System

From the **System Configuration Menu** (Figure 11) on the **Main Screen**, the **System** page (Figure 12) is accessible.

Figure 11 System Configuration Menu

System Attributes
DICOM Send
DICOM Print
Worklist
Users
Physician Preferences
Administrative Settings

Figure 12 System

The screenshot shows the 'System Configuration' dialog box for Fluoroscan InSight Software. The dialog has a title bar and a breadcrumb trail: System | DICOM Send Options | DICOM Print Options | Worklist Configuration | Users | Physician Preferences | Administrative Settings |. The main area is divided into several sections:

- Fluoroscan InSight Software:** Serial Number: 10-12345; Software: Fluoroscan InSight Version 6.1 (dropdown).
- Status:** Date of Last System Recover: N/A; Touch Screen: ON; Audit Mode: ON; Current User: Hologic; Number of Active Users: 1; Image Storage (number of images): Max Storage: 10000, Currently Stored: 11; Cine loop storage (minutes of recording): Max Storage: 40, Currently Stored: 1.
- Organization Name:** (empty text field).
- Default Physician's Name:** dr test (dropdown).
- Selected Language:** English (dropdown).
- Query on Delete:** Display warning message on deletion of images.
- Brightness / Contrast Adjustment:** Seconds to display dialog: 4 (spin box).
- DICOM Application Information:** AE Title (empty text field); Station Name (empty text field); IP Address: 10.0.2.15.

At the bottom right, there are four buttons: OK, Cancel, Apply, and Help.

Table 8 System Page

Section/Field	Description
InSight FD Software Serial Number Software	InSight FD system serial number. dropdown list to display installed software and version.
Status	Displays the following current settings:
Date of Last System Recover	Date application settings and configuration data were recovered from backup media.
Touch Screen	Enabled/Disabled.
Audit Mode	On/Off.
Number of Active Users	Number of active registered users allowed to log into the system.
Image Storage (number of images)	Maximum number of images that can be stored on the system and the number currently stored.
Cine Loop Storage (minutes of recording)	Maximum minutes of recording that can be stored on the system and the minutes currently stored.
Organization Name	Administrator can enter name.
Default Physician's Name	Select default physician to be used by the application.
Selected Language	Select language to be used by the application.
Query on Delete	Administrator can select to display warning message when deleting images.
Brightness/Contrast Adjustment	Sets the amount of time (from 1 to 60 seconds) that the Brightness/Contrast adjustment dialog will be displayed.
DICOM Application Information	AE Title and Station Name for the application. IP Address displays the network address assigned to the system the application is running on. Station Name allows multiple systems to use the same AE Title.

12.3 DICOM Send Options

Available from **System Configuration** Menu (Figure 11) on the **Main Screen**.

Figure 13 DICOM Send Options

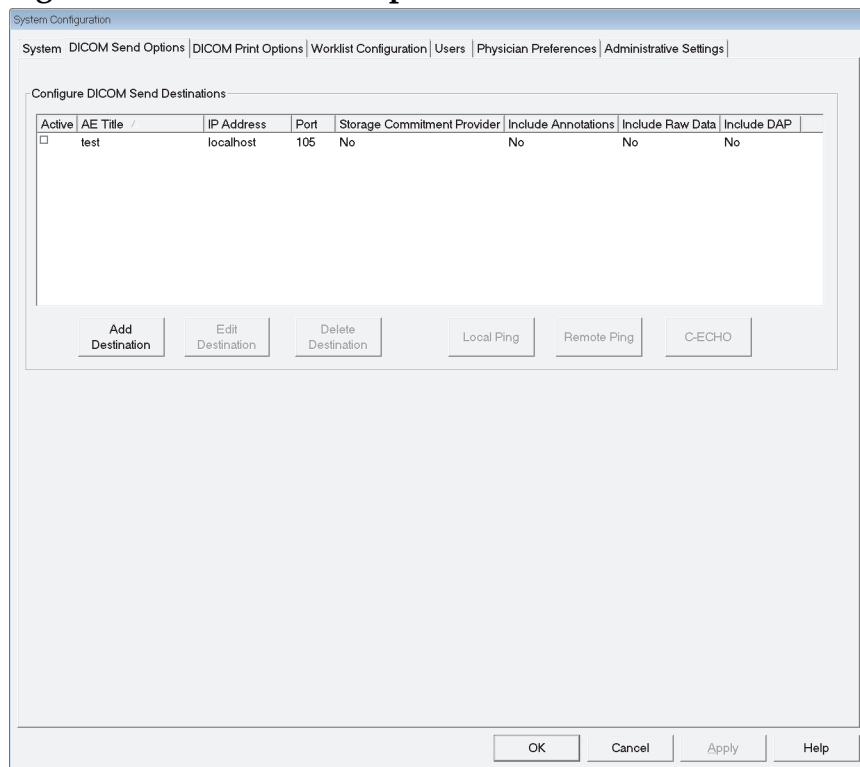


Table 9 DICOM Send Options Page

Section/Field	Description
Configure DICOM Send Destinations	Selection list of DICOM Send destinations configured on the system and six function buttons. When checked, the Active column check box enables destination for use.
Add/Edit Destination	Entry/edit fields to identify a DICOM Send destination, three check boxes, and two function buttons.
AE Title	Destination's Application Entity Title.
Host Name or IP Address	Destination's Name or IP address.
Port	Destination's Port number.

Table 9 DICOM Send Options Page

Section/Field	Description
Use as Storage Commitment Provider	When checked, indicates the destination is committed to storing information sent to it. During a normal shut down, if any DICOM Commit queue entries are greater than 7 days old, a message box appears indicating: <i>DICOM Storage Commitment Queue contains entries older than 7 days. DO you want to delete them?</i>
Include Raw Data	When checked, indicates studies sent to the destination shall include the raw data.
Include Annotations	When checked, includes annotations with exported images. Method of inclusion choices are: Embedded - Annotations are embedded into the bitmap image. As Overlay -Annotations are included as a DICOM overlay. As annotation files - Annotations are included as a separate file.

Table 10 DICOM Send Options Page, Button Functions

Button	Function
Add Destination	Adds a DICOM Send destination to the system.
Edit Destination	Edits selected destination information.
Delete Destination	Deletes selected destinations from the system.
Local Ping	Confirms connection to the local network.
Remote Ping	Confirms network connection to the selected destination.
C-ECHO	Confirms network connection to PACS.
OK	Updates the selection list with new or updated information and checks the destination's Active check box.
Cancel	Ignores all edits and hides the Add/Edit Destination section.

12.3.1 DICOM Print Options

Available from System Configuration Menu (Figure 11) on the Main Screen.

Figure 14 DICOM Print Options

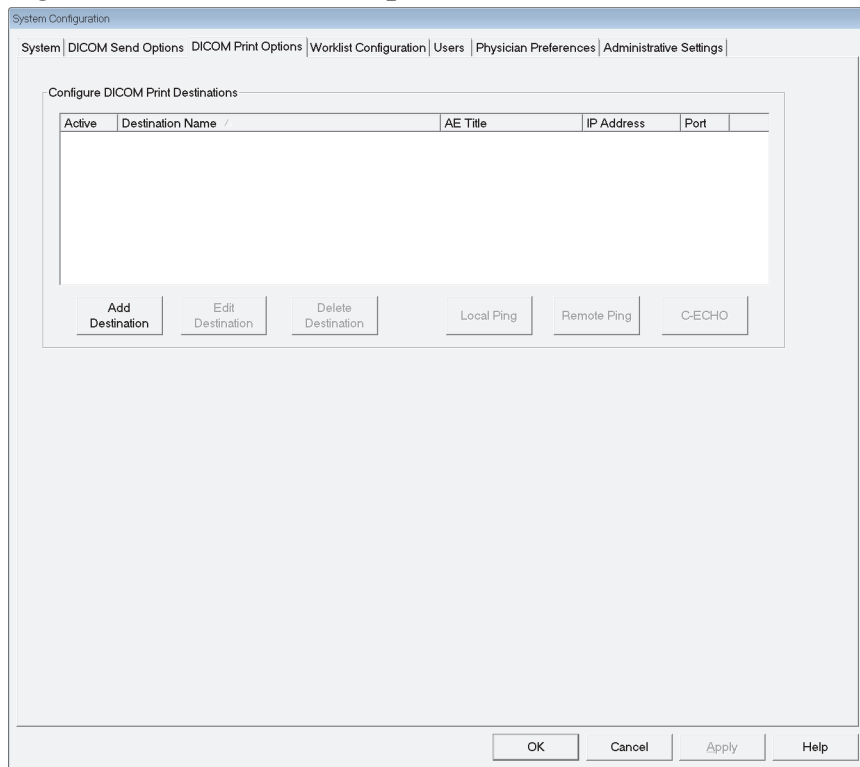


Table 11 DICOM Print Options Page, Section/Field Descriptions

Section/Field	Description
Configure DICOM Print Destinations	Selection list of DICOM Print destinations configured on the system and six function buttons. When checked, the Active column check box enables destination for use.
Add/Edit Destination	Entry/edit fields to identify a DICOM Print destination, a section for entering and selecting print parameters, and two function buttons.
AE Title	Destination's Application Entity title.
Host Name or IP Address	Destination's Name or IP address.
Port	Destination's Port number.
Destination Name	Name for destination (allows a single destination to be configured in multiple ways).

Table 11 DICOM Print Options Page, Section/Field Descriptions

Section/Field	Description
Print Parameters	
Number of Copies	Enter number of copies to print.
Image Display Format	Select (row and column) from dropdown list.
Orientation	Select image layout (Portrait or Landscape).
Film Size	Select from dropdown list.
Medium Type	Select from dropdown list.
Decimate/Crop Behavior	Select from dropdown list.

Table 12 DICOM Print Options Page, Button Functions

Button	Function
Add Destination	Adds a DICOM Print destination to the system.
Edit Destination	Edits selected destination information.
Delete Destination	Deletes selected destinations from the system.
Local Ping	Confirms connection to the local network.
Remote Ping	Confirms network connection to the selected destination.
C-ECHO	Confirms network connection to PACS.
OK	Updates the selection list with new or updated information and checks the destination's Active check box.
Cancel	Ignores all edits and hides the Add/Edit Destination section.

12.4 Worklist Configuration

Available from **System Configuration** Menu (Figure 11) on the **Main Screen**.

Figure 15 Worklist Configuration

Table 13 Worklist Configuration, Section Descriptions

Section/Field	Description
Worklist Provider	Identifies the system to access for the Worklist by providing entry fields for the provider's AE Title, Host Name or IP Address, and Port.
Use Performed Procedure Step	When checked, DICOM Modality Performed Procedure Step shall be performed when required.
Provider	Allows the user to access a system other than the one identified in the Worklist Provider section. When <i>Use Worklist Provider Data</i> is checked, AE Title, Host Name or IP Address, and Port fields are automatically populated with data identical to the contents of the corresponding fields in the Worklist Provider section. When unchecked, the fields can be populated by the user.

Table 13 Worklist Configuration, Section Descriptions

Section/Field	Description
Query Parameters	Provides entry fields to identify: <ul style="list-style-type: none"> • Requesting system (AE Title) • Type of worklist being requested (Modality) • Range of days to query • Maximum number of scheduled patient examinations to include in the worklist
Auto Query Interval	Specifies if automatic queries are to be performed and how often.
Query Retry Parameters	Specifies if queries are to time out after a specific time (Query Timeout check box and Min selection), and number of times to retry and time intervals.
Worklist sets Modality	When checked, Modality is set by the Worklist.

Table 14 Worklist Configuration, Button Functions

Button	Function
Map Worklist Fields	Displays Map Worklist Keys window. Administrators can map InSight FD data fields to Worklist provider data fields.
Local Ping	Confirms connection to the local network and display the results.
Remote Ping	Confirms network connection to the Worklist Provider or Provider system.
C-ECHO	Confirms network connection to the PACS.

12.4.1 Users

Available from **System Configuration Menu** (Figure 11) on the **Main Screen**.

Figure 16 Users

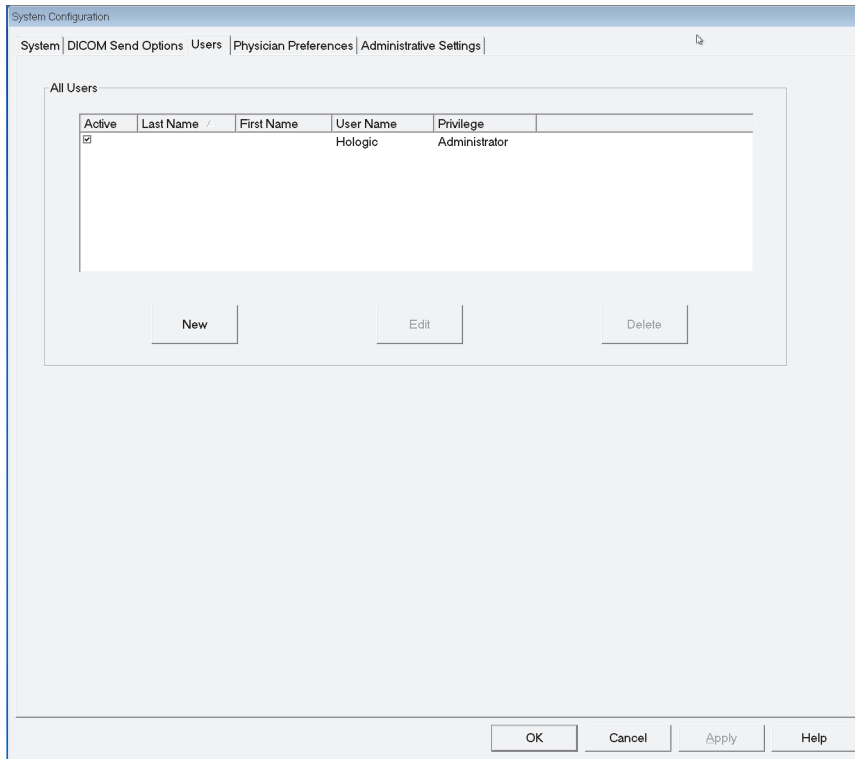


Table 15 Users Page, Section/Field Descriptions

Section/Field	Description
All Users	Displays selection list of registered system users. When checked, the Active column check box allows user to log into the system.
Add/Edit User	Administrator entry/edit fields for user's actual name, login user name, login password, and privilege.

Table 16 Users Page, Button Functions

Button	Function
New	Adds new users to the system.
Edit	Edits selected user information.
Delete	Deletes the selected user from the system.
OK	Updates the user list.
Cancel	Ignores all edits and removes the Add/Edit User section.

12.5 Physician Preferences

Available from **System Configuration Menu** (Figure 11) on the **Main Screen**.

Figure 17 Physician Preferences

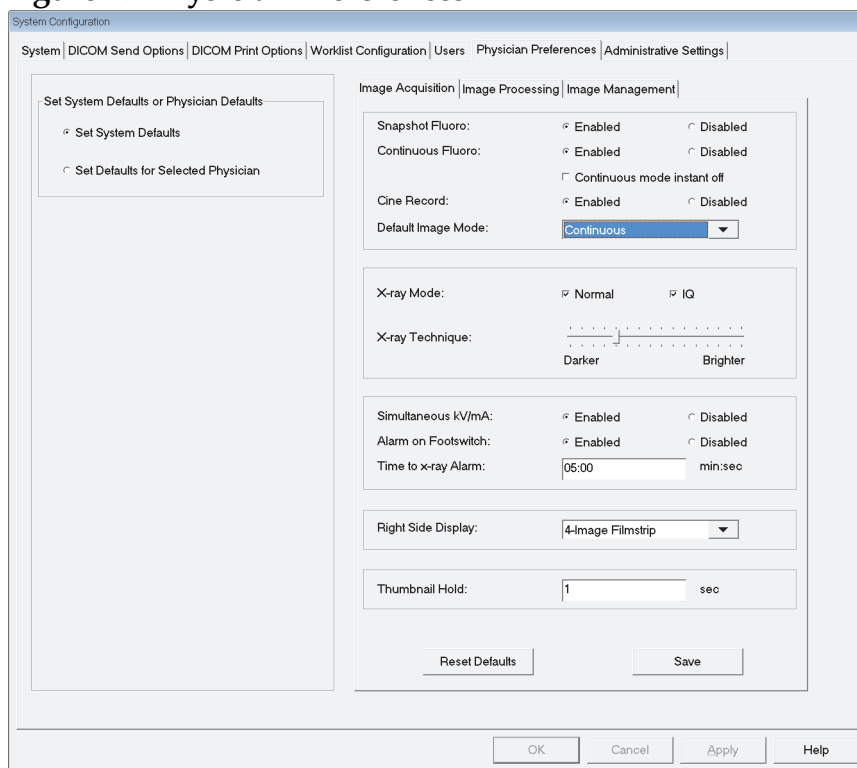


Table 17 Physician Preferences, Section/Field Descriptions

Section/Field	Description
Set System Defaults or Physician Defaults	Select system defaults to be used for visiting physician. Select physician defaults to set defaults for a selected physician.
Select Physician to set details for	Selection list of all physicians registered on the system, entry fields for physician's name, and two buttons (Add Physician and Delete Physician).

Table 18 Physician Preferences, Button Functions

Button	Function
Add Physician	Adds the physician's name and default preferences to the system.
Delete Physician	Deletes the selected physician from the system.
Reset Defaults	Resets all the preference settings to the factory defaults if editing system defaults, or to the system defaults if editing physician preferences. If system defaults have not yet been saved, all settings are reset to the factory defaults.
Save	Saves the preference settings for the System Defaults or for the selected physician.

12.6 Physician Preferences — Image Acquisition

Displays the current default preference settings for the system or the selected physician. Settings can be customized by an administrator.

Figure 18 Physician Preferences — Image Acquisition

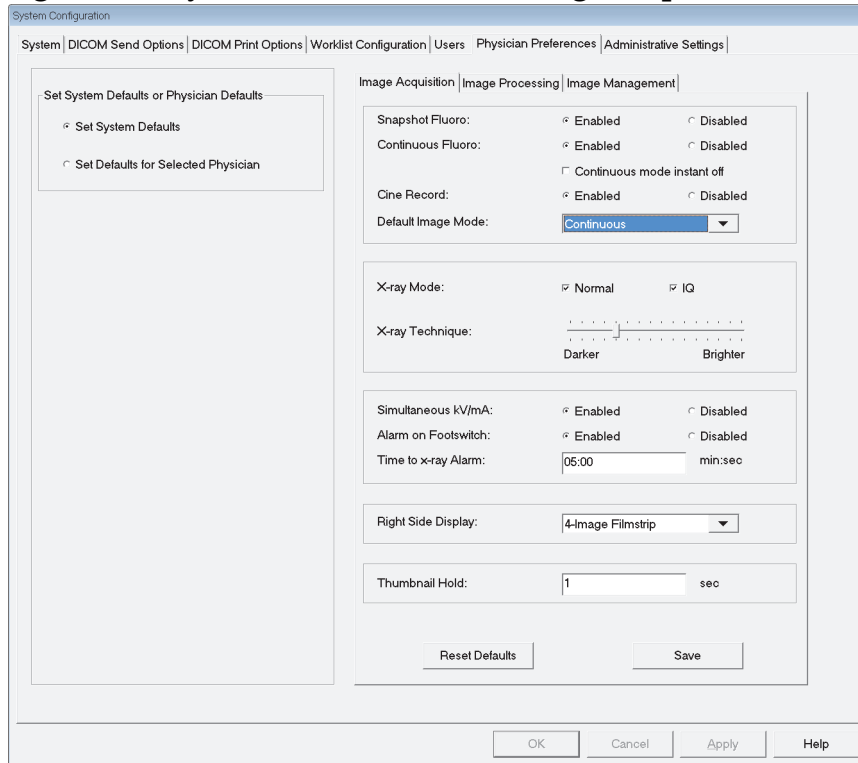


Table 19 Physician Preferences, Image Acquisition Tab, Section/Field Descriptions

Section/Field	Description
Snapshot Fluoro	Enables/disables Snapshot image acquisition mode.
Continuous Fluoro	Enables/disables Continuous image acquisition mode.
Continuous mode instant off	When checked and acquiring images in Continuous mode, X-rays remain on until X-ray button or pedal is released. When unchecked, tapping the X-ray button or pedal causes image acquisition to behave like Snapshot mode; holding the button or pedal pressed cause X-rays to remain on until released.
Default Image Mode	Selects initial image mode setting for Image Acquisition screen.
X-ray Mode	Selects X-ray power level. Check Normal, IQ, or both. You must check at least one mode or accept the system default. If you uncheck both modes, the system enables Normal mode. The system default enables Normal and IQ modes.
X-ray Technique	Slider control with 15 selectable settings from “Darker” to “Brighter” for preferred selection of the kVp and mA under automatic exposure control. Changing this value to “Darker”/ “Brighter” will marginally decrease/increase the kVp and mA chosen by the automatic exposure control and, as a consequence, the corresponding dose to the patient.
Simultaneous kV/mA	When enabled, kV / mA goes up/down together to the next dose curve pair when manually adjusting X-ray power. When disabled, kV and mA can be adjusted independently.
Alarm on Footswitch	Enables or disables setting of the alarm to sound when the X-ray footswitch is depressed.
Time to X-ray Alarm	Sets when the alarm will sound based on the accumulated X-ray exposure time. Maximum value is 5 minutes.
Right Side Display	Selects initial filmstrip/tools area setting for Image Acquisition screen.
Thumbnail Hold	Determines the time a thumbnail image is displayed after the acquisition ends.

12.7 Physician Preferences — Image Processing

Displays the current default preference settings for the system or the selected physician. Settings can be customized by an administrator.

Figure 19 Physician Preferences — Image Processing

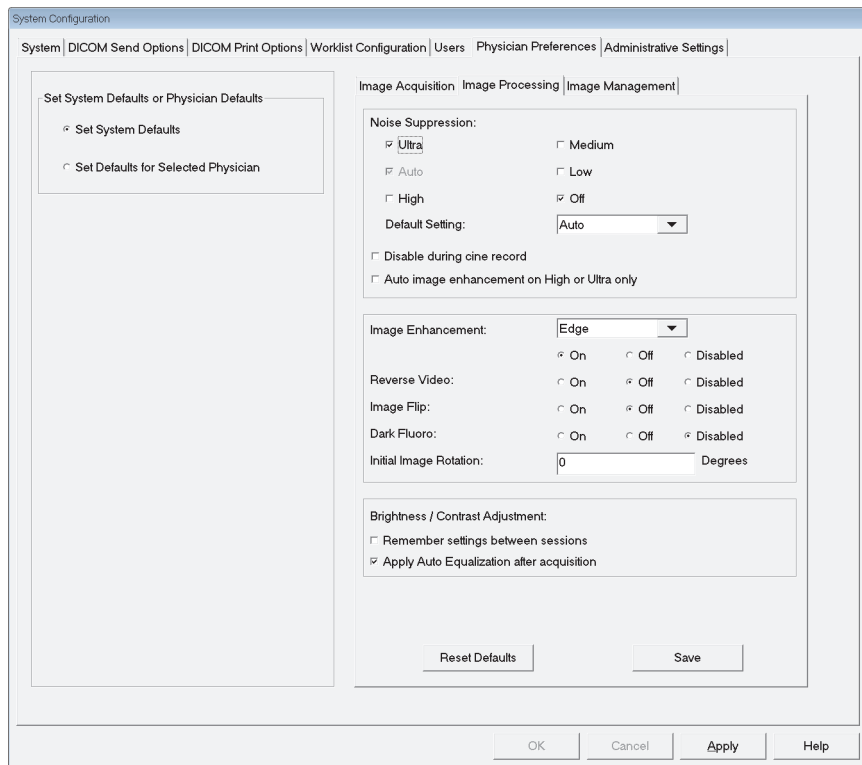


Table 20 Physician Preferences, Image Processing Tab, Section/Field Descriptions

Section/Field	Description
<p>Noise Suppression</p>	<p>Select noise suppression for Image Acquisition screen (Ultra, Auto, High, Medium, or Low). Disable during cine record - When checked, disables noise suppression when cine recording. Auto edge enhancement on High or Ultra only - When checked, image enhancement occurs only when noise suppression is set to High or Ultra. Default Setting defines the initial noise suppression for an image acquisition session. Select Ultra, Auto, High, Medium, Low, or Off from the dropdown list.</p>
<p>Image Enhancement</p>	<p>Select Edge or Sharpen from the dropdown list. Edge enhancement increases visualization between bone and soft tissue in the image. Sharpening enhancement increases visualization of fine detail. On/Off or Disabled affects the “image effects settings” in the Image Acquisition screen.</p>
<p>Reverse Video</p>	<p>Select On/Off or Disabled, which affects the “image effects settings” in the Image Acquisition screen.</p>
<p>Image Flip</p>	<p>Select On/Off or Disabled, which affects the “image effects settings” in the Image Acquisition screen.</p>
<p>Dark Fluoro</p>	<p>Select On/Off or Disabled, which affects the “image effects settings” in the Image Acquisition screen.</p>
<p>Initial Image Rotation</p>	<p>Enter number of degrees to automatically rotate an image while it is being acquired. Enter a negative number for counterclockwise rotation.</p>
<p>Brightness/ Contrast Adjustment</p>	<p>Remember settings between sessions - When checked, maintains Brightness/Contrast settings for subsequent images. Apply Auto Equalization after acquisition - When checked, applies Auto Equalization to the image to optimize contrast.</p>

12.8 Physician Preferences — Image Management

Displays the current default preference settings for the system or the selected physician. Settings can be customized by an administrator.

Figure 20 Physician Preferences — Image Management

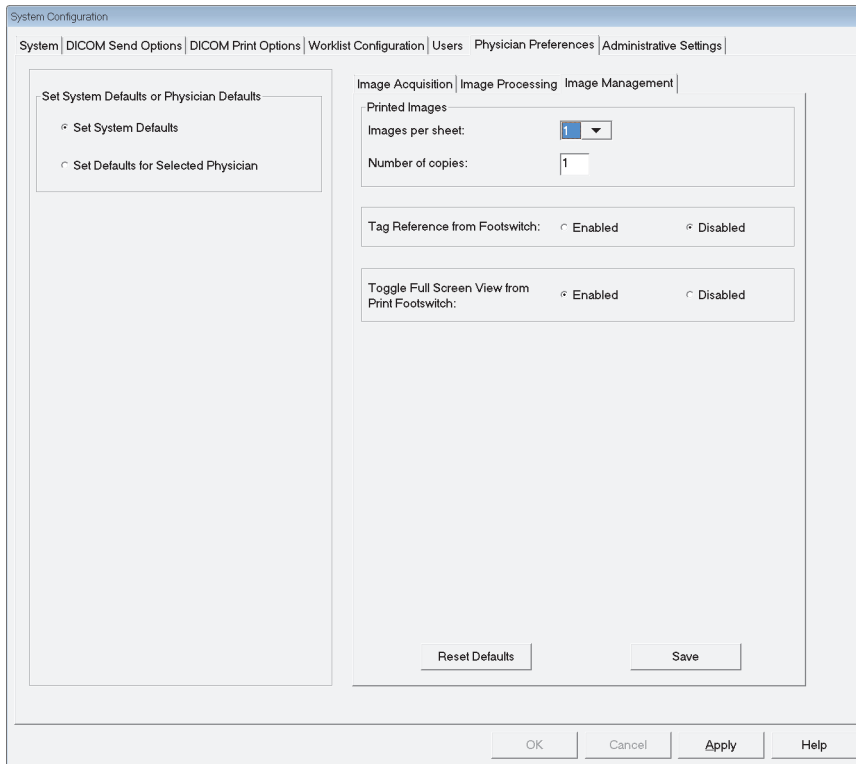


Table 21 Physician Preferences, Image Management Tab, Section/Field Descriptions

Section/Field	Description
Printed Images	Select Number of Images per sheet and Number of copies to print.
Tag Reference from Footswitch	<p>When Enabled: Press the Save footswitch once to tag the image as a reference image. Press Save a second time to save the image and remove the reference tag. Press Save a third time to re-tag the image as a reference image.</p> <p>When Disabled: Press the Save footswitch once to save the image (does not tag the image as a reference image).</p>

12.9 Administrative Settings Page

Available from System Configuration Menu on the Main Screen.

Table 22 Administrative Settings, Section Descriptions

Section/Field	Description
Log Files	Selection list of all log files.
Set Level of Detail	Administrators can set System Error log level of detail to Error Only or Error and Operations.
Audit Mode	Administrators can set on or off.
Enable Touch Screen	Administrators can enable or disable.
Automatic Image Deletion	Check Delete at the following limits to: Establish the number of images at which a deletion prompt is displayed and the number of images to be deleted and Establish the number of cine minutes at which a deletion prompt is displayed and the number of cine minutes to be deleted. Prior to deletion, the system displays the number of studies required to achieve the desired number of image, or cine minute, deletions.
Last System Backup	Displays date of last system configuration and calibration backup. Display warning if last backup is older than __ days - When checked, upon starting the application the system will display a warning message when the last backup is older than the given number of days.
System Shutdown	Administrators can check or uncheck. When checked, the full system shuts down (default setting). When unchecked, only the InSight FD application shuts down.

Table 23 Administrative Settings, Button Functions

Button	Function
Configure	Displays Level of Detail section.
Accept Change	Changes detail level for System Error log to that selected by Error Only or Error and Operations button. Hides the Set Level of Detail section.

13 Using the InSight FD Application

You can select buttons, menus, menu functions, icons, and position the insertion cursor in fields by

- Positioning the cursor on the respective item and clicking the left mouse button, or
- Touching the item on the touch screen if enabled.

In this manual “click” indicates both ways.

The main screen appears if **Audit Mode** is on. If Audit mode is off, the **Image Acquisition** screen is displayed.

13.1 Main Screen

Figure 21 Main Screen



Table 24 Main Screen Options

A	System Configuration Menu (Table 25).
B	Tools Menu (Table 26)
C	About - Displays InSight FD application’s version number, serial number, and copyright data.
D	Exit Menu. Allows the user to logout or exit the application.
E	Minimize the screen
F	Maximize the screen

Table 24 Main Screen Options

G	Close the Application screen (unavailable)
H	Help - Displays help for using the screen.
I	Starts an image acquisition session (Section 13.2 Image Acquisition Session on Page 47)
J	Starts an image review session (Section 13.3.2 Review Screen on Page 57)
K	Exits application and shuts down the system.

Figure 22 System Configuration Pull-down Menu

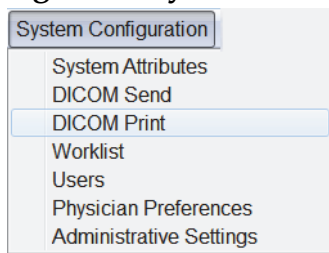


Figure 23 System Configuration Menu

The screenshot shows the 'System Configuration' window with the following sections and values:

- Fluoroscan InSight Software:** Serial Number: 10-12345; Software: Fluoroscan InSight Version 6.0
- Status:** Date of Last System Recover: N/A; Touch Screen: ON; Audit Mode: ON; Current User: Hologic; Number of Active Users: 1
- Image Storage (number of images):** Max Storage: 10000; Currently Stored: 11
- Cine loop storage (minutes of recording):** Max Storage: 40; Currently Stored: 1
- Organization Name:** (Empty text field)
- Default Physician's Name:** dr test
- Selected Language:** English
- Query on Delete:** Display warning message on deletion of images
- Brightness / Contrast Adjustment:** Seconds to display dialog: 4
- DICOM Application Information:** AE Title: (Empty text field); Station Name: (Empty text field); IP Address: 10.0.2.15

Buttons at the bottom: OK, Cancel, Apply, Help

Table 25 System Configuration Menu

Function	Description
System Attributes	Administrators can enter organization name, select default physician's name to be used by the application, select default language, enter DICOM application information and check if warning message should display when deleting images. All users can view system information. (Section 12.2 System on Page 23)

Table 25 System Configuration Menu (Continued)

Function	Description
<p>DICOM Send</p>	<p><i>Present if the DICOM Send option is installed.</i> Administrators can add, edit, delete, and activate/deactivate Send destinations. All users can view the list of destinations, view destination information, and ping destinations. Section 12.3 DICOM Send Options on Page 26</p>
<p>DICOM Print</p>	<p><i>Present if the DICOM Print option is installed.</i> Administrators can add, edit, delete, and activate/deactivate Print destinations. All users can view the list of destinations, view destination information, and ping destinations. Section 12.3 DICOM Send Options on Page 26</p>
<p>Worklist</p>	<p><i>Present if the Worklist option is installed.</i> Administrators can add and configure a Worklist source. All users can view the worklist source information and ping the source. Section 12.4 Worklist Configuration on Page 30</p>
<p>Users</p>	<p>User privileges within InSight FD are managed by Windows user accounts. Administrators can:</p> <ul style="list-style-type: none"> • Add new user mappings • Edit existing user data, status (active/inactive) and privilege settings • Delete user mapping. <p>All users can view the list of users registered on the system. Section 12.4.1 Users on Page 32</p>
<p>Physician Preferences</p>	<p>Administrators can maintain a list of registered physicians and edit default preferences. All users can view the list of physicians registered on the system and view the default preferences for each physician. Section 12.5 Physician Preferences on Page 33</p>
<p>Administrative Settings</p>	<p>Administrators can change the System/Error log level of detail, turn audit mode on or off, enable or disable the touch screen, view the date of the last system configuration backup, and turn on or off the feature to shut down the system upon application shutdown. All users can view log file statistics and the data maintained by administrators. Section 12.9 Administrative Settings Page on Page 40</p>

Figure 24 Tools Pull-down Menu

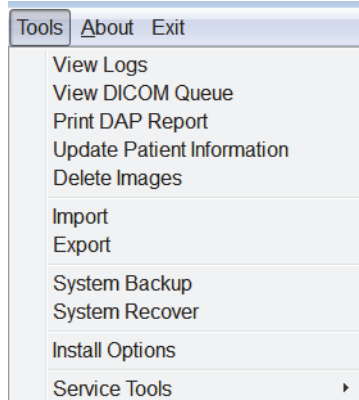


Table 26 Tools Menu


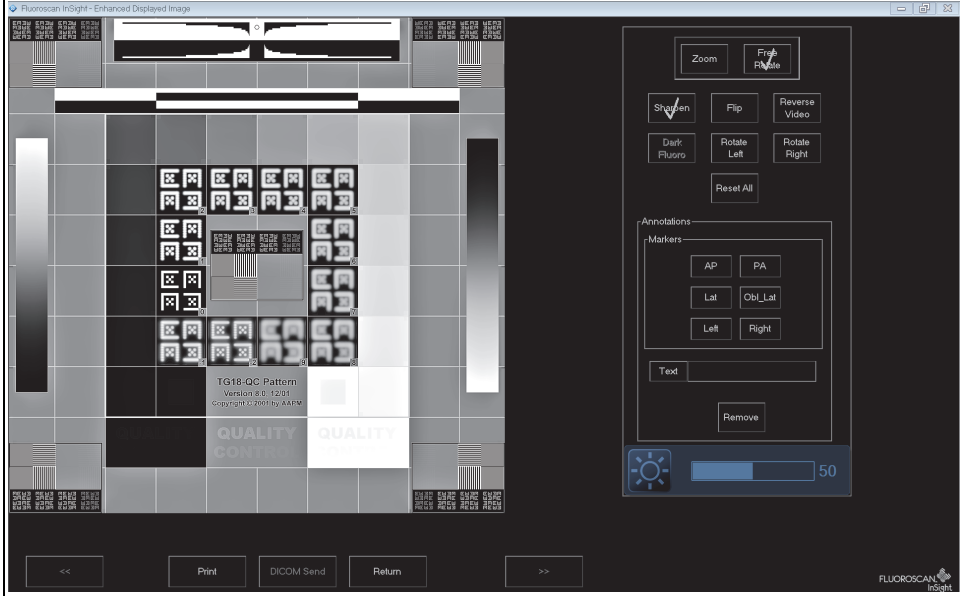
Function	Description
View Logs	View, print, export and in some cases edit various log files the application maintains.
View DICOM Queue	<i>Present if the DICOM Send option is installed.</i> View status of DICOM Send requests, and retry or delete selected requests. Section 13.8 View DICOM Queue Window on Page 71
Print DAP Report	Display and print the Dose Area Product report.  <i>Click DICOM Send, on the DAP Report display screen, to send the DAP Report to a DICOM destination. Include DAP Report must be selected to include the DAP Report with sent images. Otherwise, only the image will be sent.</i>
Update Patient Information	Administrators can update patient information for a selected patient study. Section 13.4.1 Update Patient Study Information Window on Page 61
Delete Images	Administrators can delete selected images from the system. Section 13.7 Delete Images Window on Page 68
Import	Import selected images into the system from removable media. Section 13.6 Import Images Window on Page 66
Export	Export selected images from the system to removable media (Table 40).
System Backup	Backup system configuration and calibration data to external media.
System Recover	Administrators can restore system configuration and calibration data from external media.
Install Options	Administrators can install application software options.

Table 26 Tools Menu (Continued)

Function	Description
<p>Service Tools — SMPTE Pattern</p>	<p>Administrators can open a test pattern to check monitor performance and quality.</p> 

13.2 Image Acquisition Session

13.2.1 Select or Enter Patient Window

Available from Main Screen by clicking **Perform**, from the Image Acquisition screen by clicking **Update Patient**, and from the Review screen during an image acquisition session by clicking **Update Patient**.

Figure 25 Select or Enter Patient Window

The screenshot shows the 'Select or Enter Patient' window. It features a 'Patient Information' section with fields for Patient Last Name, Patient First Name, MI, Patient ID, Accession Number, Sex, Date of Birth (Month, Day, Year), and Study Description. A 'Clear All' button is located below these fields. To the right is the 'Physician's Preferences' section, which includes a dropdown menu for 'Hill, Mark', a 'Physician Name' selection button, and entry fields for 'Visiting Physician' Last Name and First Name. Below this is the 'Select patient from Worklist' section, which contains a table with the following data:

Last Name	First Name	MI	Patient ID	Sex	DOB	Accession #	Study Description	Study Date
Coyote	Wile		66	M	4/23/1947	80427557	Exam 4	6/10/2015

Below the table are buttons for 'Query', 'Detail Query', 'Last successful query: 7/22/2015 1:10:04 AM', and 'Patient Details'. At the bottom of the window are buttons for 'Perform', 'Review', 'End Session', 'Apply Changes', and 'Cancel'.

Table 27 Select or Enter Patient Window, Section/Field Descriptions

Section/Field	Function
Patient Information	Identifies the current patient. Data is either entered by the user or filled in when a patient is selected from the "Select Patient from Worklist" selection list. Click Clear All to clear Patient Information fields.
Physician's Preferences	Selection button and dropdown selection list of physicians registered on the system.
Visiting Physician	Selection button and entry fields for physician's name to be used when a physician is not registered on the system.

Table 28 Select or Enter Patient Window, Button Functions

Button	Function
Clear All	Clears Patient Information fields
Perform	Displays the Image Acquisition screen
Review	Displays the Review screen
End Session	Terminates the entry of Patient Information and leaves the Select or Enter Patient screen
Apply Changes	Enters changes without leaving the Select or Enter Patient screen
Cancel	Cancel the entry of Patient Information without leaving Select or Enter Patient screen

13.2.2 Image Acquisition Screen

Figure 26 Image Acquisition Screen



Table 29 Image Acquisition Screen

A	Image Effects Menu (Table 30)
B	Configure Image Area Menu (Table 31)
C	Function Buttons (Table 32)

Table 29 Image Acquisition Screen

D	Displays AKR and cumulative air kerma while the X-ray tube is active. Air kerma is calculated at the entrance to the CMOS detector.
E	Displays current power settings.
F	Image Area.
G	Displays cumulative DAP
H	Displays collimation mode. Limited Field magnifies a small area. Full field displays a large area. You can cycle through fields by moving the collimation switch. The collimation switch is underneath the X-ray head (Section 9 <i>System Components</i> on Page 13).
I	Filmstrip/Tools Area (Table 33) for Tool Set description.

Figure 27 Image Effects Menu

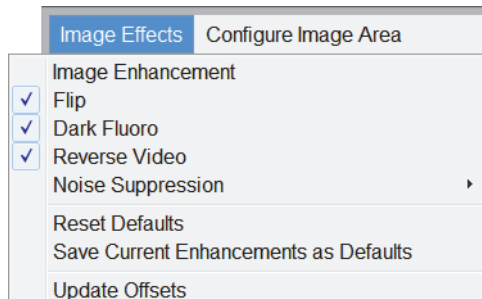


Table 30 Image Effects Menu

Function	Description
Image Enhancement*	Images enhanced according to edge and sharpening configurations set in the Image Enhancement field in Table 44.
Flip*	Images are flipped before being viewed by the user.
Dark Fluoro*	Images have their white image pixels changed to black.
Reverse Video*	Images are changed to a negative (blacks go to white; whites go to black).
Noise Suppression	Selects noise suppression level. Selections are Ultra, Auto, High, Medium, Low and Off. See Section 12.5 <i>Physician Preferences</i> on Page 33 for default setting.
Reset Defaults	Resets the image enhancements, flip, dark fluoro, reverse video, and noise suppression settings to the default settings specified by the current physician's preferences settings, or the system default preferences settings in the event of a visiting physician.

Table 30 Image Effects Menu (Continued)

Function	Description
Save Current Enhancements as Defaults	Enabled if the selected physician is registered on the system. Saves the current image effect settings as the selected physician's preference settings.
Update Offsets	Updates the offsets for Flat Field correction.

* May be disabled or initially checked or unchecked. (Section 12.5 Physician Preferences on Page 33)

Figure 28 Configure Image Area Menu

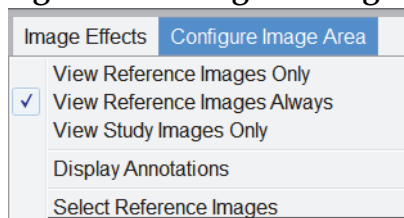


Table 31 Configure Image Area Menu

Function	Description
View Reference Images Only	Only images tagged as reference images are displayed in the filmstrip. This includes images tagged as reference during the current session and/or reference images selected from previous studies for the current patient.
View Reference Images Always	Images tagged as reference always display at the top of the filmstrip followed by reference images selected from previous studies for the current patient, followed by non-reference images acquired in the current session in reverse order.
View Study Images Only	Only images (reference and non-reference) acquired in the current session are displayed in the filmstrip.
Display Annotations	Displays annotations on the image displaying in the live image area.
Select Reference Images	Displays the Select Reference Images window allowing users to select images from any current patient study residing on the system. Selected images are put into the filmstrip following any reference images acquired during the current session unless View Study Images Only is checked.

Figure 29 Image Acquisition Screen Buttons

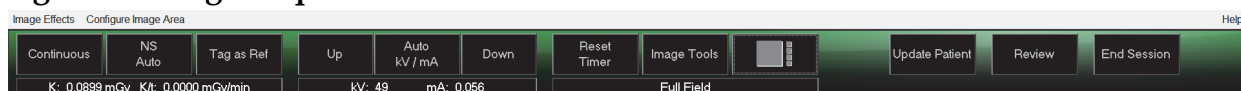


Table 32 Image Acquisition Screen, Button Functions

Button	Function
Acquisition Mode Selection	Cycles through: Continuous – Displays continuous images when X-rays are actuated. Snapshot – Captures and displays one image when X-rays are actuated. Cine Record – Gathers all images acquired into a cine loop. Continuous and/or Snapshot mode may be disabled (Section 12.5 Physician Preferences on Page 33).
NS (Noise Suppression)	Cycles through the levels selected during configuration. See Table 17 Physician Preferences, Section/Field Descriptions on Page 34. Select Ultra to acquire images for very still anatomy. Image processing for Ultra level imaging averages frames at High noise suppression.
Tag as Ref	Marks the image as a reference image for the current session or unmarks a marked image.
Up	Increases the kV and/or mA incrementally by one.
X-ray Power Adjustment Selection	Cycles through: Auto kV / mA - Automatically adjusts kV / mA. Manual kV / mA (simultaneous kV/mA enabled) - Allow the user to manually increase or decrease kV / mA. Manual kV (simultaneous kV/mA disabled) - Allow the user to manually increase or decrease the kV. Manual mA (simultaneous kV/mA disabled) - Allow the user to manually increase or decrease the mA. Auto IQ kV / mA - Automatically adjusts kV / mA. Select IQ mode to acquire an enhanced image with improved clarity. Using IQ mode increases the X-ray dose to the patient. Use Auto IQ when enhanced clarity is required in accordance with ALARA. See Section 12.5 Physician Preferences on Page 33 for setting simultaneous kV/mA.
Down	Decreases the kV and/or mA incrementally by one.
Reset Timer	Resets the X-ray warning timer to zero and turns off the audible alarm.
Image Tools	Shows or hides the tool set in the Filmstrip/Tools area.

Table 32 Image Acquisition Screen, Button Functions (Continued)





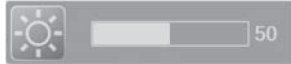

Button	Function
Format Toggle	Selects current image size and number of thumbnail images that display in the filmstrip area.
	 one filmstrip image.
	 two filmstrip images.
	 three filmstrip images.
	 four filmstrip images.
Update Patient	Returns to the Select or Enter Patient window.
Review	Displays the Review Images screen.
End Session	Ends the current patient's acquisition session. Returns to the main screen.

Table 33 Tool Set

Button	Function
Zoom	Magnifies the area under a magnifying glass two times. The magnified area can be moved to any place on the image.
Free Rotate	Allows image to be freely rotated using the mouse or touch screen.
Enhance *	Applies/removes image enhancement to/from the image according to Image Enhancement settings selected in Table 33 .
Flip *	Flips/unflips the image horizontally about the center.
Reverse Video *	Applies/removes reverse video (blacks go to white; whites go to black) to/from the image.
Dark Fluoro *	Applies/removes dark fluoro (white image pixels change to black) to the image.
Rotate Left	Rotates the image in 10 degrees increments counterclockwise.
Rotate Right	Rotates the image in 10 degrees increments clockwise.
Reset All	Resets all image processing on the image to the last saved state of the image.

Table 33 Tool Set (Continued)

Button	Function	
Annotations	<p>Contains function buttons and a field that allows the user to add markers or text to the image or remove selected markers or text from the image.</p> <p>Annotations can be moved and/or removed using the mouse or touch screen.</p> <p>To remove an annotation, deselect Free Rotate, select the annotation, click Delete.</p> <p>Annotations include: AP, PA, Lat, Obl_Lat, Left, Right, Text (Places the text in the text edit box on the image).</p>	
Brightness/ Contrast	Allows adjustment of brightness and contrast of the selected image.	
		Brightness control
		Contrast control

*This button may be configured on, off, or disabled ([Section 12.5 Physician Preferences](#) on Page 33).

13.2.3 Select Reference Images Window

Available from the Image Acquisition screen by selecting Select Reference Images from the Configure Image Area Menu.

Figure 30 Reference Images Window

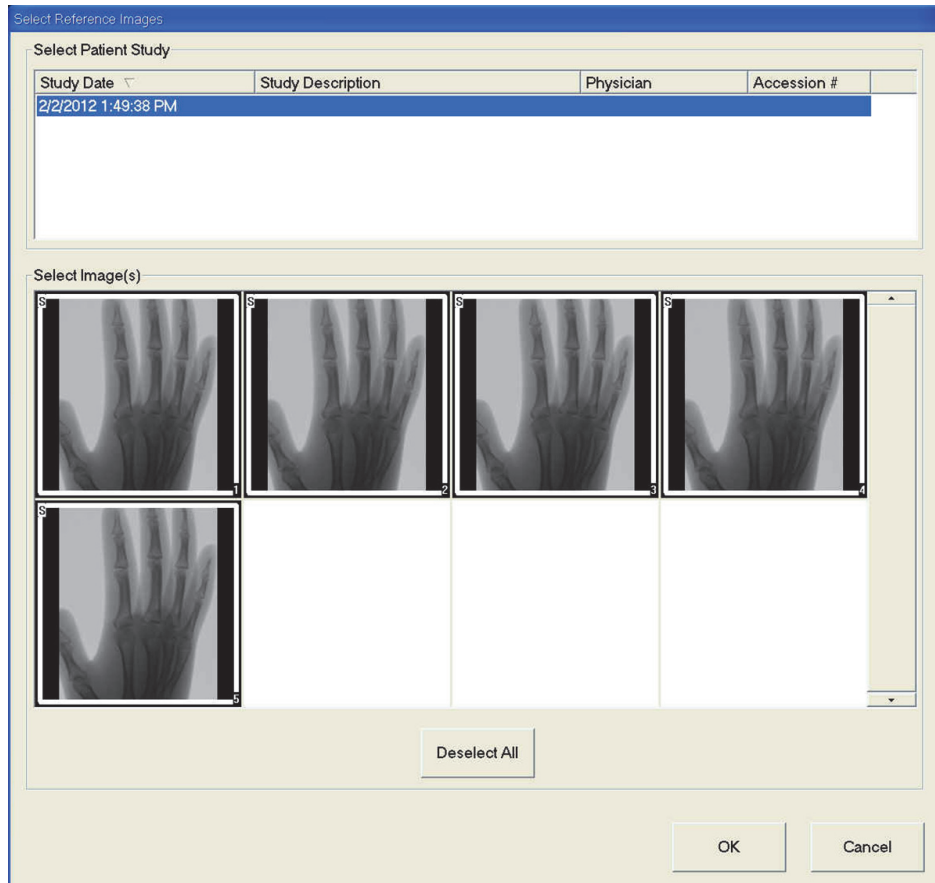


Table 34 Select Reference Images Window, Section Descriptions

Section	Description
Select Patient Study	Selection list of prior studies for the current patient stored on the system. The selection list can be sorted on any column by clicking in the respective column label. The entry field label in the Find section changes to correspond to the selected sort column.
Select Image(s)	A selection list of thumbnail images contained in the selected patient/study file. A white box surrounds selected images. Multiple images can be selected. All images can be selected/unselected by clicking Select All or Deselect All . A scroll bar is displayed if the filmstrip layout does not show all images.

Table 35 Select Reference Images Window, Button Functions

Button	Function
Select All/ Deselect All	Selects or deselects all images in the image selection list.
OK	Returns all selected images to the Image Acquisition screen's filmstrip unless View Study Images Only is checked in the Configure Image Area Menu.
Cancel	Closes the window without returning any images to the Image Acquisition screen.

13.3 Review Images Session

13.3.1 Select Study for Review Window

Available from Main Screen by clicking **Review** and from the Review screen during a review session by clicking **Select Patient**.



Note *On a wide-screen monitor, when a single study is selected, images appear top to bottom in a single column on the right side of the monitor screen.*

Figure 31 Study for Review

Select Study for Review

Search For:

Patient Last Name Patient First Name MI Start Study Date Select Physician
 Patient ID Accession Number Sex End Study Date Enter Physician
 Study Description Clear All Last Name First Name

Select patient studies for Review

Last Name	First Name	MI	Patient ID	Sex	DOB	Accession #	Study Description	Study Date	Physician
10650750	62141243	XXXXXXXX	M	7/1/1960	00108931	Exam 3	7/1/2015 00:00:00	hill, mark	
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/30/2015 9:45:03 AM		
lest	t	t	0987654321	M	9/11/1984	3045380941...	Exam 4	6/25/2015 1:12:01 PM	hill, mark
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/22/2015 8:09:13 AM		
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/22/2015 7:53:13 AM		
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/19/2015 2:12:40 PM		
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/19/2015 2:11:30 PM		
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/19/2015 12:49:50 PM		
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/19/2015 11:55:22 AM		
gray	danny	123	M		3045233182...	wrist	6/19/2015 11:50:52 AM	white, jisse	
yutaku		op9opiop			3045233025...		6/19/2015 11:42:52 AM	hill, mark	
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/19/2015 11:27:21 AM	hill, mark	
Coyote	Wile	66	M	4/23/1947	80427557	Exam 4	6/19/2015 11:21:21 AM	hill, mark	
a		a			3045237321...		6/19/2015 9:40:25 AM		


Update List DAP Report Patient Details

Review End Session

Table 36 Select Study for Review Window, Section/Field Descriptions

Section/Field	Description
Search For	Search criteria entry fields. To use Start Study and/or End Study Dates as a search criteria, the check box must be checked. The respective check box can be checked by clicking in the box, typing a date in the date field, or selecting a date from the calendar page that appears when clicking the down arrow.
Select Physician	Selection button and dropdown selection list of physicians registered on the system.
Enter Physician	Selection button and two entry fields for physician's name. Use when a physician's name is not registered on the system.
Select Patient Studies for Review	Selection list of all patient/study records on the system. Selection list can be filtered by completing one or more fields in the "Search For" section and clicking Update List . The selection list can be sorted on any column by clicking in the respective column label. One or more studies can be selected.

Table 37 Select Study for Review Window, Button Functions

Button	Function
Clear All	Clears all entries in the <i>Search For</i> section.
Update List	Updates the <i>Select Patient Studies for Review</i> selection list using criteria contained in the <i>Search For</i> section to limit the list. If all <i>Search For</i> parameters are empty, a list of all studies on the system is retrieved.
DAP Report	Displays the DAP report for the selected patient.  Click DICOM Send , on the DAP Report display screen, to send the DAP Report to a DICOM destination. Include DAP Report must be selected to include the DAP Report with sent images. Otherwise, only the image will be sent.
Patient Details	Enabled when only one study is selected. Displays patient information.
Review	Displays the Review screen. <i>At least one study must be selected to enable the Review button.</i>
End Session	Terminates the review session.

13.3.2 Review Screen

Available from the Select Study for Review window, and from the Image Acquisition screen by clicking **Review**.

Figure 32 Review Images Screen

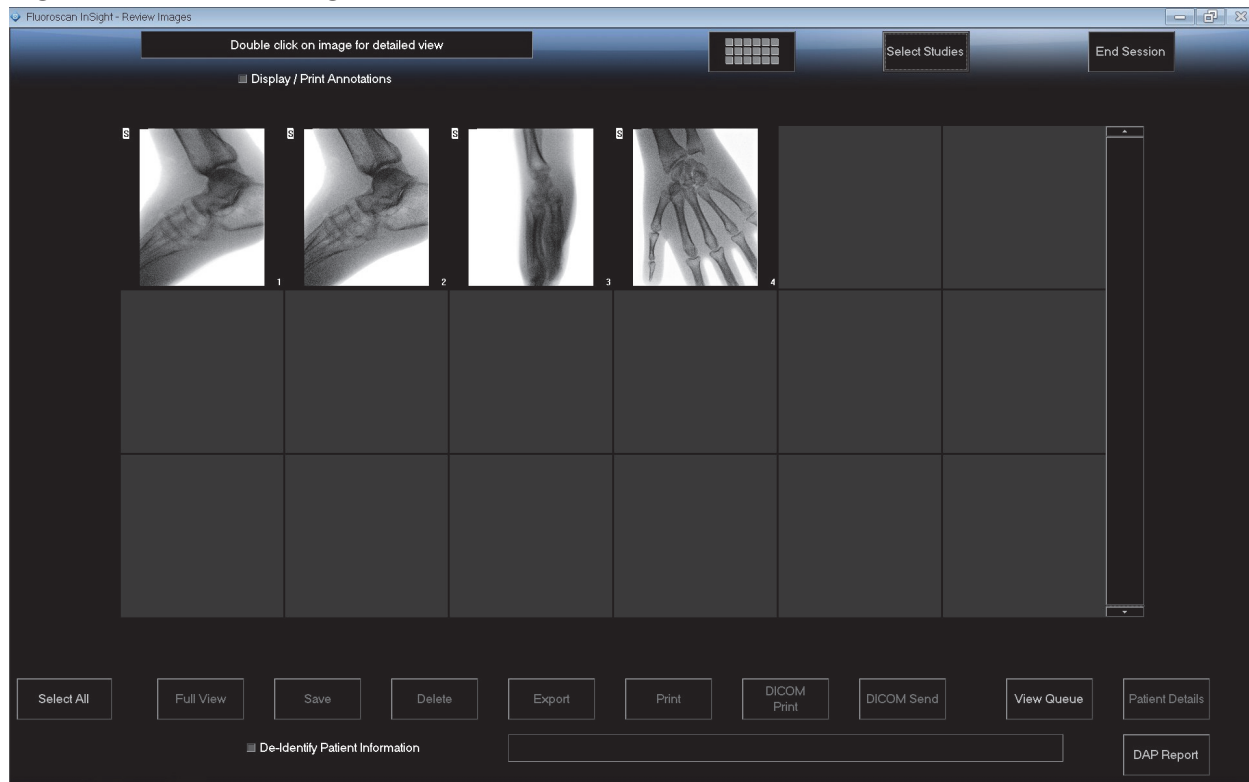


Table 38 Review Screen, Button Functions




Button	Function
Display Mode Selection	Determine how images are displayed.
	 Displays one row of two images.
	 Displays two rows of four images.
	 Displays three rows of six images.
Select Studies	Selects studies for review.
Display/Print Annotations	Displays or prints annotations on the displayed image.
Select/Deselect All	Toggles between: Select All - Selects all unselected images. Deselect All - Unselects all selected images.

Table 38 Review Screen, Button Functions (Continued)

Button	Function
Full View	Displays the selected image in the Enhanced Displayed Image screen (Section 13.3.3 Enhanced Displayed Image Screen on Page 60)
Save	Saves selected images or cine loop to the patient's record. If the number of stored images or minutes of cine loop exceed the storage limit, a message is displayed.
Delete	Deletes the selected images. If the system setting <i>Query on Delete</i> is enabled, the user is asked to confirm the deletion. Click Yes to confirm; No to cancel the deletion.
Export	Copies the selected image(s) or cine loop to removable media. Patient information will be removed from images when <i>De-Identify Patient Information</i> is checked.
Print	Prints the selected image(s) to the local printer. Images will be printed with annotations when <i>Display/Print Annotations</i> is checked. Patient information will be removed from images when <i>De-Identify Patient Information</i> is checked.
DICOM Print	Sends the selected images to a DICOM node printer for printing. Images will be printed with annotations when <i>Display/Print Annotations</i> is checked. Patient information will be removed from images when <i>De-Identify Patient Information</i> is checked.
DICOM Send	Sends the selected image(s) to a DICOM Send node. Patient information will be removed from images when <i>De-Identify Patient Information</i> is checked.
View Queue	Displays the status of all DICOM Send requests (Section 13.8 View DICOM Queue Window on Page 71)
Patient Details	Displays patient information. Click Additional Details to reveal Vista Dialog (Figure 34).
DAP Report	Displays the DAP report for the selected patient. Note: Click DICOM Send, on the DAP Report display screen, to send the DAP Report to a DICOM destination.
De-Identify Patient Information	Omits protected patient health information details related to specific patients.
Update Patient / Select Study	In an acquisition session, returns to the Select or Enter Patient window to view or update patient information In a review session, returns to the Select Patient for Review window to select a different patient study.

Table 38 Review Screen, Button Functions (Continued)

Button	Function
Perform	Visible only in an acquisition session. Returns to the image acquisition screen.
End Session	Terminates the current session.

Figure 33 Patient Details

The screenshot shows a dialog box titled "Patient Details" with the following fields and values:

- First Name:
- Last Name: q
- Patient ID: q
- Study Description:
- Physician:
- Accession #: 3044493580282435
- Birth Date:
- Location:
- Date Time: 5/13/2015

At the bottom of the dialog box, there are two buttons: "Additional Data" and "Close".

Figure 34 VISTA Details

The screenshot shows a dialog box titled "VISTA Details" with the following fields:

- Other Patient ID:
- Ethnic Group:
- Patient Comment:
- Pregnancy Status:
- Medical Alerts:
- History:
- Current Location:
- AE Title:
- Procedure Location:
- Procedure Description:
- Performing Physician:
- Procedure Date:
- Procedure Comments:
- Scheduled Code Value:
- Scheduled Code Scheme:
- Scheduled Code Description:
- Requested Code Value:
- Requested Code Scheme:
- Requested Code Description:
- Image Comments:
- Requesting Service:
- Attending Physician:
- Requesting Physician:

At the bottom center of the dialog box, there is a "Close" button.

13.3.3 Enhanced Displayed Image Screen

Available from the Review Images screen by selecting one image and clicking **Full View**, or by double clicking an image.

Figure 35 Enhanced Displayed Image Screen



The tool set is identical to the tool set described in [Table 33](#)

Table 39 Detailed Image Viewing Screen, Button Functions

Button	Function
<< (Previous Image)	Displays the previous image.
Print	Prints a copy of the image.
Delete	Deletes the image currently being displayed. If the system setting <i>Query on Delete</i> is enabled, the user is asked to confirm the deletion. Click Yes to confirm; No to cancel the deletion.
Return	Returns to the Review screen.
>> (Next Image)	Displays the next image.

13.4 Maintaining Patient Study Records

13.4.1 Update Patient Study Information Window

Available to administrators from the Tools Menu on the Main Screen.


Figure 36 Tools Menu

The screenshot shows the 'Update Patient/Study Information' window. It features a search section at the top with a 'Find' label and a 'Study Date' input field. Below this is a table with columns: Last Name, First Name, MI, Patient ID, Sex, DOB, Accession #, Study Description, Study Date, and Physician. The table contains several rows of data. The middle section is for updating patient information, with fields for Patient Last Name, Patient First Name, MI, Patient ID, Accession Number, and Date of Birth (Month, Day, Year). At the bottom right are 'Update' and 'Cancel' buttons.

Table 40 Update Patient/Study Information Window, Section Descriptions

Section	Description
Select Patient Study	Selection list of all patient/study records on the system and a search criteria section (Find). The selection list can be sorted on any column by clicking in the respective column label. The entry field label in the Find section changes to correspond to the selected sort column.
Find	Use to quickly locate a patient/study.
Update Patient Information	Displays information from the selected patient/study record.

Table 41 Update Patient/Study Information Window, Button Functions

Button	Function
Update	Updates the patient information on all images in the selected study.  <i>Note The Auto-Accession number is associated with a specific patient. It does not change when patient information is updated.</i>
Cancel	Closes the window without updating and returns to the main screen.

13.5 Export Images

Available from the **Tools Menu** on the **Main Screen** and from the **Review Images Screen**.

13.5.1 Export from Review Images Screen

1. From the **Review images** Screen ([Figure 37](#)), Select images for export.
2. Click **Export** to open the **Export Images Window** ([Figure 38](#)).
3. Select the options for export: File Type and/or Annotations
4. Click **Export**.

Figure 37 Image Export from Review Screen




Figure 38 Export Images



13.5.2 Export from Review Images Screen

1. From the **Tools** Menu (Figure 39), select **Export**.

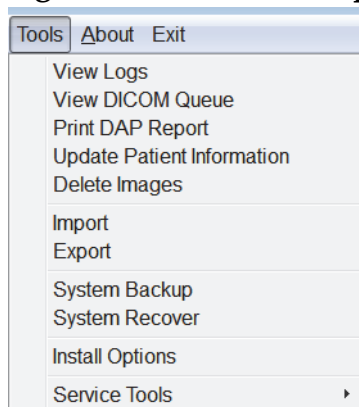
Figure 39 Tool Menu Export



2. Click **Export** to open the **Export Images** Window.

3. In the **Export Images** window (Figure 40), select the studies from which files will be exported.

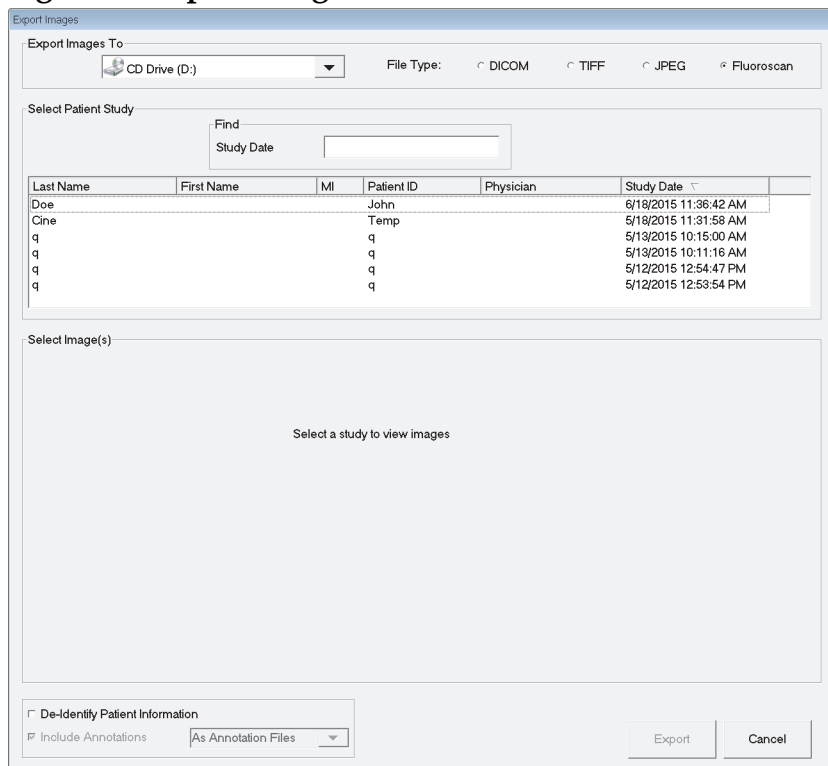Figure 40 Export Images from Tool Menu



4. Click **Export** to open the **Export Images** Window.
5. Select the images and options for the export:
 - a. Select the Images for export
 - b. Location to which the files will be exported
 - c. File Type
 - d. De-identify patient information
 - e. Annotations
6. Click **Export**.

Table 42 Export Window, Section Descriptions

Section	Description
Export Images to	Selection list of removable media devices currently available to the system. <i>Other Locations</i> allows the user to browse for a folder on the system or network.
File Type	File format selection.

Table 42 Export Window, Section Descriptions (Continued)

Section	Description
Select Patient Study	Selection list of all patient/study records on the system and a search criteria section (Find). The selection list can be sorted on any column by clicking in the respective column label. The entry field label in the Find section changes to correspond to the selected sort column. One or more studies can be selected. When one study is selected, all images in the study display in the Select Image(s) section. When multiple studies are selected, images are not displayed.
Find	Use to quickly locate a patient/study.
Select Image(s)	Images are displayed when <u>one</u> patient/study is selected. Images are not displayed when multiple studies are selected. A selection list of thumbnail images contained in the selected patient/study file. A white box surrounds selected images. Multiple images can be selected. All images can be selected/unselected by clicking Select All or Deselect All .
De-Identify Patient Information	When checked, removes patient identification data from images being exported.
Include Annotations	When checked, includes annotations with exported images. Method of inclusion choices are: Embedded - Annotations are embedded into the bitmap image. As Overlay - Annotations are included as a DICOM overlay. As annotation files - Annotations are included as a separate file.

Table 43 Export Window, Button Functions

Button	Function
Patient Details	Enabled when only one image is selected. Displays patient information.
Select All/ Deselect All	Selects or deselects all images in the image selection list.
Export	Exports the selected images to the selected media device in the selected format. When more than one study is selected, copies all images in each study. <i>Images are not deleted from the system.</i>
Cancel	Closes the window without exporting images and returns to the main screen.

13.6 Import Images Window

Available from **Tools** Menu (Figure 41) on the **Main Screen**.

Figure 41 Tools Menu

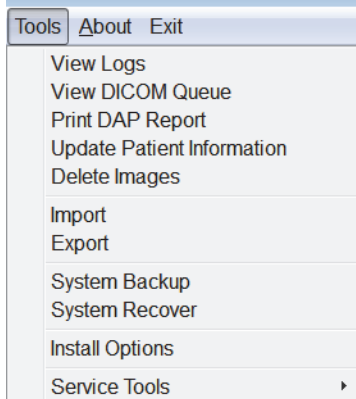


Figure 42 Import Images Window

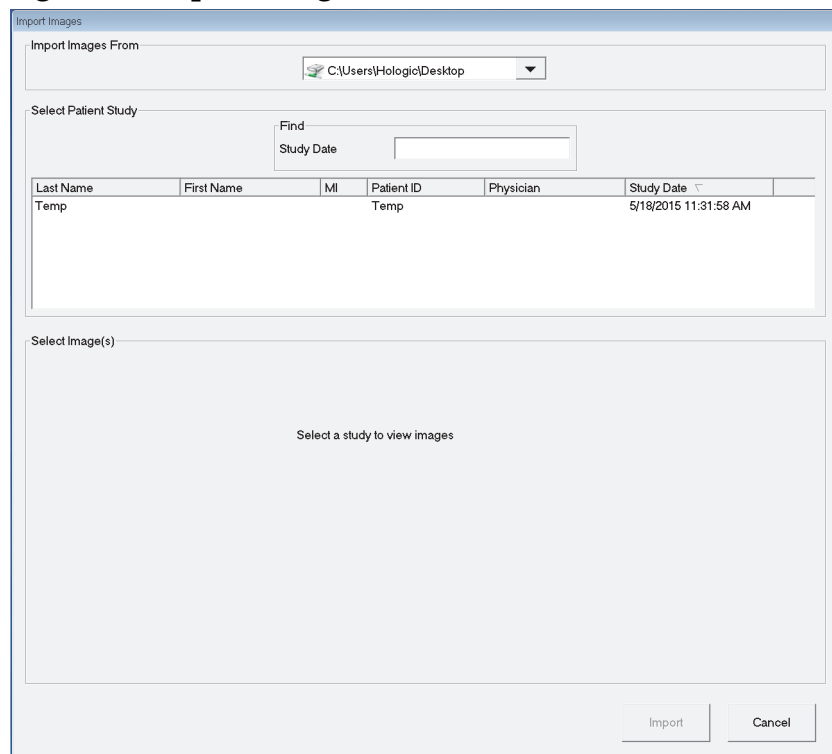


Table 44 Import Window, Section Descriptions

Section	Description
Import Images from	Selection list of removable media devices currently available to the system. Other Locations allows the user to browse for a folder on the system or network.
Select Patient Study	Selection list of all patient/study records on the selected media and a search criteria section (Find). The selection list can be sorted on any column by clicking in the respective column label. The entry field label in the Find section changes to correspond to the selected sort column. One or more studies can be selected. When one study is selected, all images in the study display in the Select Image(s) section. When multiple studies are selected, images are not displayed.
Find	Use to quickly locate a patient/study.
Select Image(s)	Images are displayed when <u>one</u> patient/study is selected. Images are not displayed when multiple studies are selected. A selection list of thumbnail images contained in the selected patient/study file. A white box surrounds selected images. Multiple images can be selected. All images can be selected/unselected by clicking Select All or Deselect All .

Table 45 Import Window, Button Functions

Button	Function
Patient Details	Enabled when only one image is selected. Displays patient information.
Select All/ Deselect All	Selects or deselects all images in the image selection list.
Import	Imports the selected images into the system.
Cancel	Closes the window without importing images and returns to the main screen.

13.7 Delete Images Window

Available to administrators from the Tools Menu on the **Main Screen**.

Figure 43 Delete Images

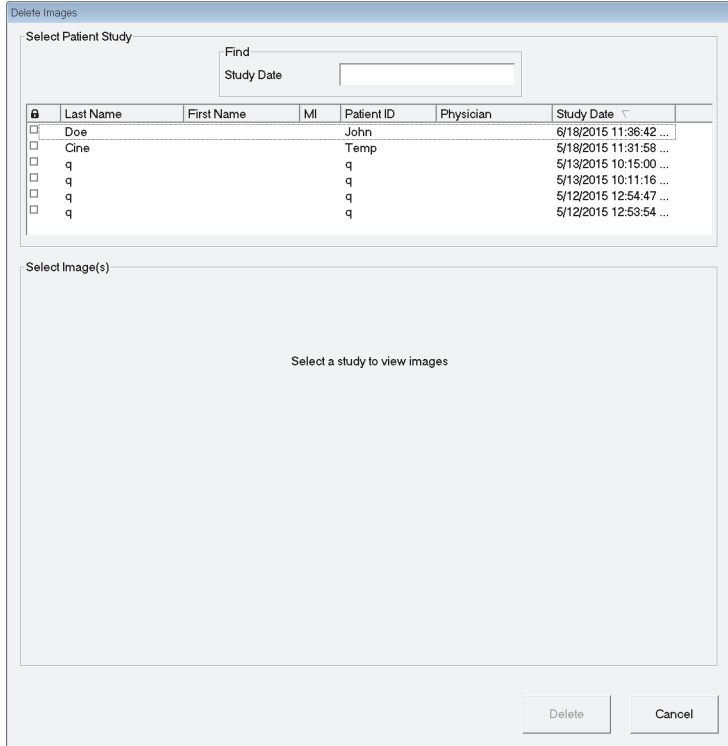


Figure 44 Patient Selected

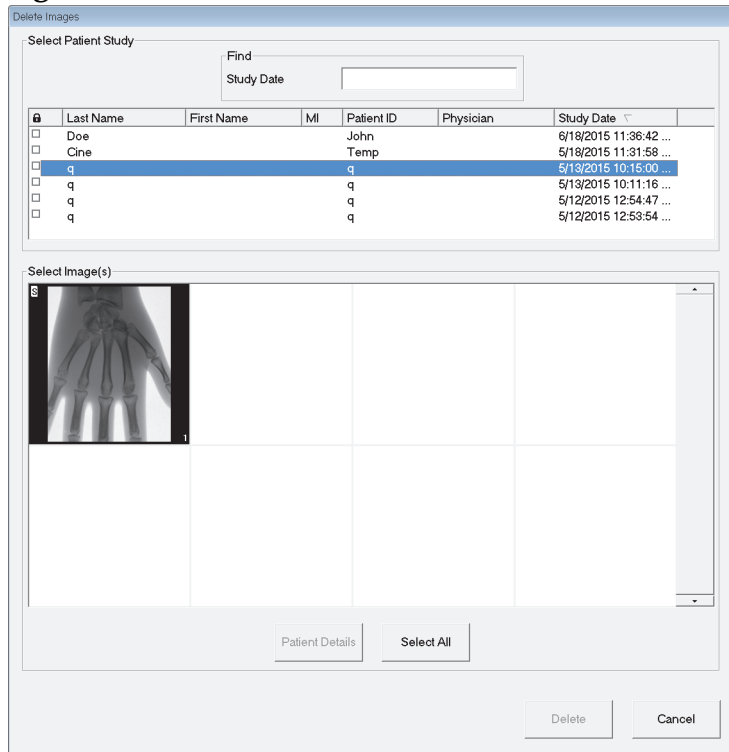


Figure 45 Image Selected for Deletion

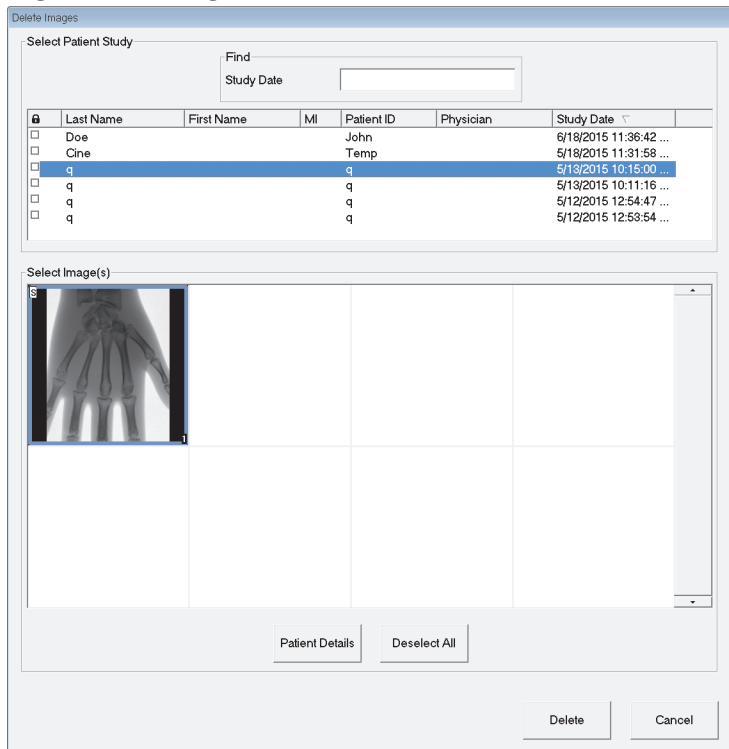


Table 46 Delete Images Window, Section Descriptions

Section	Description
Select Patient Study	<p>Selection list of all patient/study records on the system and a search criteria section (Find).</p> <p>The selection list can be sorted on any column by clicking in the respective column label. The entry field label in the Find section changes to correspond to the selected sort column.</p> <p>One or more studies can be selected. When one study is selected, all images in the study display in the Select Image(s) section. When multiple studies are selected, images are not displayed.</p>
Find	<p>Use to quickly locate a patient/study.</p>
Select Image(s)	<p>Images are displayed when <u>one</u> patient/study is selected. Images are not displayed when multiple studies are selected.</p> <p>A selection list of thumbnail images contained in the selected patient/study file.</p> <p>A white box surrounds selected images. Multiple images can be selected. All images can be selected/unselected by clicking Select All or Deselect All.</p>

Table 47 Delete Images Window, Button Functions

Button	Function
Patient Details	<p>Enabled when only one image is selected.</p> <p>Displays patient information.</p>
Select All/ Deselect All	<p>Selects or deselects all images in the image selection list.</p>
Delete	<p>Deletes the images selected in the image selection list from the InSight FD system. When more than one study is selected, deletes all images in each study. If the system setting Query on Delete is enabled, the user is asked to confirm the deletion. Click Yes to confirm; No to cancel the deletion.</p>
Cancel	<p>Closes the window without deleting images and returns to the main screen.</p>

13.8 View DICOM Queue Window

Available from the Tools Menu and from the Review Images screen.

Figure 46 View DICOM Queue

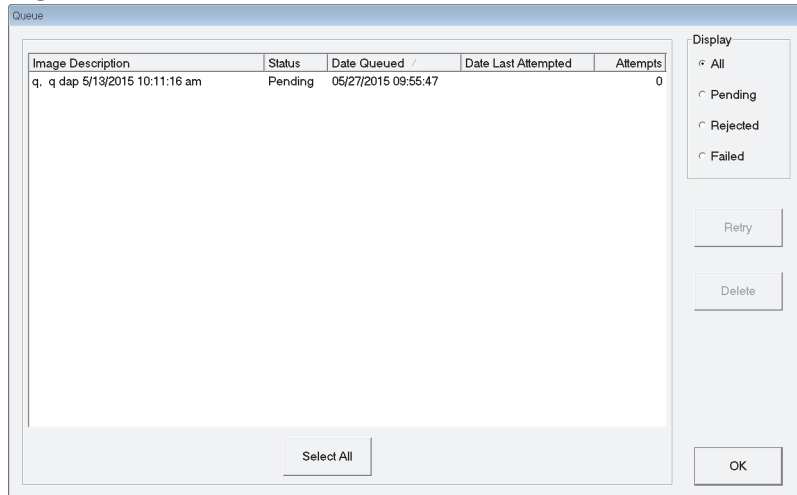


Table 48 Queue Window, Section Description and Button Functions

Section/Button	Description/Function
DICOM Queue List	Selection list of queue request information that can be filtered.
Display	Filters the selection list depending on the selected radio button.
Select All/ Deselect All	Selects or deselects all DICOM requests listed in the Queue List.
Delete	Deletes the selected DICOM request(s).
Retry	System tries to send the selected DICOM request(s) again.
OK	Closes the Queue window and returns to the previous screen (Main Screen or Review Images screen).

14 System Maintenance

14.1 System Backup

Any user can backup system configuration and calibration data to a selected removable media device by selecting System Backup from the Tools Menu on the Main Screen.

14.2 System Recover

Administrators can restore system configuration and calibration data from a selected removable media device by selecting System Recover from the Tools Menu on the Main Screen.

14.3 Cleaning

Exterior surfaces of the InSight FD Mini C-arm should be cleaned once a week and anytime after a surface has become soiled. Clean exterior surfaces of the cabinet, the flex-arm and the C-arm with a soft cloth or disposable lint-free tissue moistened with water, isopropanol or methanol. Disinfect using a 10% bleach solution. Wipe surfaces, do not scrub. A commercial glass cleaner may be used to clean the monitor's touch screen.



Warning: *Disconnect the equipment from the AC Power Source before applying any cleaners or disinfectants to the system.*

14.3.1 Recommended Disinfectants

Table 49 Recommended Wipes

Cleaner
PDI Super Sani-Cloth Germicidal Disposable Wipes
PDI Sani-Cloth plus Germicidal Disposable Cloth

Table 50 Recommended Chemicals

Base Type	Typical Products
Chlorine (maximum 10% solution)	Clorox, Novalsan
Formaldehyde	Formaldehyde solution 37%, Vinco Formaldegen
Glutaraldehyde	Aldacide 200, Lysofume, Wavicide
Phenol family	Lysol I.C., Beaucoup, Magna Clean, Tek-Trol
Alcohol	Cavicide, Isopropyl Alcohol, Medicide
Oxidizing	Hydrogen Peroxide 3% Solution, Lifegard Series
Quaternary Ammonium	Roccal-D, Bacto-Sep, Parvosol
Soap and Water	

14.4 Preventive Maintenance

Hologic recommends that preventative maintenance be performed by authorized service personnel at least once per year.

HOLOGIC®



Hologic, Inc.
36 Apple Ridge Road
Danbury, CT 06810 USA
Made in Mexico



Hologic Ltd.
Heron House Oaks Business Park
Crewe Road
Wythenshawe, Manchester
M23 9HZ, UK
Tel: +44 (0)161 946 2206

CE
0044