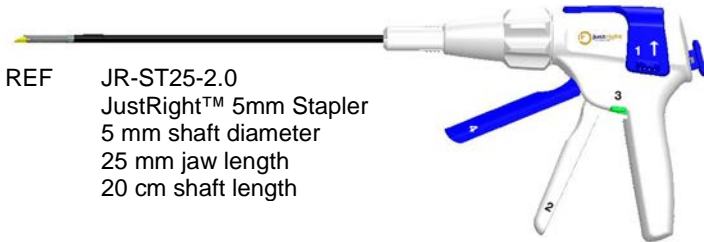
Instructions for Use

JustRight™ 5mm Stapler

REF JR-ST25-2.0
JustRight™ 5mm Stapler
5 mm shaft diameter
25 mm jaw length
20 cm shaft length

JustRight™ 5mm Reload

REF JR-REL25-2.0
JustRight™ 5mm Reload
5 mm shaft diameter
25 mm jaw length
20 cm shaft length

Caution

Read all warnings, cautions, and instructions provided with this instrument before use.

Indications for Use

The JustRight™ 5mm Stapler is intended for use in abdominal, gynecologic, pediatric, and thoracic surgery for resection, transection, and creation of anastomosis.

Titanium, as found in JustRight Surgical surgical staples, is acceptable for use with MRI and NMR procedures as it is nonferromagnetic. The implanted staples are not subject to displacement or induced torque, and do not result in localized heating. The implantable staples are categorized as MR Safe.

Contraindications

- Do not use the JustRight™ 5mm Stapler 2.0 mm staples on any tissue that compresses to less than .75mm in thickness, on any tissue that cannot comfortably compress to 1.0 mm or on the aorta.
- Do not use the JustRight™ 5mm Stapler on tissue such as liver, hepatic vasculature, spleen, pancreas, or biliary structures where compressibility is such that closure of the instrument would be destructive.
- Do not use the JustRight™ 5mm Stapler where adequacy of hemostasis cannot be verified visually after application.
- Do not use the JustRight™ 5mm Stapler with a staple line reinforcement material. Adequate staple formation cannot be verified when using staple line buttressing materials such as PERI-STRIPS or SEAMGUARD products.

Symbols

	Only
	Consult Instructions For Use
	Single Use Only
	Not made with natural rubber latex.
	Sterilized using Ethylene Oxide
	Catalog, reorder or reference number
	Manufacturer of record
	European Community Authorized Representative
	Do not resterilize
	Do not use if package is opened or damaged.
	Recycle
	Attention, consult accompanying documents.

Warnings and Cautions

Warning
This product is intended for single use only; it cannot be adequately cleaned or resterilized for safe reuse. Attempts to clean or sterilize may result in biocompatibility infection or product failure risks to the patient.
Place tissue in the center of the jaws. To avoid incomplete ligation, do not grasp structure beyond the staple line surface; do not place tissue in the jaw hinge. Any tissue extending beyond the cut mark will not be transected.
When using the stapler more than once during a SINGLE surgical procedure, be sure to remove the empty JustRight™ 5mm Reload single use unit and reload a new one. When the JustRight™ 5mm Reload is empty, the blue staple deployment and cut lever will not engage. This prevents inadvertent actuation of the integrated knife.
After staples have been deployed, do not reclamp an empty staple cartridge on tissue or use to manipulate structures. Inadvertent tissue damage may occur.
Preoperative radiotherapy may result in changes to tissue. These changes may, for example, cause the tissue thickness to exceed the indicated range for the staple size. Careful consideration with regard to staple size should be given to any pre-surgical treatment the patient may have undergone.
Device performance testing was conducted preclinically in animals. No human clinical testing was performed.
Always inspect the tissue thickness prior to application of the JustRight™ 5mm Stapler.
Endoscopic procedures should be performed only by physicians having adequate training and familiarity with endoscopic techniques. Prior to performing any endoscopic procedures, consult the medical literature relative to technique, complications, and hazards.
When using endoscopic instruments and accessories from different manufacturers in a procedure, be sure to verify compatibility and ensure that electrical isolation or grounding is not compromised.

Caution
After firing, always inspect the staple line for hemostasis. Minor bleeding or oozing may be controlled by electrocautery or manual sutures.
The anvil of the stapler must be completely visible (past the cannula sleeve) prior to opening the jaws within the body cavity.
The JustRight 5mm Stapler is intended to be used with a laparoscopic cannula with a minimum diameter of 5.5 mm.
Always close the blue Top Lever on the stapler handle prior to introducing and removing the stapler from the cannula sleeve.

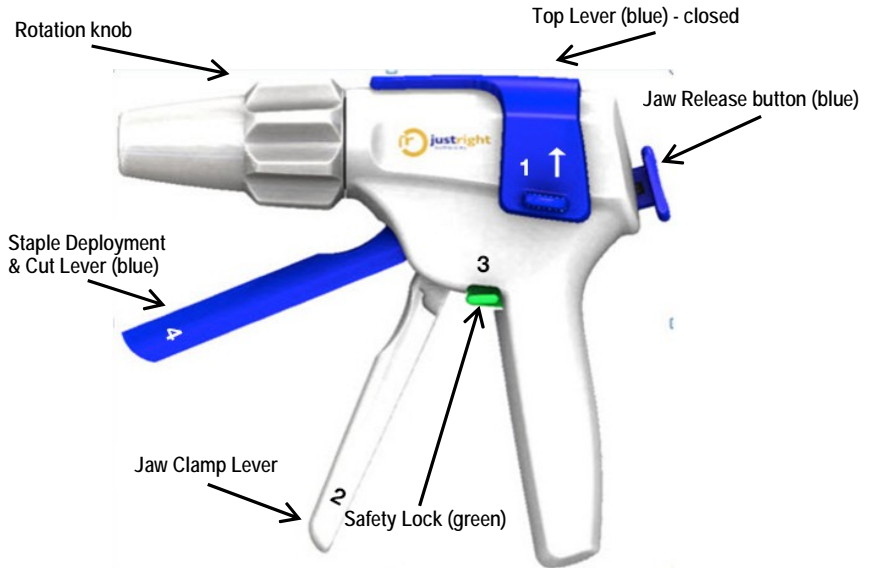
Description

The JustRight™ 5mm Stapler places two staggered rows of 2.0 mm titanium staples while simultaneously dividing the tissue between the two staple rows.

The JustRight™ 5mm Stapler is designed for introduction and use through a 5mm cannula sleeve.

Handle – JustRight™ 5mm Stapler


1. To insert the staple reload, the top blue lever must be in the closed position.

Reload – JustRight™ 5mm Reload

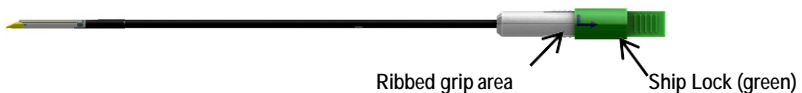2. Remove the green ship lock from the JR-REL25-2.0 staple reload by depressing the ribbed grip area on the reload while rotating the green ship lock in the direction of the arrow and pulling on it.


NOTE: Do not remove the green ship lock on the Reload until you are ready to load into the handle.

3. Align the blue dot on the reload connector with the blue dot on the handle's rotation knob and insert until fully seated.


NOTE The reload can only be inserted one way; the tabs on the reload connector are different widths and match the slots in the rotation knob.

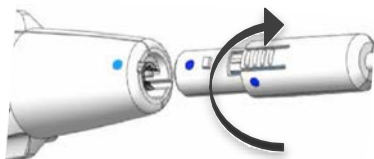4. Rotate the reload shaft clockwise until the blue dot on the outside of the reload shaft aligns with the blue dot on the rotation knob.


Using the Stapler

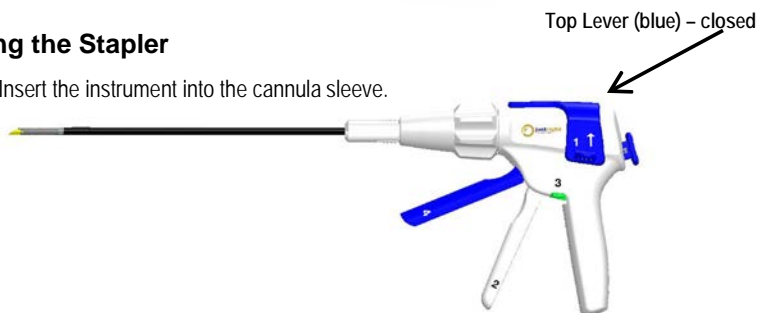1. Insert the instrument into the cannula sleeve.

NOTE: The top blue lever (1) must be in the closed position before attempting to insert the stapler into cannula sleeve.

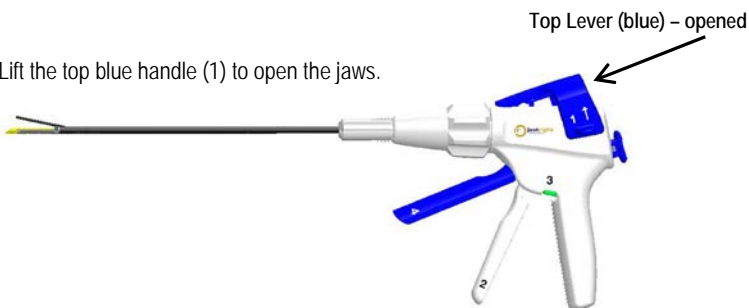2. Lift the top blue handle (1) to open the jaws.


3. Turn the rotation knob until the jaws are in the required position.

Warning

When positioning the stapler on the application site, ensure that no obstruction, such as clips, are incorporated in to the instrument jaws. Firing over an obstruction may result in incomplete cutting action and/or improperly formed staples.

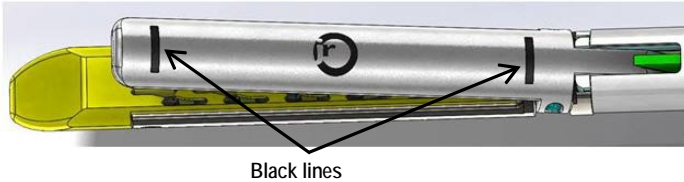4. Position the tissue to be transected between the black lines on the outside of the instrument jaws.

5. Pull back on the Jaw Clamp Lever (2) until the jaws lock on tissue to be transected.

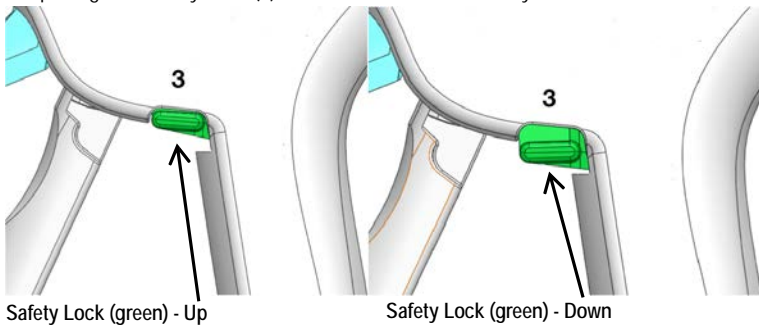
Warning

If the jaws of the device do not lock closed, the target tissue is outside the compression specifications. Reposition the jaws on to tissue within compression specification range.

6. To release and reposition jaws, depress the blue Jaw Release button (5) on the back of the handle.
7. Repeat Step 5.

NOTE: After closing, pausing for 10 seconds prior to firing may result in better tissue compression and better staple formation.

8. Flip the green Safety Lock (3) down. Do not hold the safety lock down.

9. Pull back completely on the blue Staple Deployment and Cut Lever (4) and release to resect the tissue structure.

NOTE: Only pull back on the blue Staple Deployment and Cut Lever (4) once to completely fire a staple line.

10. Press the blue Jaw Release button (5) to open the jaws.
11. Push down on the blue Top Lever (1) (closed position) and remove the device from the cannula.
12. Remove and set aside a spent staple Reload after firing.
13. Reload the device if necessary:
 - a. Remove the spent reload by depressing the ribbed grip area on the reload and rotating it until the alignment pins are disengaged, then pull the spent reload from the handle and discard it.
 - b. Remove the green ship lock from a new JR-REL25-2.0 staple reload by depressing the ribbed grip area on the reload while rotating the green ship lock in the direction of the arrow and pulling on it.
 - c. Insert a new reload by aligning the blue dot on the reload connector with the blue dot on the handle's rotation knob and inserting until fully seated.
 - d. Rotate the reload shaft clockwise until the blue dot on the outside of the reload shaft aligns with the blue dot on the rotation knob.
14. The JustRight™ 5mm Stapler may be reloaded and fired up to 15 times in a single procedure.

After Surgery

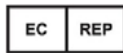
Warning
Do not reuse or resterilize instruments and accessories labeled "disposable" or "single use only".
Discard the stapler instrument after use. The instrument is not able to withstand resterilization. Do not resterilize.

Sterility is guaranteed unless the package is opened or damaged.
Do not resterilize.

JustRight Surgical®
357 S. McCaslin Blvd.
Louisville, CO 80027
Tel: 866.683.1743
Fax: 303.287.7135

Patent numbers: www.justrightsurgical.com/patents.

Healthlink Europe Services BV
De Tweeling 20-22
5215 MC 's-Hertogenbosch
The Netherlands
Tel: +31.13.5479300
Fax: +31.13.5479302

Part Number
24514
Rev G

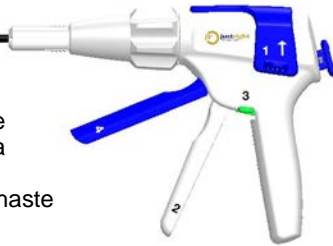
Manufactured for JustRight Surgical®
Made in USA. Printed in USA.
© 2013 JustRight Surgical®
All rights reserved.

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