

# User Reference Guide


## The ATEC® Biopsy Device Handpiece Introducer Method

The information provided here is for educational purposes only. In no way should it be considered as a replacement for the Instructions For Use (IFU).


**STEP 1**

After harvesting cores, lavage cavity prior to marker deployment. Disconnect saline Y-valve.

**Before**

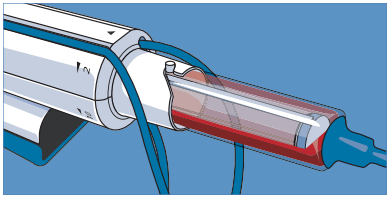


**After**

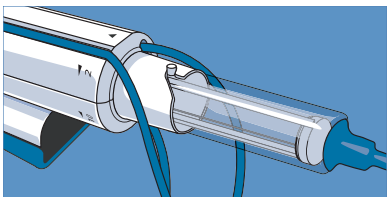


**STEP 2**

Pull back adaptor/handpiece to align marker deployment position in axial center of biopsy cavity.

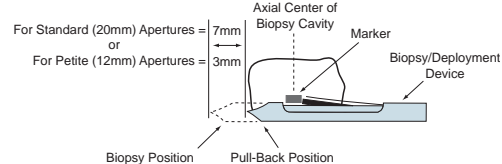

**STEP 3**

Attach deployment guide. Then insert deployment device into cannula/handpiece until light resistance is met and maintain engagement with off hand.

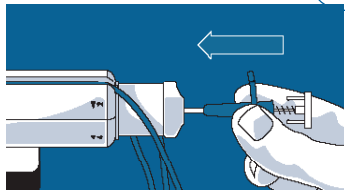


NOTE: Between steps 3 and 4, align arrow on marker with the flat of handpiece.


**STEP 4**

Press plunger all the way forward, release after audible and tactile click. Always deploy toward the center of the cavity.


**A** Pre-Deploy View

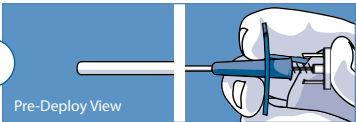

**B** Deployment View

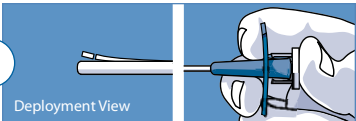

**C** Post-Deploy View

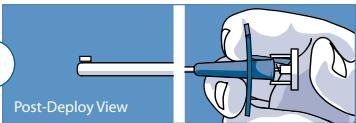


Remove your thumb off the plunger immediately after deployment


**STEP 5**

Rotate deployment device 180 degrees, then rotate cannula/handpiece 180 degrees in the opposite direction.

**A**

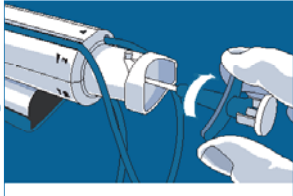


**B**

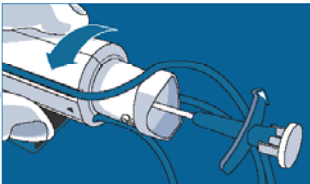

**STEP 6**

Slowly move adapter back 20mm.

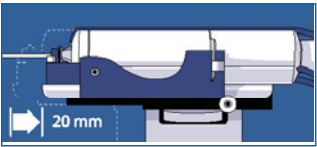

**STEP 7**

Verify marker placement.



**STEP 8**

Slowly remove handpiece/deployment device as one unit, then slowly decompress breast.

© 2008 Hologic, Inc. All Rights Reserved. ATEC, TriMark, Hologic, and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries.  
 LT378:RevE