

# User Reference Guide

The information provided here is for educational purposes only. In no way should it be considered as a replacement for the Instructions For Use (IFU).

MISC-00294-001 Rev. 002 ©2013 Hologic, Inc. All Rights Reserved. Hologic, ATEC, SecurMark, and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries.

## STEP 1


Stereotactic and Ultrasound:

After a successful biopsy,

- For standard handpieces, pull back 10mm
- For petite handpieces, pull back 6mm

## STEP 2

Stereotactic (A) and Ultrasound (B):  
Insert deployment device through back of handpiece or needle hub.

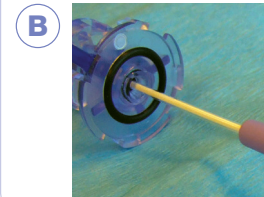

## STEP 3

**PLEASE NOTE:**


Depending on length of needle, the deployment cannula may not always be hub to hub. Do not advance deployment cannula farther than necessary.



9cm needle length,  
3-4cm showing



12cm needle length,  
1-2cm showing


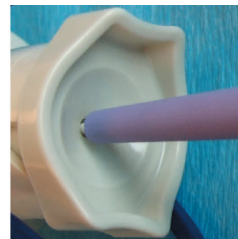


14cm needle length,  
none showing

## STEP 4

Stereotactic and Ultrasound:

Align arrows with direction of open needle aperture, and fully depress the plunger to deploy the SecurMark marker. Then release your thumb from the plunger.

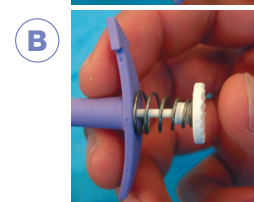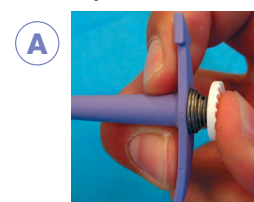

## STEP 5

Stereotactic (A) and Ultrasound (B):  
Let go of the marker and rotate the handpiece/hub 180 degrees.

