

Faxitron® Path+

Specimen Radiography System



QUICK START GUIDE

1. POWER UP

- Ensure the power switch on the back of the Faxitron Path+ cabinet is set to the **ON** position
- Turn the **key** on the front of the Faxitron Path+ cabinet to **on**
- Turn the monitor **on** (if needed)

2. START SOFTWARE

- Verify there is nothing in the Faxitron Path+ x-ray chamber
- Double click the **Vision** icon in the center of the desktop and enter password

3. CALIBRATE

- You will be prompted to calibrate once you log in
Press the **"Calibrate"** button to proceed

4. PATIENT DATA

- Start Procedure
- Select from Worklist
- Locate patient's name and double click it
- Enter any missing required fields such as Laterality
- **"Accept"**

5. X-RAY IMAGING

- Place specimen(s) in the Faxitron Path+ x-ray chamber - use magnification tray if desired
- Press the green **"X-ray"** button to activate x-rays - image will appear after a few seconds

6. IMAGE MANIPULATION

- Hold down the **left mouse button** and drag the mouse to change brightness and contrast for window/level function
- **Adjust zoom** to enlarge image for viewing

7. STORING TO PACS

- Choose **"PACS Store"** button on left side of screen

8. PRINTING

- Choose **"PACS Print"** button on left side of screen

9. SHUT DOWN

- Select the **"System Shutdown"** button on the left side and then click **"OK"**
- Turn key on Faxitron Path+ cabinet to **off**

 www.hologic.com/package-inserts



Hologic, Inc.
36 Apple Ridge Road
Danbury, CT 06810 USA
1.800.447.1856
www.hologic.com



Hologic, BV
Da Vincilaan 5
1930 Zaventem
Belgium
Tel: +32 2 711 46 80
Fax: +32 2 725 20 87

AU SPONSOR

Hologic Pty Ltd
Suite 302, Level 3
2 Lyon Park Road
Macquarie Park NSW 2113
Tel: 02 9888 8000



Faxitron Path+ system also known as PathVisionXL System.

5081-4013 Rev. 002 (8/21) Hologic Inc. ©2021 All rights reserved. Hologic, Path+, PathVisionXL, Faxitron and The Science of Sure are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and other markets and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative.