

Hologic® SuperSonic® MACH™ Ultrasound Systems Cleaning and Disinfection Products

LIST OF COMPATIBLE PRODUCTS

The table below lists the product compatibility (“YES”) with the SuperSonic MACH series probes.

Only use compatible products with the SuperSonic MACH series probes.

Solution/ system	Manufacturer	Cleaner/ Disinfectant	Active ingredient	L18-5 L10-2	MC12-3	E12-3	LV16-5	C6-1X C9-2X P5-1X LH20-6
Alkaspray GSA (Cidal- kan)	Alkapharm	Cleaner Disinfectant	Isopropyl alcohol Alkylamine	YES	YES			
Alkazyme	Alkapharm	Cleaner Disinfectant	Quat. Ammonia	YES	YES			
Anioxyde 1000	Anios	Disinfectant	Peracetic acid	YES	YES	YES		
Antigermix S1	Germitec	High-level disinfectant	UV-C	YES	YES	YES		YES
Cidex OPA (DISOPA in Japan)	Advanced Sterilization Products	High-level disinfectant	Orthophtalaldehyde	YES	YES	YES	YES	YES
Cidex Plus	Advanced Sterilization Products	Disinfectant	Orthophtalaldehyde					YES
Cidezyme	Johnson and Johnson	Cleaner	Proteolytic enzymes	YES	YES	YES	YES	
Enzol	Advanced Sterilization Products	Cleaner	Proteolytic enzymes					YES
Gigasept FF	Schuller & Mayr	Disinfectant	Succindialdehyde, dimethoxytetrahydrofuran	YES	YES	YES		YES
Hypernova Chronos	Germitec	High-level disinfectant	UV-C	YES	YES	YES		YES
KlenZyme	Steris	Cleaner	Proteolytic enzymes					YES

Solution/ system	Manufacturer	Cleaner/ Disinfectant	Active ingredient	L18-5 L10-2	MC12-3	E12-3	LV16-5	C6-1X C9-2X P5-1X LH20-6
Linget'Anios	Anios	Cleaner Disinfectant	Ethanol Digluconate de chlorhexidine Alkylaminoalkylglycine	YES	YES		YES	YES
MetriZyme	Metrex	Cleaner	Proteolytic enzymes					YES
Mikrobac Tissues	Bode	Disinfectant	Benzyl-C12-18- alkyldimethylammonium chlorides didecyldimethylammonium chloride	YES	YES		YES	
Milton	Milton	Disinfectant	Sodium Hypochlorite					YES
Sani-Cloth Active	PDI	Cleaner Disinfectant	Quat. Ammonia	YES	YES			
Sani-Cloth AF3	PDI	Cleaner Disinfectant	Quat. Ammonia	YES	YES	YES	YES	
Sani-Cloth Bleach	PDI	Cleaner Disinfectant	Sodium Hypochlorite	YES	YES	YES	YES	
Sani- Cloth HB	PDI	Disinfectant	Quat. Ammonia					YES
Sani-Cloth Plus	PDI	Disinfectant	Quat. Ammonia					YES
Sporox	Reckitt & Colman	Disinfectant	Hydrogen Peroxide					YES
Sporox II	Reckitt & Colman	Disinfectant	Hydrogen Peroxide					YES
Steranios	Anios	Disinfectant	Glutaraldehyde	YES	YES	YES		
Steranios OPA	Anios	Disinfectant	Orthophtalaldehyde					YES
Super Sani Cloth	PDI	Cleaner Disinfectant	Quat. Ammonia, isopropyl alcohol (IPA)	YES	YES	YES	YES	YES
T-Spray	Pharmaceutical Innovations Inc.	Disinfectant	Quat. Ammonia	YES	YES	YES		
Transepti spray	Parker Laboratories	Cleaner	Chlorhexidine Gluconate, Isopropyl Alcohol					YES
Tristel Duo	Tristel Solutions Limited	Disinfectant	Chlorine dioxide	YES	YES	YES		
Tristel Solo	Tristel Solutions Limited	Disinfectant	Hexamethylenebiguanide	YES	YES			
Trophon EPR	Nanosonics	High-level disinfectant	Hydrogen peroxide mist generated in a disinfection chamber	YES		YES		YES
Trophon 2	Nanosonics	High-level disinfectant	Hydrogen peroxide mist generated in a disinfection chamber	YES		YES		YES
Viraclean	Whiteley Medical	Cleaner Disinfectant	Ethanol	YES	YES	YES	YES	YES
Wavicide 01	MCC	Disinfectant	Glutaraldehyde					YES

For endocavity probes, make sure the cleaner/disinfectant is compatible with mucosa.



Please note: the above table can also be found in the SuperSonic MACH user guide AW-27955_002_01 & BUS-01596. The user guide is in the process of being updated to reflect the above probe disinfectant compatibility. Please use this table as the most up to date guidance for probe disinfection as of 5/2/23.

MISC-08770 Rev 002 5/23 © 2023 Hologic Inc., All rights reserved. Hologic, SuperSonic, MACH, ShearWave, Ultrafast, The Science of Sure and associated logos are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and other countries. This is a general information tool for medical professionals and is not a complete representation of the product(s)' Instruction for Use (IFU) or Package Insert, and it is the medical professionals' responsibility to read and follow the IFU or Package Insert. The information provided may suggest a particular technique or protocol however it is the sole responsibility of the medical professional to determine which technique or protocol is appropriate. At all times, clinicians remain responsible for utilizing sound patient evaluation and selection practices, and for complying with applicable local, state, and federal rules and regulations regarding accreditation, anesthesia, reimbursement, and all other aspects of inoffice procedures. In no event shall Hologic be liable for damages of any kind resulting from your use of the information presented.