



Date: August 7, 2024

Author: Software Engineering

System: SecurView

Subsystem: Windows 10 LTSC 2019 (v1809) & Software Security Patches

Subject: Windows 10 LTSC 2019 (v1809) & Software Security Patch release July 2024

Purpose:

To release the list of validated Microsoft critical patches on the SecurView System.

Note: Hologic would like to bring attention to the following known issue statement released by Microsoft related to the Windows cumulative security patch released on July 9, 2024. Devices that have been configured for BitLocker encryption, particularly those configured for Device Encryption, may require entering the BitLocker recovery key during restart of Windows after installing the latest security patch. If the recovery key is requested by Windows, the device will not be able to start until the key is provided. Standard BitLocker (not Device Encryption) is an optional configuration that may be enabled on Hologic SecurView and Horizon product devices.

BEFORE installing patches to a BitLocker enabled device, please ensure that you have access to the required BitLocker recovery key. For SecurView devices, BitLocker recovery key(s) can be found within E:\BitLocker and should also be stored securely, off device, by the Health Delivery Organization (HDO). For Horizon devices, a USB flash drive should have been created during the encryption process. If the BitLocker recovery key cannot be located, contact Hologic Technical Support for assistance.

Microsoft Known Issue Statement

After installing the July 9, 2024, Windows security update, you might see a BitLocker recovery screen upon starting up your device. This screen does not commonly appear after a Windows update. You are more likely to face this issue if you have the Device Encryption option enabled in Settings under Privacy & Security > Device encryption. Because of this issue, you might be prompted to enter the recovery key from your Microsoft account to unlock your drive.

List of patches/KB's validated for July 2024:

KB5040430 KB5039879

List of all patches/KB's validated on the SecurView System:

KB890830	KB5029247	KB5013941
KB5040430	KB5028168	KB5013641
KB5039879	KB5027222	KB5012647
KB5039217	KB5027131	KB5012170
KB5037765	KB5026362	KB5012128
KB5036896	KB5025229	KB5011503
KB5036604	KB5023702	KB5010351
KB5035849	KB5022840	KB5009557
KB5034768	KB5022511	KB5008873
KB5034127	KB5022286	KB5008218
KB5034119	KB5021237	KB5007206
KB5033904	KB5021126	KB5005698
KB5033371	KB5020866	KB5005112
KB5032391	KB5020627	KB4589210
KB5032196	KB5019966	KB4589208
KB5031984	KB5018419	KB4577586
KB5031361	KB5017315	KB4535680
KB5030214	KB5016623	KB4132216
KB5029931	KB5015811	
KB5029376	KB5014692	